

Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

Reorder No. 10261

LOT



SYMBOL GLOSSARY
For an explanation of symbols
used in Dynarex packaging, visit
dynarex.com/symbols.php



Urinary Drainage Bag Holder



Designed to hang from wheelchairs, geriatric chairs, or bed rails and fits most sized drainage bags. Discreetly hides urinary drainage bag from public view. Made from durable vinyl that cleans easily with soap and water, and is heat sealed to contain spills. Air dry, do not launder or place in a dryer.

INSTRUCTIONS FOR USE:

1. Feed each strap around chair frame or bed rail and through plastic on edge of bag. Adjust to preferred height.
2. Lock in place by pressing strap together.
3. Place urinary bag in holder.
4. Store and protect excess tubing in holder.



6 16784 02611 4

R180207

1

11" × 11"
(28 cm × 28 cm)