

Rescue™ Liquid Sporicidal

Ready-to-Use sporicide for hard, non-porous surfaces that is effective against *Clostridium difficile, Clostridium sporogenes* and *Bacillus subtilis,* utilizing AHP[®] technology.

Features & Benefits

- Ready-to-Use sporicide powered by AHP® technology
- Effective against *Clostridium difficile, Clostridium sporogenes, Bacillus subtilis* in 10 minutes, providing cleaning and disinfection on high touch surfaces
- The $\,AHP^{\scriptscriptstyle \otimes}$ active breaks down to oxygen and water after use

Applications

• For use on hard, nonporous surfaces such as floors, walls, porcelain, and plastic surfaces in hospitals, nursing homes, schools, colleges, hotels and public areas





Rescue[™] Liquid Sporicidal



Use instructions for Hard Non-Porous Surfaces*

Heavily Soiled Surfaces Require Cleaning Prior to Disinfection

Personal Protection: Disposable gloves, gowns, face masks, or eye coverings as appropriate, must be worn during all cleaning of body fluids, and decontamination procedures.

Cleaning Surfaces Contaminated with Body Fluids: Appropriate Personal Protection must be worn during all cleaning and disinfection procedures. Remove excess body fluids with absorbent materials. Apply the liquid to surface using a cloth or disposable wipe, let stand for 1 minute, thoroughly rinse. Disinfect contaminated area.

Sporicidal Disinfection of Hard Non-porous Surfaces: This product is NOT to be used as a terminal sterilant/High Level Disinfectant for critical instruments and medical devices. This product is intended for use as an Environmental Surface Sporicide on hard non-porous surfaces in *C. difficile* patient rooms.

- Apply liquid to surfaces using a cloth or disposable wipe
- To achieve Sporicidal disinfection ensure surface remains wet for 10 minutes at 20°C
- Rinse after disinfection to remove all residue

Notice to User: This product is not intended for use on medical equipment without device manufacturer approval.

Disposal of Infectious Material: Products contaminated with blood or body fluids should be disposed of according to Federal, Provincial, and local regulations for infectious waste disposal.

* Refer to product label for complete use instructions.

Technical data	Rescue™ Liquid Sporicidal		
DIN	02447606		
Color/Form	Colorless Liquid/Gel		
рН	2.7		
Scent	Characteristic of ingredients		
Shelf Life	2 Years		

Product	Pack size	Dilution	Product code	
Rescue™ Liquid Sporicidal	12 x 946 mL / 1 U.S. Qt. Bottles	Ready-to-use	101103667	1+1

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.