

## Connecting your LIFEPAK CR2 cellular device

Refer to: Getting Started Guide and Wireless Setup Guide that comes with device

1. Take AED to its intended location.
2. Insert battery and wait for self-test to complete. May take up to 2 minutes. **Wait until 3 audible tones sound before proceeding to next step.** (If the same battery has been previously inserted, the 3 tones will not sound.)
3. Verify the **Readiness indicator is flashing green** to indicate readiness (flashes every 6 seconds).
4. **Open the lid and wait** until the voice prompts start.
5. Immediately **press and hold the LANGUAGE and CHILD MODE buttons** simultaneously to complete a self-test until you hear “**Device ready.**” After the voice prompts start, you must press the 2 buttons within 10 seconds.
6. **Immediately close the lid** and the AED will attempt a wireless connection.
7. Wi-Fi will try to be connected first. Let the Wi-Fi connection fail. You will hear some prompts, pauses, and tones during this process.
8. Once the Wi-Fi connection fails, **cellular connection will begin.** Once device is connected to a cellular connection, you will hear these prompts:
  - a) Mobile connection initiated
  - b) Connection in progress
  - c) Connection established
  - d) Communication complete
  - e) Powering off
9. LIFEPAK CR2 cellular device will be updated in LIFELINKcentral.