


How to Use ENFit™ Transition Connector

INDICATIONS FOR USE: The ENFit™ Transition Connector with protective cap is pre-attached to ENFit™ connection. Unscrew to remove ENFit™ Transition Connector if being used with a gastroonomy tube with ENFit™ connection.


DO NOT REUSE: For single use only, there is potential risk of patient or user infection. Do NOT use if package is open or damaged.

DIRECTIONS:

1. Remove protective cap from ENFit™ Transition Connector. (The ENFit™ end of ENFit™ Transition Connector is attached to ENFit™ enteral feeding set.)
2. Attach the stepped distal tip to the funnel port of gastroonomy feeding tube. (Remove ENFit™ Transition Connector if Enteral Feeding Device is already equipped with an ENFit™ connection.)
3. Proceed with feeding as directed.
4. After feeding is complete, discard ENFit™ Transition Connector together with the enteral feeding set within a maximum of 24 hours.

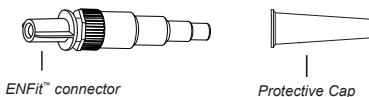

ENFit™ Transition Connector

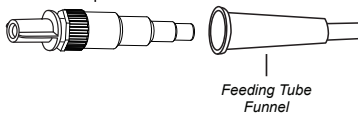

Feeding Tube Funnel



Securely Fit

(Be sure to check all your connections are firmly secure.)

WARNINGS: Attach the ENFit™ Transition Connector to connect enteral feeding systems ONLY if an enteral feeding device equipped with the ENFit™ connector is not available. The ENFit™ Transition Connector does not protect against inadvertent misconnections with connectors of non-enteral applications. Serious injury or death is possible in the event of a misconnection. Due to the risk of misconnections, always confirm prior to use that feeding set with ENFit™ Transition Connector is connected to an enteral feeding device. Not for intravenous use. Do NOT use for more than 24 hours.

ENFit™ is a registered trademark of GEDSA used with their permission

Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com
Made in China R180913

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

