


Bari+Max dynarex®
Support Where You Need It

Bariatric HD Overbed Table Cherry



ASSEMBLY INSTRUCTIONS

TOOLS NEEDED FOR ASSEMBLY:
Phillips screwdriver • 5/8" wrench

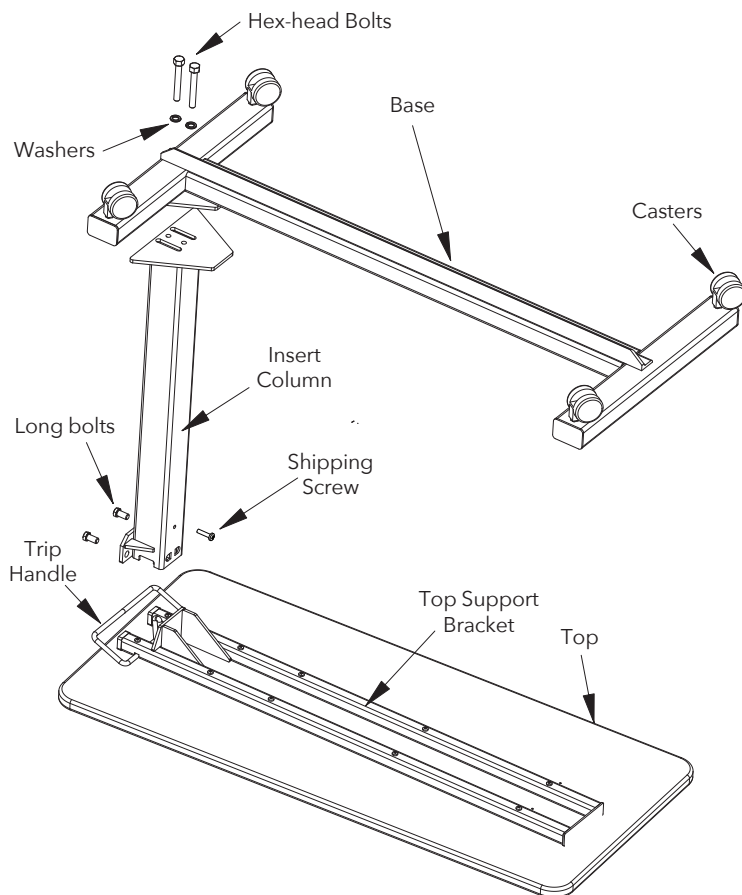

1. Position the Table Top on a solid, unobstructed surface with Top Support Bracket as illustrated (See Fig. 1).
2. Raise Trip Handle to an angle of approximately 40 degrees. Next, insert Column into the Top Support Bracket, ensuring that the rectangular notches are aligned with the Top Bracket Embossing (See Fig. 2). Ensure that the Trip Handle tab is in the Trip Channel in the Column (See Fig. 3). Attach securely using 3/4" Long Bolts and Lock Washers.
3. Place the Base on a solid, unobstructed surface and attach the Casters.
4. Attach Base to the Column with Casters pointing upward. Tighten the Hex-Head Bolts through the Lock Washers and Base).
5. Set the table in an upright position on its Casters.
6. Remove the Shipping Screw from the front of Column and dispose of the screw. **CAUTION: DO NOT REMOVE THE SHIPPING SCREW BEFORE COMPLETING ASSEMBLY STEPS 1- 5. This can result in personal injury and spring damage.**

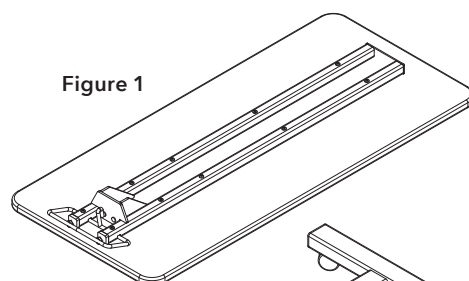


Figure 1

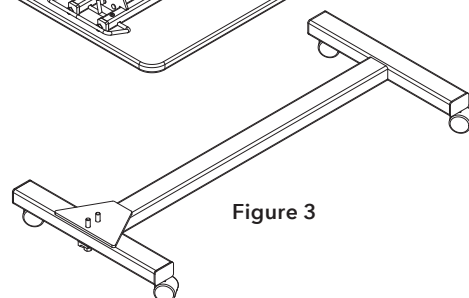


Figure 3

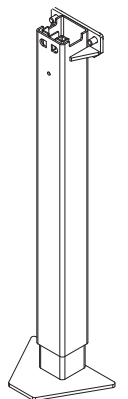


Figure 2

CAUTION: SPRING LOADED ASSEMBLY

Do not remove the shipping screw before completing assembly steps 1- 5.

Removing the shipping screw before this can result in personal injury and spring damage.

HEIGHT ADJUSTMENT INSTRUCTIONS

TO RAISE TABLE: Push upward on the underside of the Table Top to desired height.

TO LOWER TABLE: Squeeze the Trip Handle to release the Column and push down on the Table Top to desired height.

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments

- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

*This product meets or exceeds all federal safety and environmental regulations.



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov