
Items 3683-3684



High Strength Pressure Bandage

Tourniquet

Instructions for use

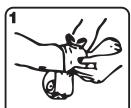

Apply dressing pad to wound and wrap elastic bandage around limb.



Draw bandage through pressure applicator to tighten firmly.



Wrap bandage over pressure applicator, overlapping pad edges.



Slip elastic bandage through V-notch of pressure applicator.



Fold bandage back over applicator to apply pressure to wound.



Designed for a fast response to severe bleeding and trauma situations, the High Strength Pressure Bandage (tourniquet) combines a thick non-adherent dressing, elastic bandage, pressure applicator and closure clip in one simple, sterile unit. The bandage is also a tourniquet that gives you an effective wound dressing that is quick and easy to apply without the need for tape, safety pins or other bandages.

- Store in a cool, dry place, out of direct sunlight.
- Do not use if packaging is open or damaged.

STERILE R () Not made with natural rubber latex

Manufactured for: Dynarex Corporation 11 Dynarex Drive • Middletown, NY 10941 USA • www.dynarex.com

Symbol Glossary: dynarex.com/symbols.php Made in China R231128