Backup Accumulator Accessory for Etude Plus HC

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I General

I.1 About this manual

This manual contains important information about assembly and maintenance of an accessory. Read the manual carefully and follow the safety instructions. For more information about the product, read the user manual or service manual of the product. For additional information about the accessory, please contact the manufacturer.

Symbols in this manual

In this user manual, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

WARNING!

Risk of injury or death

– Do not perform service or maintenance on any electrical parts unless properly trained.

2 Assembly

2.1 Installation of backup accumulator

Risk of damage to property

The motors might fall while being removed.

– Make sure to hold the motor with a firm grip, when pulling out the pipe pin.

1. Lift the backrest section up to reach the control box
2. Remove the leg rest motor plug from the control box.
- 3.

Fold out the clamp and pull out the pipe pin.


4. Remove the motor from the bed.

DEALER: Keep this manual.
The procedures in this manual MUST be performed by a qualified technician.


Yes, you can.®

5.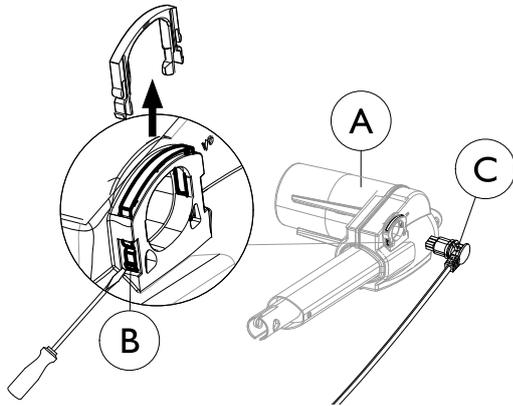

Place the motor (A) on a steady surface. Release the locking cam by pressing in the pin (B) with a screw driver. Pull the locking cam up and remove it. Then pull out the plug (C) from the motor.


6.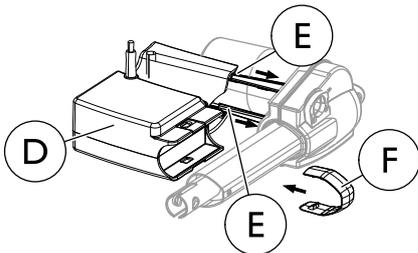

Fit the battery backup (D) over the rails (E) on the motor and slide it all the way in to a stop.

7. Join the parts together properly by attaching the locking cam (F) over the motor and onto the battery backup. Make sure it engages with a clicking sound.

8.

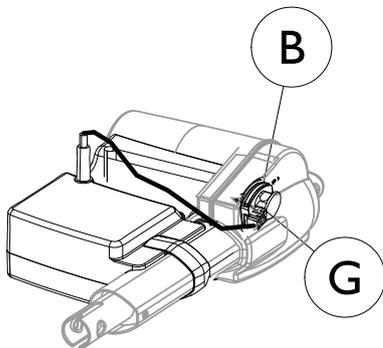Connect the plug (G) (from the accumulator backup) and lock it by pressing the locking cam (B) down.

9.

Push the cotter pin through motor and attachment. Fold the clamp over the pin. The motor is now secured.

10. Lift the backrest section up to reach the control box

11. Connect the accumulator wire to the control box.

12. Make sure the wires are not squeezed after the motor have been installed.

2.2 Charging the batteries

1. Connect the control unit to the mains.

- The control box must be connected to the mains for 24 hours prior to first use.
- Accumulators must be charged, at least, every 6 months for maintenance purposes.

2

3 Maintenance

3.1 Maintenance

IMPORTANT!

Low accumulator capacity

- When the accumulator only has capacity left for one more lifting cycle, the control box will make an audible sound, when using the hand control.

WARNING!

Risk of explosion

Old or faulty accumulators may generate an explosive gas mixture during charging.

- The accumulator box is provided with vents to ensure adequate ventilation of the box. Do not block or cover the vents, as this may result in pressure built-up and the risk of explosion.

Preventive maintenance

Frequent, sudden discharges reduce the backup's life.

We recommend the accumulator be tested at least once a year.

Accumulators are not damaged by continuous connection to the mains.

Exchanging the accumulators

- Accumulators have to be exchanged after 4 years. Depending on how the backup is used the exchange may have to take place earlier.
- Accumulators must be replaced by a new complete kit from Invacare.
- When exchanging the battery back up - dismount the battery backup in reverse order as described in 2.1 Installation of backup accumulator.

Prior to insertion make sure that the accumulator set is correctly connected according to the drawing (in the accumulator box), and that none of the connections are loose.

Cleaning

WARNING!

Risk of Injury or Death

Electrical components must always be cleaned according to instructions.

- For cleaning instructions, follow instructions for Electrical components, in main manual for the bed.

4 Technical data

4.1 Backup Accumulator

For full details, also read the documentation from the manufacturer. Download manual(s) from www.linak.com or contact your Invacare dealer.

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