



1st Healthcare Kit

IH020, IH173, IH210, IH217, IH224, IH231, IH246, IH247, IH342


Reversible Case

The reversible case keeps all of baby's grooming items in one convenient place. In the cube shape, this case has its greatest capacity for bulky items. Reversed into the flat, pencil-case shape, it still holds all of the kit components but fits more compactly into a purse, diaper bag or suitcase.

To reverse from cube to pencil-case: Turn unzipped case inside-out and gently pull on ends, to straighten and flatten the case. Bring zipper pull to the outside of the case.

To reverse from pencil-case to cube: Turn unzipped case inside-out and gently re-shape ends, to "square off" and form the ends of the cube. Bring zipper pull to the outside of the case.

Use and Care: Spot clean. Wipe with a damp cloth.

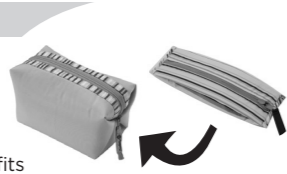

Medicine Dropper

Safety 1st understands that giving a child medicine takes special attention. Our Medicine Dropper is specially designed to provide accurate dosage readings. The convenient spill guard plug helps reduce messy spills at home or away.

To Use:

- 1 Remove spill guard from dropper tip and squeeze air out of dropper.
- 2 Insert dropper into the medicine bottle and slowly release pressure on the bulb until the medicine reaches the desired dosage marking.
- 3 Remove dropper from the medicine bottle and tip upside down while securing the spill guard over the dropper tip.
- 4 Administer medicine while child is in an upright sitting position. Remove spill guard and point tip of dropper to the inside of baby's cheek and gently squeeze the bulb.
- 5 Administer medicine slowly enough so your child may swallow comfortably. Pause and continue if necessary.

To Clean:

Wash with warm, soapy water before first use and after each use. Rinse thoroughly.

CAUTION: KEEP ALL MEDICATIONS OUT OF CHILD'S REACH. WE RECOMMEND THAT YOU USE A CHILD RESISTANT TOP TO SEAL THE MEDICINE BOTTLE AND STORE IT IN A LOCKED CABINET OR OTHER LOCATION THAT IS NOT ACCESSIBLE TO A CHILD.



Newborn Nasal Aspirator

Our Newborn Nasal Aspirator quickly and conveniently removes excess mucus so your baby can breathe comfortably again.

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean:

Clean before first use and after each use. Gently squeeze air from Aspirator. Rinse the bulb in warm, soapy water. Squeeze and release bulb several times to thoroughly wash interior of Aspirator. Rinse in the same fashion with warm, clean water. Make certain that all of the water is drained during drying. DO NOT place in microwave.

CAUTION: NEVER PUSH THE ASPIRATOR TOO FAR INTO THE NOSTRIL AND ALWAYS DRY THOROUGHLY AFTER EACH USE. NEVER USE IN BABY'S MOUTH OR EARS. WHEN STORING, KEEP AWAY FROM HEAT AND BRIGHT LIGHT.



Soft Grip Toothbrush

The contoured, Soft Grip handle makes it easy for small hands to grip the toothbrush. Its soft, fine bristles are especially gentle on your toddler's sensitive teeth and gums. Place it on the table or sink and the convenient, roll-resistant handle keeps it from falling off.

Your child may begin to use the toothbrush once all primary teeth are in or when recommended by your dentist. With a pea-sized amount of toothpaste, let your child brush for 2-3 minutes using an up and down motion. Continue to help and supervise your child during brushing – do not allow child to swallow toothpaste. **NOTE:** Toothbrushes should be replaced every 2-3 months.

To Clean:

Rinse brush thoroughly before first use and after each use.

Important Note: Questions about your child's oral health should be answered by your family dentist.

CAUTION: ADULT SUPERVISION REQUIRED. DO NOT ALLOW CHILD TO RUN OR WALK WHILE CARRYING OR USING TOOTHBRUSH. DO NOT LET CHILDREN PLAY WITH TOOTHBRUSHES.



5 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.



FOR EXTERNAL USE ONLY.

Emergency Information Card

Record important phone numbers and medical information for quick reference.

CAUTION:

THESE PRODUCTS ARE NOT TOYS. Do not allow children to play with them. To prevent accidental injury, your child should never be left unattended. Products should only be used with adult supervision. When not in use, keep out of reach of children. FOR INDOOR HOME USE ONLY.

©2018 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. Safety 1st™ y el logotipo de Safety 1st y su diseño son marcas comerciales de Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com. Styles and colors may vary. Los estilos y los colores pueden variar. Made in CHINA. Hecho en CHINA. Distributed by (distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2855 Argentia Road, Unit 4, Mississauga, ON L5N 8G6. Importado y Distribuido en Chile por: Comercial e Industrial Silfa S.A. - San Ignacio 0201 / Parque Industrial Portezuelo Quilicura, Santiago Chile. Fono (562) 2339 9000. En Perú por: Comexa S.A. - Los Libertadores 455 - San Isidro, Lima - Perú. Fono (511) 4469014. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Via 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle, Benito Juárez C.P. 03100 México, Ciudad de México R.F.C. DJG140305GN9 Tel. (55) 67199202 05/23/18 4358-5237D



Digital Thermometer with Case

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.

NOTE: Fahrenheit only (except for IH246 & IH247 which contain Celsius only thermometers).



WARNING:

BEFORE USING DIGITAL THERMOMETER, PLEASE READ CAREFULLY.

- **POTENTIAL CHOKING HAZARD.**
- This is **NOT** an ear thermometer. **NEVER** put probe in ear. Take temperature measurements Underarm, Orally or Rectally.
- **NEVER** allow a child to play with this thermometer. It is not a toy. It is a sensitive electronic device and may be damaged if not used properly under parental supervision.
- **ALWAYS** supervise your child when using thermometer. **NEVER** allow your child to use it alone or without close parental supervision.

WARNING:

- Keep battery out of child's reach. Swallowing battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode.
- Used batteries should be disposed of properly. Do not dispose of batteries in fire, recharge, put in backwards, disassemble, mix with used or other battery types – they may explode or leak and cause injury.

BEFORE USING THE DIGITAL THERMOMETER, PLEASE READ CAREFULLY:

- Avoid sudden impact or dust.
- Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock.
- Do not chew on or bend the thermometer.
- Make sure the thermometer has been at room temperature for 15 minutes before use.

Important Information:

- High, prolonged fever requires medical attention especially for young children. Please contact your physician.
- Do not allow children to take their temperatures unattended.
- Read the enclosed instructions carefully to ensure accurate temperature readings. Temperature readings and measurement times are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.
- Do not allow children to walk or run during temperature taking. The probe could present a hazard.

How To Care For This Product:

- Use isopropyl alcohol or soap and water to clean thermometer tip and probe after each use.
- Thermometer is water-resistant (not waterproof) for easy cleaning.
- Store the unit in the protective case while not in use.
- Do not store or keep in direct sunlight, or in high temperature areas.
- Do not drop the thermometer, as this may ruin the unit.

What Is A Normal Temperature?

Temperature readings vary in different individuals and also vary based upon the body location from which the temperature is measured. A rectal temperature tends to be approximately 1° higher than an oral temperature and 2° higher than an underarm temperature for the same individual.

"Normal" temperature refers to each individual's average temperature while they are well. You may want to measure and record your baby's temperature a few times when well, so that you will have a baseline of what is normal for your own child. On average, temperature ranges are as follows:


Temperature Method	Normal Range	Fever Range
Rectal:	98° to 100.3°	100.4°F +
Oral:	97° to 99.5°	99.6°F +
Underarm:	96° to 98.5°	98.6°F +

How To Take Temperature:

- 1 Select rectal, oral or underarm as your measurement site based on the individual requiring a temperature measurement.
- 2 Push the on-off button located on the cap. The thermometer display will read: "188.8 °F". This is a function check indicating the unit is working properly. The thermometer will read L°F and begin measuring temperature when exposed to body heat.
- 3 The °F symbol will continuously flash until completion of the temperature taking process. Please note that the "▼" indicates a low battery. An "ERR" message in the window indicates your unit is not functioning properly. Do not expect an accurate temperature reading if this low battery indicator is present.
- 4 When peak temperature is reached, the °F symbol will stop flashing.
- 5 Read and record the temperature and time for reference.
- 6 Push the on/off button to turn the unit off. If you do not turn the unit off, it will shut off automatically in approximately 10 minutes.
- 7 Clean the thermometer tip and probe as recommended.

Rectal Use: (For babies and young children)

- Lubricate the probe tip with a water-soluble lubricant or petroleum jelly.
- Gently insert the probe tip no more than 1/2 inch or 1.3 cm into the rectum. Do not force the tip into the rectum if resistance is encountered.
- Peak temperature should be reached in approximately 60 seconds.
- To purchase thermometer covers (optional), call Consumer Care at (800) 544-1108, and ask for part number 49520.

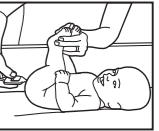


Oral Use: (For children 4 years and older)

- Do not drink hot or cold fluids, exercise or perform other activities that will raise or lower temperature readings when compared to your normal, average temperature. The mouth should remain closed up to 5 minutes before attempting a reading.
- Place the probe tip well under the tongue as indicated by the ✓ in the diagram.
- Instruct children not to bite down on the probe.
- The peak temperature should be reached in approximately 60 seconds. Opening the mouth could result in a longer time for a reading.



Underarm (Axillary) Use: (For child who will not cooperate for oral or rectal temp)

- Wipe underarm with a dry towel.
- Place probe tip in armpit and bring arm down against the body. The thermometer should be pointing in the same direction as your body if possible.
- The time required for an accurate underarm temperature may be 90 seconds or longer depending on placement of the probe and closure of the arm against the body.

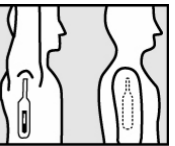


Battery Information:

- Long life, non-replaceable, mercury free battery (approximately 6000 temperatures).
- Automatic switch off after approximately 10 minutes for longer battery life.
- Low battery indicator "▼" appears in the lower right of the display when the battery is exhausted.

Dispose of the thermometer as the battery is non-replaceable. Be aware of any local regulations concerning disposal of this product. The battery is well sealed inside the thermometer to reduce the risk of being swallowed by children.

Measurement Range: 89.6°F to 111.2°F.

Measurement Accuracy: 0.2°F between 93.2°F to 107.6°F at an ambient temperature of 64.4°F to 82.4°F.

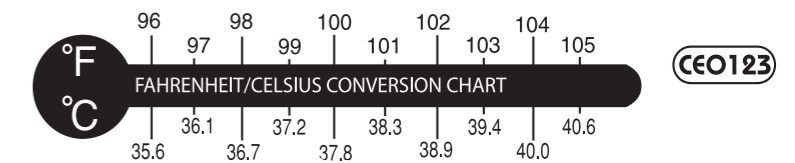

One Year Limited Warranty for Digital Thermometer

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Digital Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on front page and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or inconsequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

