



Accel PREvention™ Concentrate

48

Concentrated One-Step Surface Cleaner & Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A broad spectrum, intermediate level concentrated disinfectant based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus and Poliovirus, providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 5 minutes at 1:40. It is also a 30-second broad-spectrum sanitizer at 1:128
- Compatible with most hard, non-porous surfaces while harnessing the power of AHP® technology. The AHP® active breaks down to oxygen and water after use.
- Best possible safety rating in dilution, and non-irritating to users

Applications

- Clean and disinfect non-critical medical devices, equipment and non-porous hard surfaces





Accel PREvention™ Concentrate

Concentrated One-Step Surface Cleaner & Disinfectant

Use instructions

Preparation of Use-Solution

| Ratio | Per Liter of Water | Per 32 oz. of Water |
|-------|--------------------|---------------------|
| 1:40 | 25 mL | 0.8 oz. |
| 1:128 | 8 mL | 0.25 oz. (1/4 oz) |
| 1:256 | 4 mL | 0.125 oz. (1/8 oz) |

For use as a One-Step Surface Cleaner & Disinfectant

- Heavily soiled surfaces require cleaning prior to disinfection.
- Dilute 1:40 and apply to surface with cloth or disposable wipe.
- Let treated surface remain wet for 5 minutes.
- Air dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.



For use as a Non-Food Contact Sanitizer

- Dilute 1:128 and apply to surface.
- Allow to remain wet for 30 seconds.
- Wipe surface and let air dry.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

* Refer to product label for complete use instructions

| Technical data | Accel PREvention™ Concentrate |
|----------------|--------------------------------------------------|
| DIN | 02448629 |
| Certifications | EcoLogo |
| Color/Form | Colorless, liquid |
| pH | 0.45 (Concentrate) 2.2 (Use Dilution 1:40) |
| Scent | Characteristic of ingredients |
| Shelf Life | 2 years (Concentrate), 30 days (Use Dilution) |

| Product | Pack size | Dilution | Product code |
|-------------------------------|-------------------------------------|-------------|-------------------------------------------------------------------------------------------------|
| Accel PREvention™ Concentrate | 4 X 3.78 L / 1 U.S. Gal. Containers | 1:40, 1:128 | 101103668  |
| Accel PREvention™ Concentrate | 2 x 1.5 L RTD® / 50.7 oz. | 1:40, 1:128 | 100940209  |

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.