

Reorder No. 10712

# Bari+Max<sup>dynarex</sup>

Support Where You Need It

## Bariatric Trapeze Bar with Stand



 Manufactured for: Dynarex Corporation  
10 Glenshaw Street | Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com) | Made in China

Congratulations on your purchase of our Bariatric Trapeze Bar with Stand! Our trapeze bar is engineered and designed with durability and easy maintenance in mind.


The trapeze bar will allow patients to lift themselves while in bed.

Please read through this manual and get acquainted with your Bariatric Trapeze Bar with Stand. Follow the maintenance schedule to ensure proper functioning of your unit.

**SAFETY PRECAUTIONS**

Do not operate your Bariatric Trapeze Bar with Stand until you read and fully understand this manual.

- Do not operate your trapeze bar until it is fully assembled and checked.
- Do not exceed the maximum weight capacity (1000 lb.) of the Bariatric Trapeze Bar with Stand.
- Do not modify or replace with parts other than those manufactured by Dynarex.

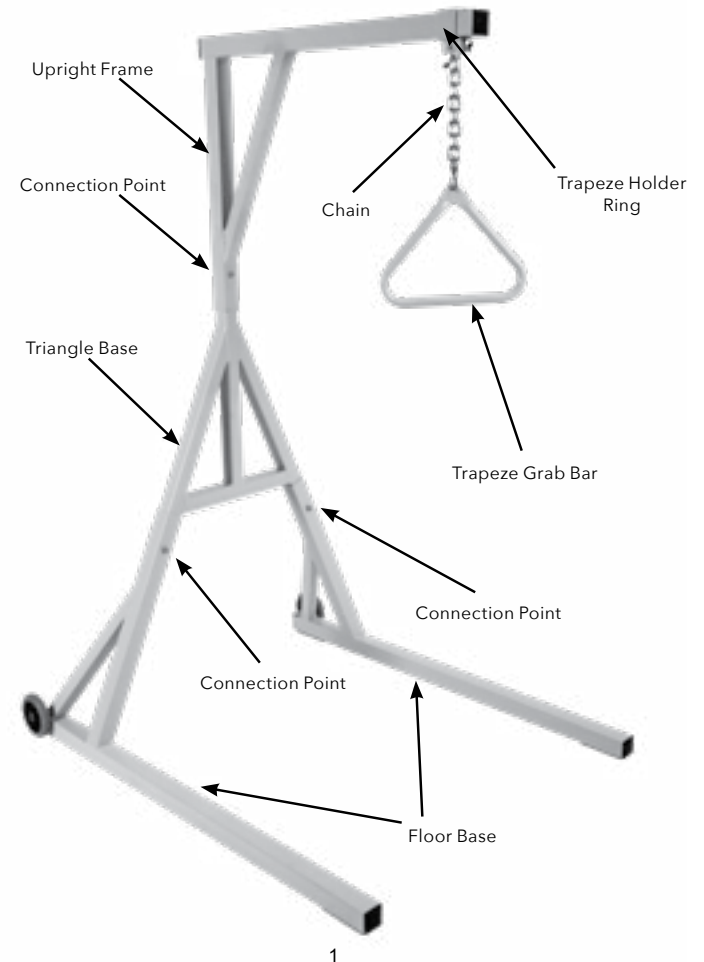

**ASSEMBLY INSTRUCTIONS**

A. Please carefully unpack the container and remove the Bariatric Trapeze Bar with Stand from the shipping carton.

B. Inspect the packaging, and make sure that there are no shipping damages and all the components are in the shipping carton.

2 Floor Bases

1 Triangle Frame

1 Upright Frame

1 Trapeze Grab Bar (with chain)

3 Complete Screw-Washer-Nut-Bolt sets

C. Assistance is needed and highly recommended in completing the final assembly due to heavy weight of parts.

**ASSEMBLY INSTRUCTIONS**

1. Insert Triangle Frame to the Floor Base (see figures 1 and 2).

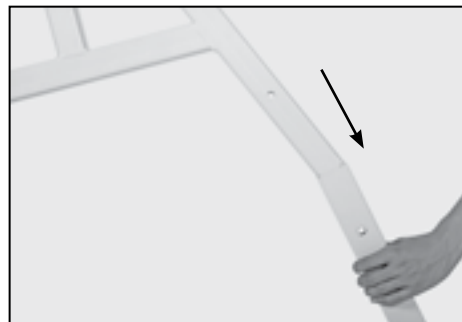


Figure 1



Figure 2

2. Insert screw, ensuring it goes through all four holes and is extended through the opening. Add the washer (figure 3) and tighten the nut (figure 4).



Figure 3



Figure 4

3. Repeat step 1 to 2 for the other floor base.

**ASSEMBLY INSTRUCTIONS**

4. Insert Upright Frame to the top of the Triangle Base (figure 5).

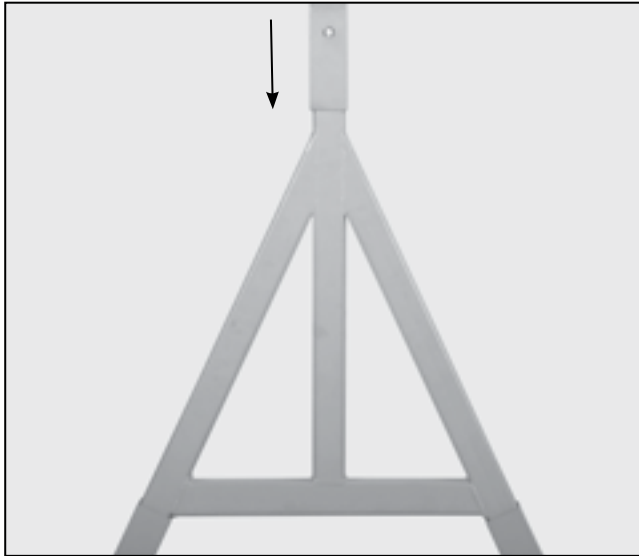


Figure 5

5. Insert screw, ensuring it goes through all four holes and is extended through the opening. Add the washer and tighten the nut.

**ASSEMBLY INSTRUCTIONS**

6. Slide trapeze holder ring over the tube. Insert thumb screw through ring and turn clockwise until tight to secure in place (figure 6).



Figure 6

**MAINTENANCE**

1. Clean the Bariatric Trapeze Bar with Stand regularly with non-abrasive cleaners.
2. Check monthly that all screws are tight and have not loosened.
3. Make sure all rubber tips and ends are in good condition for use.

**LIMITED LIFETIME WARRANTY**

Your Dynarex Product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments

- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

**SYMBOL GLOSSARY**

For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)