


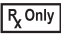












Prefilled Sterile Water for Inhalation, USP

(Nebulizer)

These instructions contain important information for safe use of the product. Read the entire contents of these Instructions For Use, including Warnings and Cautions, before using this product. Failure to properly follow warnings, cautions and instructions could result in death or serious injury to the patient and/or clinician.

NOTE: DISTRIBUTE THIS INSERT SHEET TO ALL PRODUCT LOCATIONS.

1. **DESCRIPTION:** Disposable, single patient use respiratory therapy bottle. The internal contents of the bottle are sterile unless opened or damaged.
2. **INDICATIONS FOR USE:** For single patient use in respiratory therapy.
3. **CONTRAINDICATIONS:** None known.
4. **WARNINGS:**
 - 4.1 These products are intended for inhalation only, not for parenteral use or for injections or in preparations to be used for injection. Incorrect use can result in patient injury.
 - 4.2 This product is intended for single patient use. Discard unused solution. Failure to do so may result in patient injury.
5. **CAUTIONS:**
 - 5.1 Federal (USA) Law restricts this product to sale by or on the order of a physician.
 - 5.2 Do not use if the package or container is opened or damaged.
 - 5.3 Avoid excessive heat or freezing as this may damage the vial and result in leaking.
 - 5.4 Follow standard infection control procedures as specified by the Centers for Disease Control and Prevention (USA) or local equivalent.
 - 5.5 Do Not Reuse: Products require specific material characteristics to perform as intended. These characteristics have been verified for single use only. Any attempt to re-process the product for subsequent re-use may adversely affect the integrity of the product or lead to deterioration in performance.
6. **SET-UP AND USE:**
 - 6.1 Using aseptic technique, break off cap and discard.
 - 6.2 Pour desired dosage as directed by physician.
 - 6.3 Discard remaining contents
7. **STORAGE AND HANDLING:**
 - 7.1 Dispose of this product in a safe manner according to local guidelines for disposal of contaminated waste.
 - 7.2 Store at room temperature. Avoid freezing or excessive heat.

	Sterilized using aseptic processing techniques		CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.
	Do Not Reuse		Do not use if package is damaged
	Do not resterilize		Not made with natural rubber latex
	Instructions for use		Warnings
	Lot number		Non-pyrogenic
	Manufactured date		Expiration date
	Keep dry		Keep away from sunlight