



Accel[®] Five TB

Ready-to-Use One-Step Surface Cleaner and Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A ready-to-use cleaner disinfectant based on proprietary AHP[®] technology to deliver fast, effective cleaning performance.

Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus, HIV and Poliovirus providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 5 minutes. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces, while harnessing the power of AHP[®] technology. The AHP[®] active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPEs. Non-irritating to eyes and skin
- Clean and disinfect high-touch surfaces in one labor saving step

Applications

- Clean and disinfect high-touch surfaces in 5 minutes
- Non-irritating to eyes and skin





Accel[®] Five TB

Ready-to-Use One-Step Surface Cleaner and Disinfectant

Use instructions

As a One-Step Surface Cleaner & Disinfectant*:

- Pre-clean heavily soiled areas.
- Apply product to hard, non-porous inanimate surfaces.
- All surfaces must remain wet for 5 minutes.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

* Refer to product label for complete use instructions

| Technical data | Accel [®] Five TB |
|----------------|-------------------------------|
| DIN | 02247924 |
| Color/Form | Colorless, liquid |
| pH | 1.8 |
| Scent | Characteristic of ingredients |
| Shelf Life | 3 years |

| Product | Pack size | Dilution | Product code | |
|----------------------------|-------------------------------------|--------------|--------------|----|
| Accel [®] Five TB | 12 x 1 L / 1.06 U.S. Qt. Bottles | Ready-to-use | 100906726 | 🇨🇦 |
| Accel [®] Five TB | 4 x 4 L / 1.06 U.S. Gal. Containers | Ready-to-use | 100906727 | 🇨🇦 |

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

Accelerated Hydrogen Peroxide^{TM/MC} and AHP[®] logos are trademarks owned by Virox Technologies used under license.