

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

oxygen cylinder rack

for use with item # 18141



www.drivemedical.com


assembly instructions

- For use with E, D, C, and M9 cylinders.

1. Screw in leveling bolts (B) into four posts (A) and clip the sleeves (E) onto the four posts (A).
2. Slide basket (D) onto posts (A) and clip on the sleeves for the upper rack (C).
3. Slide the upper rack (C) onto the four posts (A).

parts

a. posts	4
b. leveling bolts	4
c. rack	1
d. basket	1
e. sleeves	8

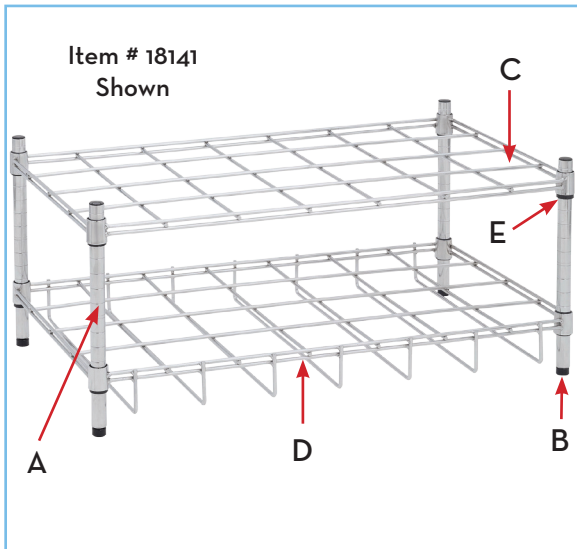

driveTM