

**Bari+Max**<sup>dynarex</sup>  
Support Where You Need It

# Bariatric HD Home Care Half Length Bed Rail

For use with the Bariatric HD Full Electric 48" and 54" Home Care Beds



Read all instructions and warnings carefully before installing or operating the Bed Rail.

## INSTALLATION

1. Attach the Bed Rail to the frame of the moving head deck. **DO NOT** attach to the fixed bed frame.
2. Align the mounting holes on the rail bracket to the bed frame.
3. Insert a bolt from the rail side into the mounting hole. On the other side, slide a metal square plate on to the bolt, then secure the bolt with a nut.



← Metal Square Plate

4. Repeat the procedure to install the rail on the other side.

## RAIL OPERATION

To raise Bed Rail: Pull the Bed Rail upwards until the rail is securely locked in place.

To lower Bed Rail: Pull the plungers on the side of the rail bracket and slowly lower the Bed Rail to the lowest position.

## WARNINGS

Before use, ensure that the Bed Rail is assembled and installed as described, that all attaching hardware is securely tightened, and that the Bed Rail is securely in place. Inspect frequently to ensure attaching hardware is securely tightened and that the Bed Rail is securely in place.

**DO NOT** use the Bed Rail as a handle for getting into or out of bed; this could result in personal injury and damage to the Bed Rail.

**DO NOT** exert side pressure on the Bed Rail; this could deform or break it.

**DO NOT** use the Bed Rail as a push handle for moving the bed; this could deform or break it.

**DO NOT** use the Bed Rail if a proper fit in installation cannot be achieved. Contact your dealer.



**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

**WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

R221201