

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

*This product meets or exceeds all federal safety and environmental regulations.

⚠ WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

R221115

Bari+Max dynarex
Support Where You Need It

Reorder No. 10213

Platform Attachment

User's Manual



Manufactured for: Dynarex Corporation
10 Glenshaw Street | Orangeburg, NY 10962
USA • www.dynarex.com | Made in China

WARNINGS

Important! Read and understand these instructions before installing or using Bariatric Walker Platform Attachments. If you do not understand any part of these warnings or instructions, contact a healthcare professional for direction in the use of this product. Personal injury and damage to the Platform Attachments could result if the Platform Attachments are not properly installed.

Ensure each Platform Attachment is securely installed as instructed before use.

- Ensure walker is securely locked into the open position before installing the Platform Attachments, as shown, or serious personal injury may result.
- Dynarex recommends installing the Platform Attachments on both sides of the walker for better stability.
- Do not use the Platform Attachments with any other walker accessories. The Platform Attachments are recommended for use on Dynarex walkers.
- If any components are damaged or missing, contact your distributor immediately. DO NOT use substitute parts. Use only Dynarex replacement parts.
- Maximum weight capacity 500 lb. EVENLY DISTRIBUTED.

INSTALLATION INSTRUCTIONS


1. Determine which side of the Platform Attachment will be used.
2. Attach clamps to vertical platform post and loosely attach handle to arm support.
3. Loosely attach the bottom clamp to the lower horizontal walker crossbar.
4. Loosely attach the top platform clamp to the upper horizontal walker crossbar.
5. Adjust platform post to desired height.
6. Tighten both clamps carefully adjusting both sides of clamps equally.
7. Adjust arm support and handle to increase patient comfort and support.
8. Tighten all adjustment knobs to ensure patient safety.

INSTALLATION INSTRUCTIONS (CONT'D)

LOWER ATTACHMENT

Install Platform Attachment support tube clamps on walker crossbars as shown at right. Install knobs and hardware as shown; tighten until secure. Repeat for other side.

Note: Install Platform Attachment on Walker crossbars

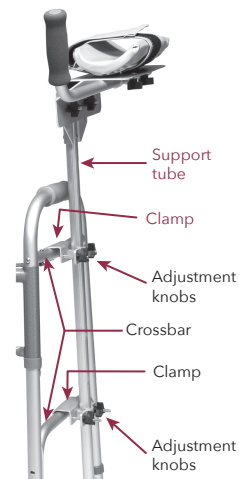

UPPER ATTACHMENT

Armrest Tube

Install Armrest Tube below the platform as shown. Insert carriage bolts, plain washers, and knobs. Tighten until secure.

Adjust Armrest Tube Length. Loosen the armrest tube length adjustment knobs as shown below and slide armrest tube in or out to the desired position. Tighten knobs securely to secure armrest tube

