

WARNINGS

In order to ensure the safe installation, assembly and operation of the Bariatric Home Care Half Length Bed Rails, these instructions **MUST** be followed:

- Check all parts for shipping damage before using. In case of damage, **DO NOT** use the equipment. Contact your Dealer/Carrier for further instruction.
- **DO NOT** install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.
- When used with an electrical bed, the bed rails **DO NOT** fall under any weight limitations. Bed rails can become deformed or broken if excessive side pressure is exerted on the bed rails.
- These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are **NOT** intended (nor may be used) for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.
- Replacement mattresses and bed rails with dimensions **DIFFERENT** than the original equipment supplied or specified by the bed manufacturer are **NOT** interchangeable. Variations in bed rail design, width and thickness or firmness of the mattress could cause/contribute to entrapment. Use **ONLY** authorized replacement parts.
- **DO NOT** use the bed rails as push handles when moving the bed.
- These bedrails are **NOT** handholds for getting into or out of bed.
- After **ANY** adjustment, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely.
- Always test to make sure that the bed rails are properly and securely in place **BEFORE** using.
- Because of the heavier weight limitations associated with the Bariatric Bed, use the mounting brackets attached to the bed frame to assemble the half bed rail side adjustment clamps. These side adjustment clamps are designed to support heavier weight limitations and **MUST** be used when installing half-length bed rails onto the Bariatric Bed.
- We recommend that the mattress be centered on the bed frame. Otherwise, entrapment may occur between the bed rail and the bed frame.
- After raising/lowering the head/foot end of the electric bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which entrapment may occur, adjust the height of the bed rail (if applicable), **OR** provide alternative means of patient protection.

Bari+Max dynarex
Support Where You Need It

Bariatric Home Care Half Length Bed Rail

For use with the Bariatric Full Electric 42" Home Care Bed



SYMBOL GLOSSARY
For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

⚠ WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov


R221201

Manufactured for: Dynarex Corporation
10 Glenshaw Street | Orangeburg, NY 10962
USA • www.dynarex.com | Made in China


INSTALLATION MANUAL

1. Remove the mattress from the bed.
2. Raise the head end of the bed.
3. Remove hardware kit labeled BARIATRIC BED HARDWARE KIT BAR600 located in the bed rail box and set aside for use in installing the cross-braces.


4. Locate the two mounting holes on the top of the deck. They are positioned over the cross brace so that the mounting hole will be visible.

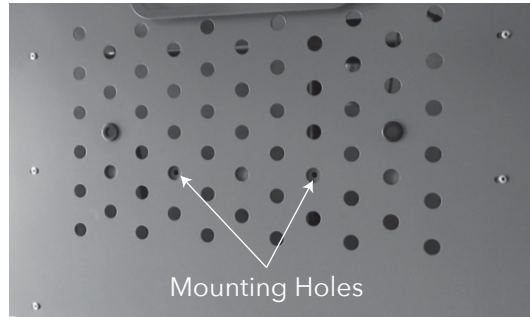


5. Locate the two mounting holes on the sides of the bed frame.

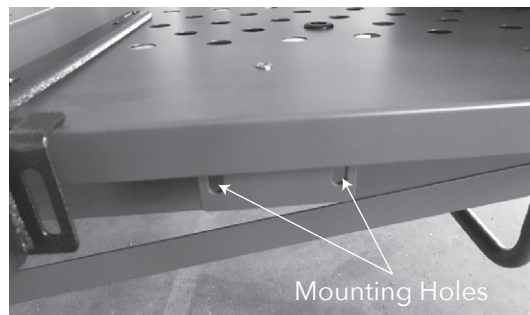

6. Place one of the side adjustment clamps on right side of the bed so the hole at the end of the bar is aligned with the mounting hole located in step 4.

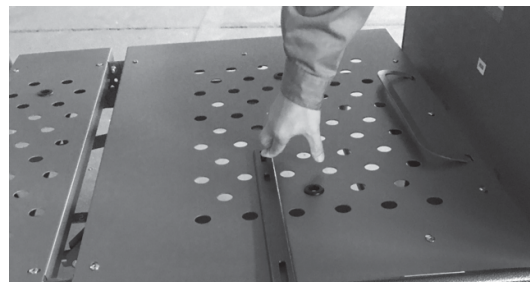


7. Secure the side adjustment clamp to the top of the pan deck using one long bolt and one locknut. Hand tighten only.
8. Align the two mounting holes on the side of the side adjustment clamp with two of the mounting holes of the bed rail mounting bracket located in step 5.

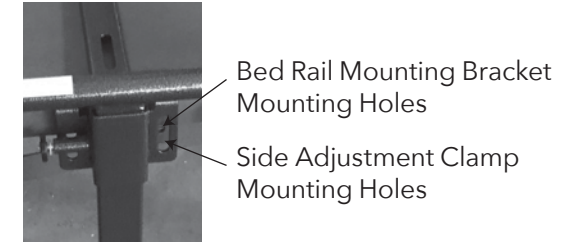

9. Secure the side adjustment clamp to the bed rail mounting bracket, two short bolts, spacers and locknut. Hand tighten only.
10. Securely tighten all hardware.
11. Repeat steps 6-10 for the left side of the bed.

12. Installing the Bed Rails:
Pull the plunger pin on the side adjustment clamp and insert the T-rail shaft into the barrel of the side adjustment clamp. Allow the shaft to slide to the bottom until the top support of the bed rail rests securely against the top of the barrel.

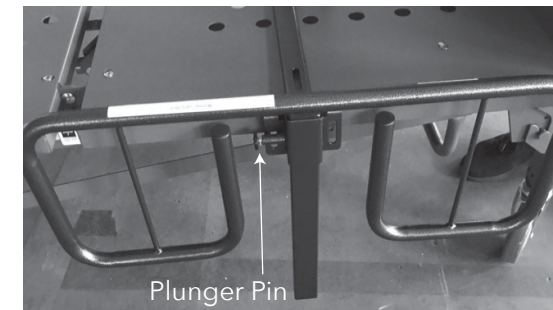

13. To raise the rail, lift the rail until the plunger pin "clicks" securely into the adjustment hole of the T rail shaft.
14. Repeat steps 12 & 13 to install the other bed rail.
15. Position the bed in the flat position and place the mattress on the bed frame.

