**SAVE THESE INSTRUCTIONS**

*DEALER: These instructions MUST be given to the user of this product.*

*NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.*

**⚠ WARNING**

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner’s manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.
- Each individual should **ALWAYS** consult with their physician or therapist to determine proper adjustment and usage.
- The Rollator is designed to provide support, increased stability and assistance to the end user while walking. The Rollator is **NOT** designed to support the total weight of an individual and has a weight limitation of 285 lbs (130 kg).
- The Rollator can provide ambulatory assistance for an individual weighing up to 285 lbs (130 kg), including the weight of the contents of the basket and tray. The seat can support an individual weighing up to 285 lbs (130 kg).
- **DO NOT** use the rollator as a wheelchair or transport device. The Rollator is **NOT** intended for propelling while seated.
- The brakes **MUST** be locked before using the seat. When using the rollator in a stationary standing position, the hand brakes **MUST** be locked.
- **DO NOT** use the rollator on an escalator.
- A physical/occupational therapist should assist in the height adjustment for maximum support and correct brake activation.
- Care should be taken to ensure that all hand and height adjustments are secure, and that casters and moving objects are in good working order before using this or any mobility aid.
- All casters **MUST** be in contact with the floor at all times during use. This will ensure the rollator is properly balanced.
- **ALWAYS** observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.
- The basket, tray and shelf have a weight limitation of 11 lbs (5 kg). Items placed in or on the basket, tray or shelf should **NOT** protrude from the basket, tray or shelf.
- If push handles are exposed to extreme temperature (above 100°F or below 32°F), high humidity and/or becomes wet, prior to use, ensure hand grips **DO NOT** twist on rollator handle - otherwise damage or injury may occur.
- After installation and before use, ensure that all attaching hardware is tightened securely.
- **DO NOT** hang anything from the frame of the rollator. Items should be placed in the basket or on the tray.

**⚠ ACCESSORIES WARNING**

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

**Assembling the Rollator**

*NOTE: For this procedure, refer to FIGURE 1.*

1. Remove plastic caps from the side frame tubes.
2. Unfold the rollator.

*NOTE: Make sure that the red locking mechanism fully engages. An audible “click” will be heard.*

*NOTE: The locking lever and washer are installed on the outside of the frame tubes.*

3. From the inside of the frame, put a bolt through the locking lever mounting holes.
4. Put a washer onto the bolt.
5. Thread the locking lever onto the bolt. **DO NOT** tighten the locking lever.
6. Repeat STEPS 3-5 to install the opposite locking lever.
7. On both sides of the rollator, insert and adjust the handle to the appropriate height. Tighten the locking lever onto the bolt.

**⚠ WARNING**


The handles **MUST NOT** be angled outside the marked position as this reduces the rollators lateral stability. Refer to Detail “B”.

**Removing the Handles**

1. On each side of the rollator, loosen but **DO NOT** remove the locking lever.
2. Remove the handles from the frame tubes.

**Adjusting Handle Height**

1. On both sides of the rollator, loosen, but **DO NOT** remove, the locking levers.
2. Adjust each handle to the appropriate height and tighten the locking lever. Make sure both handles are adjusted to the same height.

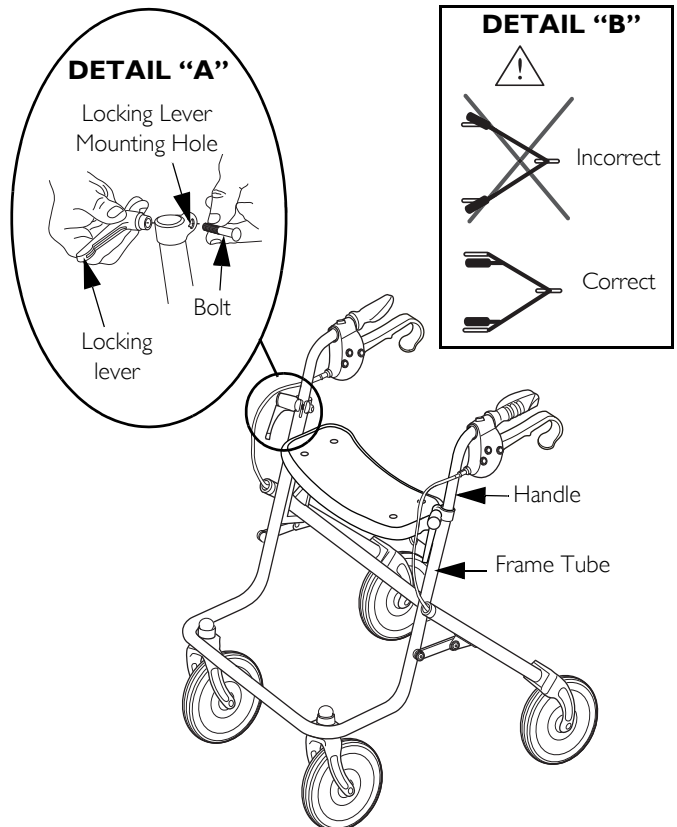**FIGURE 1** Assembling the Rollator

For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

## Installing the Shelf, Basket and Tray

NOTE: For this procedure, refer to FIGURE 2.

NOTE: To remove the shelf, basket, and tray reverse these procedures.

### Shelf

Snap the shelf onto the bottom rollator frame tubes.

### Basket

Install the basket onto the hooks on the rollator frame.

### Tray

Put the tray in place on top of the seat.

## Using the Hand Brakes

NOTE: For this procedure, refer to FIGURE 3.

1. Perform one of the following:
  - To Stop - Squeeze the brake handles up towards the rollator handles.
  - To Continue Mobility - Release the brake handle.
2. To lock, push down on the bottom portion of the brake handle.
3. To unlock, pull up on the top portion of the brake handle.

## Folding the Rollator

NOTE: For this procedure, refer to FIGURE 4.

1. Remove the tray and basket. Refer to [Installing the Shelf, Basket and Tray](#).

NOTE: It is NOT necessary to remove the shelf to fold the rollator.

### **⚠ WARNING**

**Pinch points exist between the rollator frame tubes. Make sure hands and body are clear of all pinch points before folding the rollator.**

2. Pull up on the tab to release the locking mechanism.
3. To fold the rollator, lift up on the cross tube.

### **CAUTION**

**Lay the rollator down on a flat stable surface. DO NOT lean the rollator against a wall or other vertical surface.**

## Care and Maintenance

1. Verify brake operation. Contact your Invacare dealer if brake adjustment is required.
2. If hand grips are loose, DO NOT use the rollator. Contact your Invacare dealer.
3. Periodically inspect the casters tightness, wear and damage.
4. Make sure that the wheels are free of hair, lint and other debris.

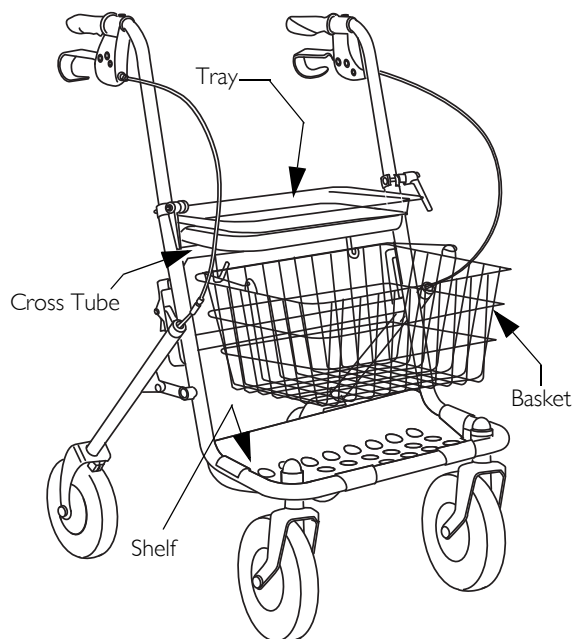
FIGURE 2 Installing the Shelf, Basket and Tray


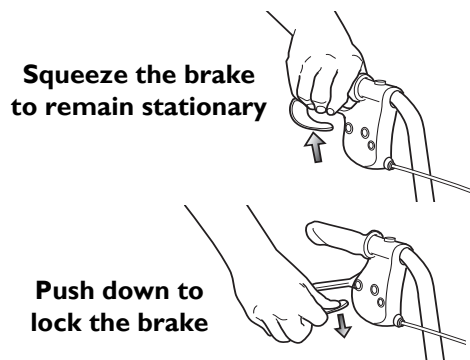
FIGURE 3 Using the Hand Brakes

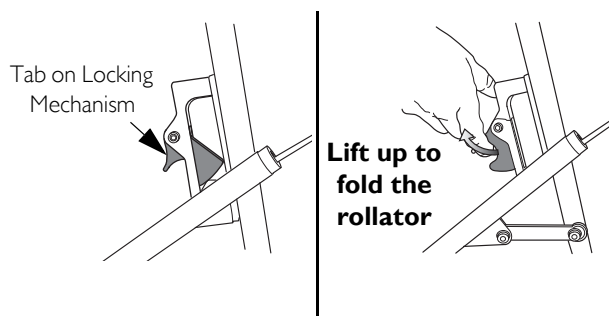**Be aware of pinch points when folding the rollator**

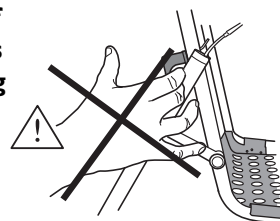
FIGURE 4 Folding the Rollator

---

# NOTES

---

# LIMITED WARRANTY

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the brakes to be free from defects in materials and workmanship for five years, and all other components for one year for the original purchaser (excluding tires and wheels). If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. **C.O.D.** shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**

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© 2009 Invacare Corporation