Important: Be sure to retain this manual for future reference and read it completely before using the Evenflo Silicone Steam Sanitizing Bag.

CAUTION: Use care when removing from microwave as bag will be filled with hot water and steam after use, which could cause burns. DO NOT tip the bag when removing from the microwave.

⚠ WARNINGS


- For adult use only.
- To avoid suffocation keep this bag away from babies and children. This bag is not a toy. DO NOT use in cribs, beds, carriages or play pens. Cut bag along edge before discarding.
- Evenflo Silicone Steam Sanitizing Bag is to be used with parts that are microwave-safe only. DO NOT place the steam bag on a microwave metallic grill. DO NOT place the bag in convection oven or on an oven grill.
- Be sure to follow the provided Heating Time Chart on reverse side as heating for longer than the recommended time may warp bag contents. Use of less than 2oz (60mL) of water may warp bag contents.
- Discard Silicone Steam Sanitizing Bag at first sign of weakness, tears, or damage.
- Avoid placing sharp objects in or on the bag, as they may damage the bag.
- Follow the manufacturer's instructions for use of your microwave.

BEFORE FIRST USE

Wash thoroughly in warm, soapy water or wash in the dishwasher. Rinse and dry thoroughly.

INSTRUCTIONS FOR USE

1. Add 2 ounces (60mL) of water to the bag.
2. Place clean, microwave-safe nipples, pacifiers, bottles or pump parts into the sanitization bag.
NOTE: Soiled items must be washed prior to placing in sanitization bag.
3. Ensure each button **A** on the bag is secured tightly.
NOTE: DO NOT overload bag (if any of the buttons cannot be tightly secured, the bag is overloaded, and item(s) should be removed until the bag can seal correctly).
4. Reference Heating Time Chart on reverse side.
5. **After microwaving, allow bag to cool to room temperature before handling the bag.**
6. Carefully remove bag from microwave by pinching the top outer **B** corners on each side of the bag.
7. If the bag becomes soiled, wash with warm, soapy water or in the dishwasher to clean.
8. Let the bag dry completely before storing.

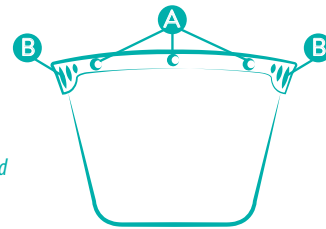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