


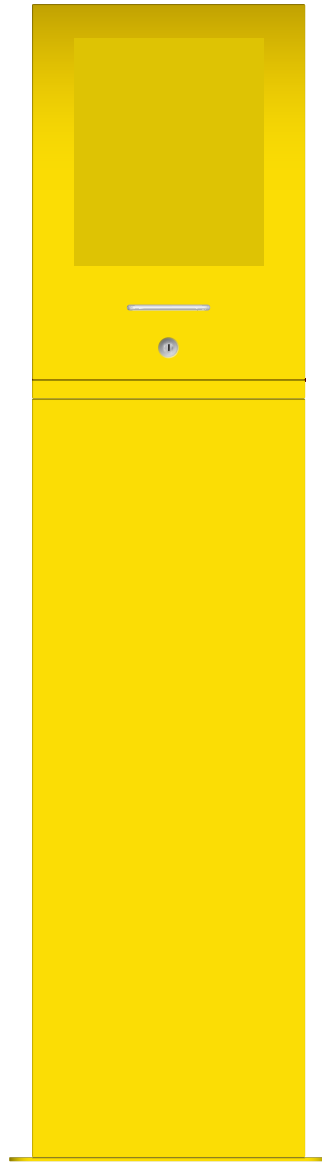


CARDIACT YELLOW FLOOR MOUNT AED CABINET

CABINET INSTALLATION

Attention

Before installing the AED standing cabinet, confirm whether the floor material can be drilled to ensure the stability and safety of the cabinet installation.

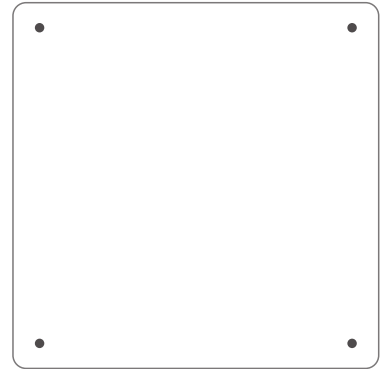


- 1 Drill four 10mm diameter holes in the ground
- 2 Place four expansion bolts into each of the four holes
- 3 Tighten the hexagonal nuts of the expansion bolts a few turns
- 4 The expansion tube is fixed to the ground after being opened by force
- 5 Unscrew the nuts, remove the spacers and spring washers
- 6 Align the four holes of the cabinet base with the expansion bolts
- 7 Put the spacer and spring washer on the expansion bolt
- 8 Tighten the hexagonal nuts to complete the installation of the cabinet

POST-MAINTENANCE

Replace battery

When taking out the battery box, do not pull it too hard to avoid loosening the wires.

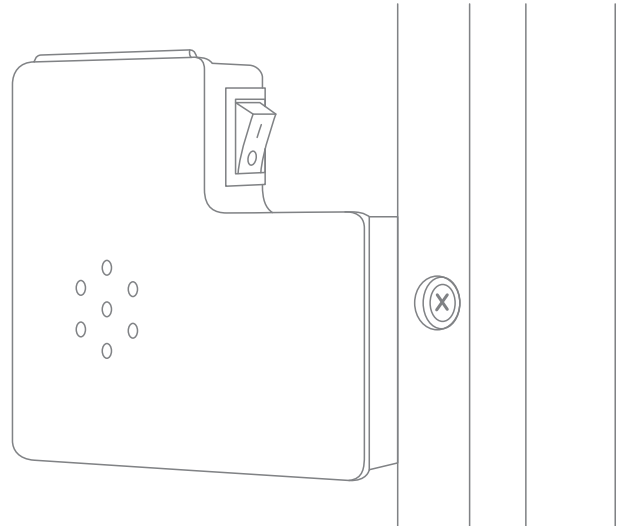

- 1 Open the door of the cabinet
- 2 Take out the battery box
- 3 Open the cover of the battery box
- 4 Take out the battery
- 5 Put a new battery back into the battery box
- 6 Close the battery cover
- 7 Put the battery box back into the cabinet