



# Accel<sup>®</sup> Five TB Wipes

## Surface Cleaner & Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

### Disinfectant Cleaner Wipes

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP<sup>®</sup> technology to deliver fast, effective cleaning performance.

### Features & Benefits

- Effective against key pathogens including MRSA, VRE, TB, Norovirus, HIV and Poliovirus providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 5 minutes. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces, while harnessing the power of AHP<sup>®</sup> technology. The AHP<sup>®</sup> active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPEs. Non-irritating to eyes and skin
- Convenient pre-wetted wipes clean and disinfect in one labor-saving step

### Applications

- Clean and disinfect high-touch surfaces in 5 minutes
- Non-irritating to eyes and skin





# Accel<sup>®</sup> Five TB Wipes

Surface Cleaner & Disinfectant

## Use instructions

### As a One-Step Surface Cleaner Disinfectant\*:

- Pre-clean heavily soiled areas.
- Apply disposable wipe to hard, non-porous inanimate surfaces.
- All surfaces must remain wet for 5 minutes.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

\* Refer to product label for complete use instructions

Technical data	Accel <sup>®</sup> Five TB Wipes
DIN	02247932
Color/Form	Colorless, wipe
pH	1.8
Scent	Characteristic of ingredients
Shelf Life	3 years

Product	Pack size	Dilution	Product code	
Accel <sup>®</sup> Five TB Wipes	12 x 160 count wipes 15.24 cm x 17.78 cm / 6 in. x 7 in. Canisters	Ready-to-use	100906729	
Accel <sup>®</sup> Five TB Wipes	12 x 60 count wipes 25.4 cm x 25.4 cm / 10 in. x 10 in. Canisters	Ready-to-use	100906588	

## Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

Accelerated Hydrogen Peroxide<sup>TM/MC</sup> and AHP<sup>®</sup> logos are trademarks owned by Virox Technologies used under license.