



Accel PREvention™ Wipes

One-Step Surface Cleaner and Disinfectant

Tuberculocidal, Virucidal, Bactericidal, Fungicidal, Broad-Spectrum Sanitizer

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- Effective against key pathogens including SARS-CoV-2 (cause of COVID-19), MRSA, VRE, TB, Norovirus, HIV and Poliovirus providing broad spectrum disinfection
- Virucidal, Bactericidal, Tuberculocidal and Fungicidal all in 3 minutes. It is also a 30-second broad-spectrum sanitizer
- Compatible with most hard, non-porous surfaces while harnessing the power of AHP® technology
- The AHP® active breaks down to oxygen and water after use
- No VOCs, no added fragrance and no NPES
- Convenient pre-wetted wipes clean and disinfect in one labor-saving step

Applications

- Clean and disinfect high-touch surfaces in 3 minutes





Accel PREvention™ Wipes

One-Step Surface Cleaner and Disinfectant

Use Directions: As a One-Step Cleaner Disinfectant*

- Pre-clean heavily soiled areas.
- Apply disposable wipe to hard, non-porous inanimate surfaces.
- All surfaces must remain visibly wet for 3 minutes.
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

All food contact surfaces must be rinsed with potable water.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

* Refer to product label for complete use instructions

Technical data	Accel PREvention™ Wipes
DIN	02448459
Color/Form	Colorless/Wipe
pH	2.1
Scent	Characteristic of ingredients
Shelf Life	3 years (100906721, 100906722) 2.5 years (100906590)

Product	Pack size	Dilution	Product code	
Accel PREvention™ Wipes	12 x 160 count Wipes 15.24 cm x 17.78 cm / 6 in. x 7 in. Canisters	Ready-to-use	100906721	🇨🇦
Accel PREvention™ Wipes	12 x 60 count Wipes 25.4 cm x 25.4 cm / 10 in. x 10 in. Canisters	Ready-to-use	100906722	🇨🇦
Accel PREvention™ Wipes	120 x 8 count Wipes 19 cm x 25.4 cm / 7.5 in. x 10 in. Soft Packs (8 wipes per pack, 20 packs per carton, 6 cartons per case)	Ready-to-use	100906590	🇨🇦

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.