



# GPS TRACKING DEVICE

## FOR INSTALLATION IN AEDS



### Directions for Installation and Use

1



Remove all items from the zip seal bag and check against the advised inventory list. Remove tracking device from the box provided.

2



Remove the screws from the tracking device and fit the 2 AA batteries as directed.

3



Replace the screws to ensure the tracking device is securely closed.

4



Peel the backing from one side of each self-adhesive tape square.

5




Apply the tape squares to the tracking device as shown in the image, as this positioning is vital to keep the screws accessible for future battery replacement.

6



Align the tracker on the AED as depicted, ensuring it does not obstruct the battery. Apply firm pressure to ensure strong adhesion.

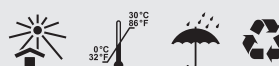



# GPS TRACKING DEVICE

## FOR INSTALLATION IN AEDS



### Directions for Installation and Use

1



Remove all items from the zip seal bag and check against the advised inventory list. Remove tracking device from the box provided.

2



Remove the screws from the tracking device and fit the 2 AA batteries as directed.

3



Replace the screws to ensure the tracking device is securely closed.

4



Peel the backing from one side of each self-adhesive tape square.

5



Apply the tape squares to the tracking device as shown in the image, as this positioning is vital to keep the screws accessible for future battery replacement.

6



Open the navy accessory compartment on the HeartSine case. Position the tracking device on the inside of the compartment, firmly against the bottom edge of the case as shown in the picture, for stability. Apply firm pressure to ensure strong adhesion.

