



CLEANPatch[®] Mattress repair system

A medical surface repair patch for hospital beds and stretchers that restores damaged mattresses to an intact and hygienic state.

Features & Benefits

- FDA and CE registered Class I medical device that restores damaged mattresses to an intact and hygienic state
- Made of latex-free, biocompatible materials. Extends the life of mattresses and stretchers and reduces the risk of cross contamination from damaged mattresses
- Peel and stick application
- Provides savings from equipment repair vs. premature replacement
- Proactively improves patient safety
- Fully cleanable with most hospital disinfectants



Applications

- Hospital bed mattresses
- Stretcher mattresses
- Operating room table pads





Use Instructions

- Assess the damage to determine if appropriate
- Measure the damage to determine which size of CleanPatch[®] to use
- Clean mattress as per hospital protocol. Let it dry.
- Remove the white backing from CleanPatch[®]
- Hold edges of the patch and position it over the damage
- Apply firm pressure to the patch, especially around the edges
- Slowly remove the clear top liner, pulling from the arrow toward the dot
- Smooth CleanPatch[®] from the center outwards. For best results wait 1 minute.
- Clean the patch and surrounding area per hospital protocol

| Technical data | Clean Patch |
|-----------------------|---|
| Regulatory/Compliance | Health Canada (MDEL); FDA (Class I); CE |
| Color/Form | Dark blue material |
| Shelf Life | 3 Years |

| Product | Shape Damage Size* | Shape | Quantity | Product code | |
|--------------------------|---|---|-------------------------|--------------|--|
| Clean Patch [®] | 2 in. (5 cm) Up to 1/2 in. (1.3 cm) | Small Round | Box of 10 | 11003 | |
| Clean Patch [®] | 3.5 x 3.5 in. (9 x 9 cm) Up to 2 in. (5 cm) | Medium Square | Box of 5 | 10003 | |
| Clean Patch [®] | 3.5 x 6 in. (9 x 15 cm) Up to 4 in. (10 cm) | Large Rectangle | Box of 5 | 12003 | |
| Clean Patch [®] | CleanPatch Mix Box, 2" (5), 3.5" (3), 3.5" X 6" (2) | Small Round, Medium Square, Large Rectangle | Box of 10 for mixed box | 18003 | |

*There must be at least 1/2 inch margin around the damage to ensure proper adhesion

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.