boon[®]

TROVE[™]

+ Passive Silicone Pump
Bandeja de polos y congelador

B11531A1-02-INT-01

3 fl oz / 88 ml

IMPORTANT:

- This product is not a toy.
- Use the included plug only with Boon Trove™.
- Inspect frequently for tears or cracks. If any are found, discontinue use immediately.

PUMPING WITH TROVE™

1. Place TROVE onto your breast so that your nipple is centered in the hole. It should feel comfortable and not be pushing against any part of TROVE.
2. Press firmly on the Boon logo to create suction. Confirm that your nipple is still centered.
3. Pull your nursing bra over TROVE. Make sure it is still firmly in place with no gaps around the edges.
4. If you feel TROVE loosening, push firmly to increase the suction. This suction keeps the milk flowing and may need to be repeated several times while pumping. Be cautious of your milk level before pressing on the pump.
5. Watch the milk level and remove TROVE when it reaches the 3 oz line.

REMOVING TROVE & STORING MILK

1. When finished, gently use your finger to break the seal. Be sure to hold it securely so that it won't slip off of your breast when the suction is released.
2. TROVE is designed to stand easily on the tabletop while you get ready to pour.
3. Slowly tilt TROVE in the direction of the arrow and pour it directly into a bottle or your preferred milk storage container.

NOTE: Always remove TROVE when finished. We recommend wearing for no longer than two hours at a time. It is NOT intended for milk storage. Milk collected or stored at room temperature for too long should be discarded and not fed to your baby.

CLEANING & CARE

To ensure hygiene before first use, place into boiling water for 5 minutes. To ensure hygiene, wash thoroughly after each use or before each subsequent use. Top rack dishwasher-safe or hand wash in warm, soapy water. Rinse with clear water. Store in a dry clean place away from direct sunlight and heat. Can be sterilized in microwave or using sterilizing equipment. Do not leave the product in disinfectant (sterilizing solution) for longer than recommended. Hand wash bag in warm, soapy water and hang to dry.

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© 2024 TOMY
45 SHAWMUT RD,
CANTON, MA 02021, USA
US: 800-704-8697
WWW.TOMY.COM

MADE IN VIETNAM
FABRICADO EN VIETNAM

Please retain all product information for future reference. Color and contents may vary. Trademarks and copyrights are exclusively owned by TOMY. 240725

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