
## 3-in-1 Nursery Thermometer

User's Guide TH063, TH064, TH075, TH086, TH087, TH096

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

### Basics of Your Thermometer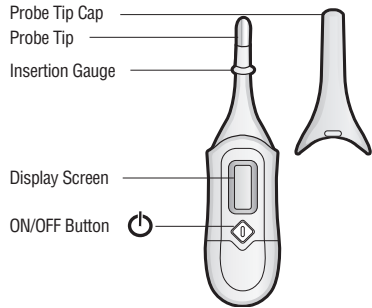

### Care and Maintenance

#### Cleaning Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

#### Cleaning Thermometer Body:

- Use a soft, dry cloth to clean thermometer body.

#### Notes:

- **DO** remove battery if stored for a long period of time.
- **DO** store thermometer in a dry place at room temperature with the protective cap in place.
- **DO** take special care not to scratch the display window.
- **DO NOT** allow electronics to get wet.
- **DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning.
- **DO NOT** disassemble for any reason other than battery replacement.
- **DO NOT** drop thermometer.

### Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +

# WARNING

- This is **NOT** an ear thermometer. **NEVER** put probe into ear.
- Take temperature measurements Underarm, Orally or Rectally.
- **ALWAYS** supervise your child when using thermometer. **NEVER** allow your child to use it alone.
- **NEVER** allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision.
- **DO NOT** expose the thermometer to extreme temperatures, humidity, direct sunlight, or shock. Avoid sudden impact and dust.
- Keep battery out of child's reach. Swallowing a battery could be fatal.
- Battery should not be charged or placed into extreme heat, as it may explode.
- Used batteries should be disposed of properly.

## How to Measure Temperature

Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

- Remove probe tip cap. Press and release ON/OFF button. Thermometer beeps once and all segments display briefly as the thermometer powers on.

- The last temperature measured displays with the **M** icon on screen for a few seconds.


- Confirm that °F (or °C) is flashing. This indicates that the thermometer is ready to take measurement.

### Specifics for each Temperature Mode

#### ORAL—For children 4 years and older.

1. Ensure that °F (or °C) is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.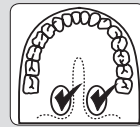


#### RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position baby so that the back and legs form a 90-degree angle (L-shape).
2. Ensure that °F (or °C) is still flashing, then insert probe tip no more than 1/2 inch (1.3 cm) into rectum. Do not force the tip into rectum if resistance is encountered.

#### UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

1. Wipe underarm with a dry towel.
2. Ensure that °F (or °C) is still flashing, then place probe tip at center of underarm. Bring arm down against the body. Probe must touch skin.

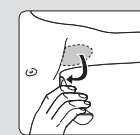- When °F (or °C) stops flashing, the measuring is complete. Approximate measurement time is 30 seconds. Thermometer beeps 10 seconds for normal temp and approximately 15 seconds for fever temps.
  - Remove probe tip from measurement site and check temperature.
  - Press button to shut off or thermometer will shut off automatically after 8-9 minutes.
  - Thermometer must be shut off and turned on again before another measurement can be taken.
- NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included.
- DO NOT** allow child to walk or run during temperature taking. Probe could present a hazard.

### Memory Recall

Thermometer stores the last temperature reading. As a reference for parents, that temperature reading displays automatically the next time the thermometer is powered on.

If the thermometer is powered on and off, the reading in memory is erased.

### Fahrenheit / Celsius

Press button to power on the thermometer. As soon as "----" displays on the screen, press and hold the button for at least 3 seconds to switch between F and C. Release button when the desired indicator displays on screen.

### How to Replace the Batteries


#### Dead Battery Indicator:

When the "▽" icon flashes, the battery is exhausted and needs to be replaced.

#### To Replace Battery:

- 1 Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
- 2 Use a toothpick to remove battery.
- 3 Insert new 1.5 volt battery (LR41 or equivalent), correctly positioned with the rounded negative side facing up as shown in the illustration.
- 4 Place the battery cover on the thermometer and secure the cover.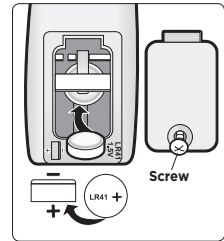

### Technical Specifications

Measurement Temperature Range: 90° F to 109.9° F

#### Measurement Accuracy:

- +/- 0.2° F for temps 95° F to 107.6° F
- +/- 0.4° F for temps below 95° F or above 107.6° F.

Operation Environment: 50° F to 104° F

Storage Environment: -13° F to 140° F

#### Test Protocol:

- 1 Put the tip of thermometer into water bath, approximately 1/2" below the surface.
- 2 Wait for 10 seconds, then turn on the thermometer by pressing the ON/OFF button. The thermometer is now in standard mode.
- 3 When the measurement is completed, record the temperature reading.

### One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc. will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

**SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.**

**THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.**

### Troubleshooting

Display	Meaning	Recommended Action	Display	Recommended Action
H	<ul style="list-style-type: none"> <li>• Thermometer may have been stored in a hot environment.</li> <li>• Measured temperature is above this thermometer's range.</li> </ul>	<ul style="list-style-type: none"> <li>• Wait 30 minutes to allow thermometer to reach room temperature, then try again.</li> <li>• See Technical Specifications section for Measurement Temperature Range.</li> </ul>	▽	<ul style="list-style-type: none"> <li>• Batteries are low. Replace them. (See How to Replace Batteries section).</li> </ul>
Err	<ul style="list-style-type: none"> <li>• Thermometer may have been stored in a hot or cold environment.</li> <li>• The thermometer cannot function if the environment's temperature is outside the permissible range.</li> <li>• Thermometer may have been mishandled. You may have irreversibly damaged it.</li> </ul>	<ul style="list-style-type: none"> <li>• Wait 30 minutes to allow thermometer to reach room temperature, then try again.</li> <li>• See Technical Specifications section for Operation Environment.</li> <li>• Shut off and restart before discarding.</li> </ul>		<ul style="list-style-type: none"> <li>• Do not return this product to the place of purchase. If any parts are missing, email consumer@djgusa.com, call Consumer Relations at (800)544-1108, or fax at (800)207-8182. You can also visit our website at www.safety1st.com. Have ready the model number ((TH063, TH064, TH075, TH086, TH087, TH096) and date code (manufacture date located on package).</li> <li>• Install new batteries and re-try function before calling Consumer Relations on electronics.</li> </ul>

© 2017 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com Styles and colors may vary. Made in CHINA.

Distributed by Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494  
Dorel Juvenile Canada, 2855 Argentia Road, Unit 4, Mississauga, ON, L5N 8G6

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