



Oxivir[®] Tb Wipes

Ready-to-use Surface Cleaner & Intermediate Disinfectant, General Virucide, Tuberculocide

A ready-to-use disinfectant cleaner based on proprietary hydrogen peroxide (AHP[®]) technology to deliver fast, effective cleaning performance.

Features & Benefits

- Bactericidal, Tuberculocidal and Virucidal in one minute;
Fungicidal in three minutes
- Meets Health Canada's standard for cleaning and decontaminating blood and body fluids
- Compatible with most hard, non-porous surfaces while harnessing the power of Accelerated Hydrogen Peroxide[®] (AHP) technology
- AHP[®] active breaks down into water and oxygen after use
- Delivers fast and effective cleaning performance

Applications

- Clean and disinfect high-touch surfaces
- Non-irritating to eyes and skin





Oxivir[®] Tb Wipes





Ready-to-use Surface Cleaner & Intermediate Disinfectant,
General Virucide, Tuberculocide

Oxivir[®] Tb Wipe is a ready-to-use, pre-wetted disinfectant cleaner wipe based on proprietary hydrogen peroxide (AHP[®]) technology to deliver fast, effective cleaning performance.

Use Instructions

- Pre-clean heavily soiled areas
- Apply solution by cloth or disposable wipe to hard, non-porous environmental surfaces
- All surfaces must remain wet for one-minute to kill bacteria, HIV-1, HBV and HCV
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary
- Provides broad spectrum sanitizing of non-food contact surfaces in just 30 seconds
- Not recommended for use on brass, copper, aluminum, bronze, granite, zinc or marble
- Food contact surfaces require rinsing with potable water after disinfection

Technical data	Oxivir [®] Tb Wipes
DIN	02283530
Color/Form	Colorless, wipe
pH	2.2 – 3.2
Scent	Characteristic of ingredients
Shelf Life	3 Years

Product	Pack size	Dilution	Product code
Oxivir [®] Tb Wipes	12 x 160 count (6 x 7 in. Wipes) / 15.24 x 17.78 cm Canisters	Ready-to-use	5144708 
Oxivir [®] Tb Wipes	4 x 160 count (11 x 12 in. Wipes) / 27.9 x 30.5 cm Buckets	Ready-to-use	5627460 
Oxivir [®] Tb Wipes	12 x 60 count (7 x 8 in. Wipes) / 17.78 x 20.32 cm Canisters	Ready-to-use	5995422 
Oxivir [®] Tb Wipes	4 x 160 count (11 x 12 in. Wipes) / 27.9 x 30.5 cm Refills	Ready-to-use	100823907 

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide[®] and Design are trademarks of Diversey, Inc.