

Easy Read 4-in-1 Thermometer

User's Guide TH053

Thank you for purchasing this Advanced Solutions™ thermometer. We're proud to be your choice to help you keep your child healthy and safe. Please be sure to read all warnings, instructions and notations before using this product.

Basics of Your Thermometer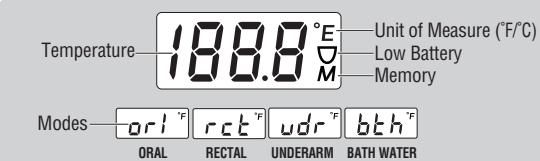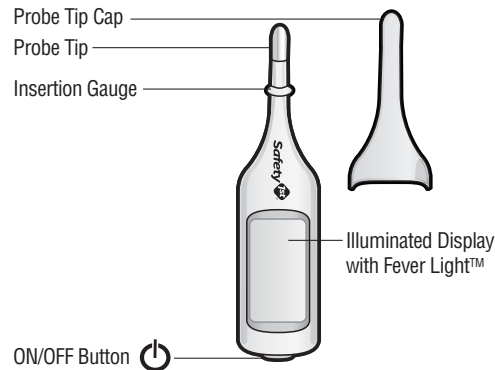

Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings can also be affected by other factors, including exercise, consuming hot or cold food/beverages prior to taking a reading, and technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +

Fever Light™

The backlight illuminates red and thermometer beeps when:

- body temperature is in the fever range (varies by measurement site, see chart above)
- bath water temperature is too hot (exceeding 100.0°F)

High or prolonged fever requires medical attention, especially for young children. Contact your physician. A fever chart is included on the product.

WARNING

- This is **NOT** an ear thermometer. Take temperature measurements Underarm, Orally or Rectally.
- NEVER** allow a child to play with this thermometer. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision.
- ALWAYS** supervise your child when using thermometer. **NEVER** allow your child to use it alone or without close parental supervision.
- Avoid sudden impact and dust.
- Do not expose the thermometer to extreme temperatures, humidity, direct sunlight or shock.
- Replace only with CR2032 or equivalent lithium battery. Use of other types of batteries may present a risk of fire or rupture.

BATTERY CAUTION

- Keep batteries out of child's reach. Swallowing a battery could be fatal.
- Used batteries should be disposed of properly.
- Battery should not be charged or placed into extreme heat as it may explode.
- Remove batteries when stored for long periods of non-use.


IMPORTANT INFORMATION

- Read these instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.
- Rectal temperatures vary by approximately 0.9°F/0.5°C.
- DO NOT** allow child to walk or run during temperature taking. Probe could present a hazard.

How to Measure Temperature

Please read this section carefully and all the way through before using the thermometer. Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.


- Remove probe tip cap. Press and release ON/OFF button. All segments display for 1 second (Figure A) as the thermometer powers on.
- Thermometer beeps once, and the last temperature measured displays on screen (Figure B) for a few seconds.
- The current selected mode flashes and then °F (or °C) flashes. Confirm that the desired temperature mode is selected. (See "Select Mode" to change temperature mode.)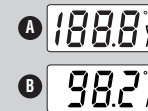

Specifics for each Temperature Mode

ORAL—For children 4 years and older.

- Confirm that oral mode is selected **ori**.
- Ensure that **L** displays and that °F (or °C) is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.

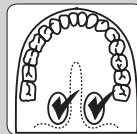RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

- Confirm that rectal mode is selected **rct**.
- Lubricate probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position baby so that the back and legs form a 90-degree angle (L-shape).
- Ensure that **L** displays and that °F (or °C) is still flashing, then insert probe tip no more than 1/2 inch (1.3 cm) into rectum. Do not force the tip into rectum if resistance is encountered.

UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

- Confirm that underarm mode is selected **udr**.
- Wipe underarm with a dry towel. Probe must touch skin.
- Ensure that **L** displays and that °F (or °C) is still flashing, then place probe tip at center of underarm. Bring arm down against the body.

BATH WATER—Ideal bath water temp is between 90° F and 100° F.

- Confirm that bath water mode is selected **bth**.
- Ensure that **L** displays and that °F (or °C) is still flashing, then place probe tip into bath water, up to the insertion gauge. **DO NOT** allow electronics to get wet.


- When °F (or °C) stops flashing, the measuring is complete. Approximate measurement times are: 5 seconds for oral and rectal temps, 8 seconds for underarm temp and 8-15 seconds for bath water temp.
- Remove probe tip from body location or bath water and check temperature.
- Thermometer beeps and the display glows with red backlight to indicate that fever has been detected or that bath water is too hot. (See Fever Light™ section for details.)
- Thermometer will shut off automatically after one minute, or press button to shut off.

Trouble Shooting

Display	Recommended Action	Display	Recommended Action
L	<ul style="list-style-type: none"> Probe may not have been positioned correctly. Proper probe placement is essential for an accurate reading. Review probe placement instructions and try again. If thermometer has been stored in a cold environment, wait 15 minutes to allow thermometer to reach room temperature, then try again. If you are measuring bath water temp, confirm that thermometer is in bath temperature mode (bth). A reading below 89.6°F will cause the L display. If thermometer has been mishandled, you may have irreversibly damaged product. Confirm other possible solutions (above) before discarding. 	U	<ul style="list-style-type: none"> Battery icon displays when battery is getting low. A few more readings may be possible, but it is time to purchase a new battery. Battery icon flashes when battery is completely exhausted and needs to be replaced.
H	<ul style="list-style-type: none"> Turn thermometer off, wait one minute, then re-take temp. If thermometer has been stored in a hot environment, wait 15 minutes to allow thermometer to reach room temperature, then try again. If you are measuring bath water temp, confirm that thermometer is in bath temperature mode (bth). A reading above 111.1°F will cause the H display. If thermometer has been mishandled, you may have irreversibly damaged Product. Confirm other possible solutions (above) before discarding. 	Red backlight is illuminated	<ul style="list-style-type: none"> Measured body temperature is in fever range or bath water temperature is too hot. (See Fever Light™ section for details).
			<ul style="list-style-type: none"> Do not return this product to the place of purchase. If any parts are missing, email consumer@djgusa.com, call Consumer Relations at (800)544-1108, or fax at (800)207-8182. You can also visit our website at www.safety1st.com. Have ready the model number (TH053) and date code (manufacture date located on package). Install new batteries and re-try function before calling Consumer Relations on electronics.
		Err	<ul style="list-style-type: none"> If thermometer has been mishandled, you may have irreversibly damaged it. Shut off and restart before discarding.

Select Mode

- While the thermometer is off, press and hold the ON/OFF button for 3 seconds.
- Press the button again to cycle through the four mode options (Figure C).
- When the desired mode displays on screen, wait for auto shut-off to select it.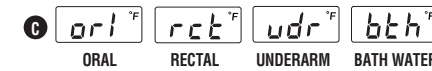

Memory Recall

Thermometer stores the last temperature reading with the site from which it was measured. As a reference for parents, that temperature reading displays automatically when the thermometer is powered on.

Fahrenheit / Celsius

- Power on the thermometer.
- During the first 5 seconds, press and hold the ON/OFF button until °F or °C flashes.
- Quickly press ON/OFF button to switch between °F and °C modes.
- When the desired unit of measure (°F or °C) displays on screen, wait for auto shut-off to select it.

Care and Maintenance

Cleaning Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

Cleaning Thermometer Body:


- Use a soft, dry cloth to clean thermometer body.

Notes:

- DO** store thermometer in a dry place at room temperature with the protective cap in place.
- DO** take special care not to scratch the display window.
- DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning.
- DO NOT** expose thermometer to extreme temperatures, humidity or direct sunlight.
- DO NOT** drop thermometer.

How to Replace the Battery

- When the "U" icon appears while the unit is ON, (Figure D), the battery is nearly exhausted, but a few measurements may still be taken.
- When the "U" icon flashes, (Figure E), the battery is exhausted and need to be replaced.

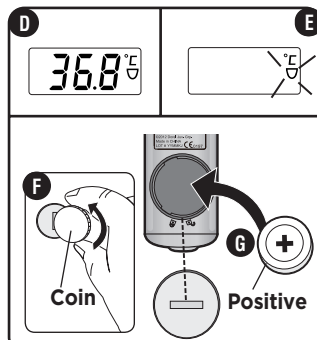To Replace Battery:

- Use a coin to loosen battery cover. Remove cover (Figure F).
- Pry battery from compartment and discard properly.
- Insert a new 3-volt battery (CR2032 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration (Figure G).
- Place the battery cover on the thermometer and secure the cover.

Technical Specifications

- Measurement Range:** 89.6° F to 111.1° F
- Measurement Accuracy:**
 - +/- 0.2° F for temps 98.6° F to 102.2° F
 - +/- 0.3° F for temps 96.4° F to <98.6° F or >102.2° F to 105.8° F
 - +/- 0.5° F for temps <96.4° F or >105.8° F
- Operation Environment:** 60.8° F to 104° F
- Storage Environment:** -4° F to 120° F
- Test Protocol:** Unit must be tested using water bath.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

©2012 Cosco Management, Inc.
 All Rights Reserved. Todos derechos reservados. www.djgusa.com
 (800) 544-1108 www.safety1st.com
 Made in CHINA. Hecho en CHINA.
 Styles and colors may vary. Los estilos y los colores pueden variar.
 Distributed by (distribuido por) Dorel Juvenile Group, Inc.,
 2525 State St., Columbus, IN 47201-7494
 Dorel Distribution Canada, 873 Hodge, St. Laurent, QC H4N 2B1
 11/07/12 4358-6048A

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