
Deluxe Healthcare & Grooming Kit IH025, IH213, IH234, IH261, IH266, IH324

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

Deluxe Wrapping Clutch Case

The wrapping clutch case lays flat when open and has two roomy compartments with clear-view front panels for easy location of stored items. Each compartment zips securely closed. To close the case, close the zipper on each side to contain the items within the compartments; bring the two halves of the case together with the clear-view panels facing each other; wrap the top of the two compartments over so that D-rings are as shown in the image at right; and use the grosgrain ribbon and D-rings to secure the wrapping clutch case.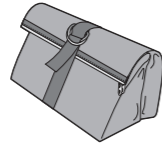

Use and Care: Spot clean. Wipe with a damp cloth.

5 Digital Thermometer Covers

NOTE: Use thermometer covers when taking temperature rectally; not for oral use (see Warning section).

To Use:

- 1 Hold edge of cover and slip over. Pull the outside protective cover off. Make sure no air remains between the probe and the cover.
- 2 **Rectal Use:** Lubricate the thermometer cover with a water-soluble lubricant or petroleum jelly. Follow instructions under "Rectal Use." Discard thermometer cover after use. Do not reuse covers.

NOTE: To purchase thermometer covers call Consumer Relations at (800) 544-1108, and ask for part number 49520.

Bottle Medicine Dispenser

The Bottle Medicine Dispenser has easy-to-read dosage markings and measures up to 3 tsp. (15 ml).

To Use:

- 1 Remove top from measuring cup and fill cup to the prescribed dosage.
- 2 Snap top securely onto the measuring cup.
- 3 Bottle-feed child as normal, pausing as necessary so that child may swallow comfortably.

To Clean: Wash before first use and after every use. Disassemble top ring, nipple and measuring cup. Wash with warm, soapy water and rinse thoroughly. Alternatively, it is dishwasher safe. TOP RACK ONLY.

CAUTION: Keep all medications out of child's reach. We recommend that you use a child resistant top to seal the medicine bottle and store it in a locked cabinet or other location that is not accessible to a child. inspect bottle nipple regularly for tears and cracks, and discontinue use if any are noted. Made without BPA.

Nasal Aspirator

To Use:

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean: Aspirator is top-rack dishwasher safe or can be washed in warm soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

CAUTION: NEVER push the aspirator too far into the nostril and always wash and dry thoroughly after each use. **NEVER** use in baby's mouth or ears. when storing, keep away from heat and bright light.

Steady Grip Nail Clippers & 5 Emery Boards

These clippers have been specially designed with an extra large, easy-grip handle for convenient adult use.

To use:

- 1 Lift arm from hand grip and rotate it 180°.
- 2 Position nail between cutting edges and squeeze handle to cut nail.
- 3 Emery board use is recommended to smooth any remaining sharp nails after clipping and also to smooth minor rough or jagged fingernail edges at other times.

To clean: Wipe clippers with a damp cloth and dry after each use. Clippers may be sterilized by wiping with rubbing alcohol.

Note: It is best to trim nails after a bath when they are soft, or while baby is sleeping.

CAUTION: Be careful not to trim too close to baby's sensitive skin.

Gentle Care Brush & Comb Set

The Gentle Care Brush & Comb Set features large, Soft Touch handles so parents and kids get a secure and comfortable grip. The hairbrush's soft bristles are extra gentle on baby's sensitive scalp while the comb's dual-teeth design lets you adjust for the thickness or dampness of your child's hair.

To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

Cradle Cap Comb

Many babies develop the temporary dry skin condition called "Cradle Cap." The dry skin usually appears on the scalp. Use the Cradle Cap Comb to gently remove the loosened skin from baby's scalp. The teeth on this comb may also be great for gently combing a newborn baby's extra-fine hair.

To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

Gentle Care Toddler Toothbrush

The Gentle Care Toddler Toothbrush can be placed on the table or sink and the convenient, roll-resistant handle keeps it from falling off.

To Use:

- 1 Begin use once all primary teeth are in or when recommended by your dentist.
- 2 Apply a pea-sized amount of toothpaste to the brush and have your child brush for 2-3 minutes using an up and down motion.
- 3 Continue to help and supervise your child during brushing - do not allow child to swallow toothpaste.

NOTE: Toothbrushes should be replaced every 2-3 months.

To Clean:

Rinse each brush thoroughly before first use and after each use.

Important Note: Questions about your child's oral health should be answered by your family dentist.

CAUTION: Adult supervision required. Do not allow child to run or walk while carrying or using toothbrush. Do not let children play with toothbrushes.

5 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.

FOR EXTERNAL USE ONLY.

Emergency Information Card

Record important phone numbers and medical information for quick reference.

CAUTION: These products are not toys. **DO NOT** allow children to play with them. When not in use, keep out of reach of children. **ALWAYS** supervise your child when using this kit. **NEVER** allow your child to use these products alone or without close adult supervision. We recommend that you store this kit in a locked cabinet or other location that is not accessible to a child. **FOR INDOOR HOME USE ONLY.**

3-in-1 Digital Thermometer

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.

Fahrenheit / Celsius

While thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This will take approximately 5 seconds.

How to Measure Temperature


Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

-
- 1 Remove probe tip cap. Press and release ON/OFF button. All segments display briefly as the thermometer powers on.
 - 2 The last temperature measured displays with the **M** icon on screen for a few seconds.
 - 3 **L** displays and °F flashes when the thermometer is ready to take measurement.

Specifics for each Temperature Mode

ORAL—For children 4 years and older.

1. Ensure that °F is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.

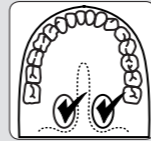
RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position baby so that the back and legs form a 90-degree angle (L-shape).
2. Ensure that °F is still flashing, then insert thermometer tip gently into the rectum, taking care not to insert past the insertion gauge (ring on the probe tip). Do not force the tip into rectum if resistance is encountered.

UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

1. Wipe underarm with a dry towel.
2. Ensure that °F is still flashing, then place probe tip at center of underarm. Bring arm down against the body. Probe must touch skin.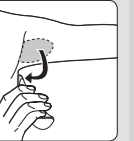

- When °F stops flashing, the measuring is complete. Approximate measurement times are: 60 seconds for oral and rectal temps, 90 seconds or longer for underarm temp.
- Remove probe tip from measurement site. Read and record the temperature and time for reference.
- Press button to shut off or thermometer will shut off automatically after 8-9 minutes.

NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included. **DO NOT** allow child to walk or run during temperature taking. Probe could present a hazard.

Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +

How to Replace the Batteries


Dead Battery Indicator:

- When the "□" icon appears, the battery is exhausted and needs to be replaced.
- Battery is to be inserted with the correct polarity.
- Check that all contact surfaces are clean and bright before installing battery.

To Replace Battery:

- 1 Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
- 2 Use a toothpick to remove battery.
- 3 Insert a new battery (LR41 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration.
- 4 Place the battery cover on the thermometer and secure the cover.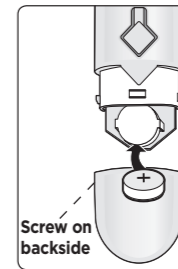

Care and Maintenance

Cleaning Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

Cleaning Thermometer Body:

- Use a soft, dry cloth to clean thermometer body.

Notes:

- **DO** remove battery if stored for a long period of time.
- **DO** store thermometer in a dry place at room temperature with the protective cap in place.
- **DO** take special care not to scratch the display window.
- **DO NOT** allow electronics to get wet.
- **DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning.
- **DO NOT** drop thermometer.
- Install new battery and retry function before calling Consumer Relations on electronics.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

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