



Oxivir® Tb Wipes

Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance.

Features & Benefits

- One-step cleaner disinfectant powered by AHP® technology
- Effective against key pathogens – TB, MRSA, Norovirus, VRE and *Candida auris*
- Kills HBV, HCV and HIV in 1 minute. Tuberculocidal in 5 minutes and Fungicidal in 3 minutes and 5 minutes for *Candida auris*
- No VOCs, no added fragrance and no NPEs. Environmentally responsible – active ingredient breaks down to oxygen and water
- Kills SARS-CoV-2 that causes COVID-19 on hard non-porous surfaces in 15 seconds
- Convenient – pre-wetted wipes make even the toughest job easier and compatible with most hard, non-porous surfaces

Applications

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin





Oxivir® Tb Wipes

Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

Use Instructions

As a One-Step Cleaner Disinfectant

- Pre-clean heavily soiled areas.
- Apply product by coarse trigger spray or disposable wipe to hard, non-porous inanimate surfaces.
- All surfaces must remain visibly wet for 1 minute.
- Use 5-minute contact time for TB and a 3 minute contact for fungi and 5 minute contact time for *Candida auris*. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Note: Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

Technical data		Oxivir® Tb Wipes
EPA Reg. No.		70627-60
Color/Form		Colorless, wipe
pH		2.2 - 3.2
Scent		Characteristic of ingredients
Shelf Life		3 years

Product	Pack size	Dilution	Product code	
Oxivir® Tb Wipes	12 x 160 ea, Small Wipes, Canisters	Ready-to-use	4599516	
Oxivir® Tb Wipes	4 x 160 ea, Small Wipes, Canisters	Ready-to-use	101105152	
Oxivir® Tb Wipes	12 x 60 ea, Medium Wipes, Canisters	Ready-to-use	5388471	
Oxivir® Tb Wipes	4 x 160 ea, XLarge Wipes, Buckets	Ready-to-use	5627427	
Oxivir® Tb Wipes	4 x 160 ea, XLarge Wipes, Refills	Ready-to-use	100823906	
Oxivir® Tb Wipes	120 x 10 ea, Medium Wipes, Soft Packs	Ready-to-use	101107341	

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

AHP® and Design are trademarks of Diversey, Inc.