

CC-90 AED ALARM OUTDOOR CABINET


INSTALLATION INSTRUCTIONS

Select an appropriate mounting kit for the cabinet's support structure. Ensure it can support at least 20kg.

Use M6 to M8 screws and a washer plate up to 14mm in diameter. The combined thickness of the screw and washer should not exceed 8mm.

Secure the cabinet using its three mounting holes.

Mounting Example

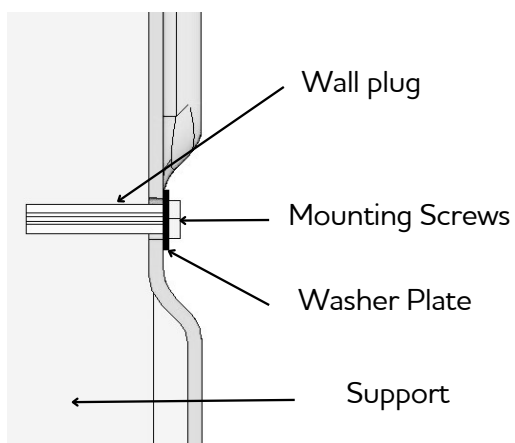

AED Mounting Instructions



Hook the strap in the holes on either side of the cabinet.
Pull the holding strap and slide the AED in place (shown above)

INSTRUCTIONS FOR ALARM INSTALLATION AND REPLACEMENT

This cabinet requires 2 x AA batteries

1. Remove alarm casing from cardboard packaging



2. Insert AA batteries



3. Locate the slots in the top right of the cabinet



4. Slide the alarm inside with the sticker facing downwards until it clicks into place.



5. Shut the door to activate the alarm.



Note that the alarm will continue to ring for 20 seconds after closing the cabinet