



# Oxivir® Plus

## Disinfectant Cleaner Concentrate

Dilutable one-step disinfectant cleaner, hydrotherapy tub cleaner and sanitizer, formulated with proprietary hydrogen peroxide technology. Ecologo certified.

### Features & Benefits

- Quick disinfection in just 5 minutes against Human Immunodeficiency Virus (HIV-1) (AIDS Virus), Polio virus, Influenza A Virus, *Pseudomonas aeruginosa*, *Staphylococcus aureus*, *Salmonella enterica*, MRSA and VRE
- 5 minute Bactericidal, Tuberculocidal, Virucidal, Fungicidal with a 30 sec 1:128 non food contact sanitizer claim
- AHP® active breaks down into water and oxygen after use
- No VOCs, no added fragrance and no NPEs
- Delivers fast and effective cleaning performance while being non-irritating to eyes and skin in use dilution

### Applications

- Clean and disinfect non-critical medical devices, equipment and hard, non-porous surfaces





### Use instructions

Oxivir® Plus Disinfectant Cleaner Concentrate is a one-step disinfectant cleaner using patented AHP® technology for use on walls, stainless steel, glazed porcelain, plastic surfaces, hydrotherapy tubs and other hard, nonporous surfaces.

- To disinfect, apply a 1:40 dilution to thoroughly wet surface.
- Let solution remain on surface for a minimum of 5 minutes.
- Air dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Prepare a fresh solution if the solution becomes visibly dirty or diluted.

### Dilutable Disinfectant Cleaner

- 1:40 dilution – Bactericidal and virucidal, including antibiotic-resistant bacteria, in 5 minutes.
- 1:128 dilution – Non-food contact sanitizer in 30 seconds or general purpose cleaning.

Technical data	Oxivir® Plus
DIN	02403684
Color/Form	Colorless, liquid
pH	0.8 (Concentrate) 1.9 (Use Dilution 1:40)
Scent	Characteristic of ingredients
Shelf Life	2 years (Concentrate), 30 days (Use Dilution)

Product	Pack size	Dilution	Product code	
Oxivir® Plus	4 x 3.78 L / 1 U.S. Gal. Containers	1:40, 1:128	5919024	🇨🇦
Oxivir® Plus	2 x 1.5 L RTD® Units / 50.7 oz.	1:40, 1:128	5919032	🇨🇦
Oxivir® Plus	2 x 2.5 L J-Fill® / 84.5 oz.	1:40	5919059	🇨🇦
Oxivir® Plus	12 x 946 mL / 32 oz. Empty Spray Bottles		D1225339	🇨🇦

### Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

Accelerated Hydrogen Peroxide™/MC and AHP® logos are trademarks owned by Virox Technologies used under license.