Ready! Growing Baby Nursery Kit

IH340, IH362

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

Fabric Storage Box

To assemble box, straighten all four sides of the box and put rigid base board into bottom, fabric side up. Surface wash only.

Bottle Medicine Dispenser

The Bottle Medicine Dispenser has easy-to-read dosage markings and measures up to 3 tsp. (15 ml).

To Use:

- 1 Remove top from measuring cup and fill cup to the prescribed dosage.
- 2 Snap top securely onto the measuring cup.
- 3 Bottle-feed child as normal, pausing as necessary so that child may swallow comfortably.

To Clean: Wash before first use and after every use. Disassemble top ring, nipple and measuring cup. Wash with warm, soapy water and rinse thoroughly. Alternatively, it is dishwasher safe. TOP RACK ONLY.

CAUTION: Inspect bottle nipple regularly for tears, cracks or other sign of wear and discontinue use if any are noted. Made without BPA.

Gentle Care Brush & Comb Set

The Gentle Care Brush & Comb Set features large, Soft Touch handles so parents and kids get a secure and comfortable grip. The hairbrush's soft bristles are extra gentle on baby's sensitive scalp while the comb's dual-teeth design lets you adjust for the thickness and dampness of your child's hair.

To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

Cradle Cap Brush & Comb

A cradle cap brush & comb will help you gently remove flaking skin from your baby's scalp. Cradle cap (also known as Seborrheic dermatitis) occurs on the scalp and is a common condition that affects infants; in most cases it is not serious or long-lasting.

To Use:

- 1 Apply a small amount of olive oil, baby oil, or shampoo to baby's head.
- 2 Slide brush onto your hand and place bristles on scalp. Apply gentle pressure, using a circular motion to massage the scalp and loosen the flakes.
- 3 With the fine-toothed comb, remove flakes and gently loosen tough flakes.
- 4 Shampoo and continue to use comb to help remove flakes. Rinse and repeat as needed. **Note:** If condition persists contact your pediatrician.

To Clean:

Wash in hot, soapy water after each use. Rinse with warm water and let drip dry.

Newborn Nasal Aspirator

A Our Newborn Nasal Aspirator quickly and conveniently removes excess mucus so your baby can breathe comfortably again.

To Use:

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean: Aspirator is top-rack dishwasher safe or can be washed in warm soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

CAUTION: NEVER push the aspirator too far into the nostril and always wash and dry thoroughly after each use. NEVER use in baby's mouth or ears. When storing, keep away from heat and bright light.

5 Digital Thermometer Probe Covers

NOTE: Use thermometer probe covers when taking temperature rectally; not for oral use (see Warning section).

- 1 Hold edge of cover and slip over end of thermometer. Pull the outside protective cover off. Make sure no air remains between the probe and the probe cover.
- 2 **Rectal Use:** Lubricate the thermometer probe cover with a water-soluble lubricant or petroleum jelly. Follow instructions under "Rectal Use." Discard thermometer probe cover after use. Do not reuse probe covers.

NOTE: To purchase thermometer probe covers call Consumer Relations at (800) 544-1108, and ask for part number 49520.

Fingertip Toothbrush/Gum Massager

To Use:

- 1 Place on adult's fingertip.
- 2 Use the gum massager (nubby side) to help soothe baby's sensitive gums and the infant toothbrush side to clean emerging teeth and soothe tender gums.

Use and Care: Rinse thoroughly before first use and after each use.

CAUTION: Do not allow children to play with it. When not in use, keep out of reach of children. **ADULT SUPERVISION REQUIRED.**

Fold-Up Nail Clipper & 5 Emery Boards

These clippers have been specially designed with an easy-grip extension for convenient adult use. Lift arm from hand grip and rotate it 180°. Position nail between cutting edges and squeeze handle to cut nail. The curved cutting edges are perfect for trimming little nails closely. Emery board use is recommended to smooth any remaining sharp nails after clipping and also to smooth minor rough or jagged fingernail edges at other times.

To Clean: Wipe clippers with a damp cloth and dry after each use. Clippers may be sterilized by wiping with rubbing alcohol.

Note: It is best to trim nails after a bath when they are soft, or while baby is sleeping.

CAUTION: BE CAREFUL NOT TO TRIM TOO CLOSE TO BABY'S SENSITIVE SKIN!

4 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.


FOR EXTERNAL USE ONLY.

CAUTION: These products are not toys. **DO NOT** allow children to play with them. When not in use, keep out of reach of children. **ALWAYS** supervise your child when using this kit. **NEVER** allow your child to use these products alone or without close adult supervision. We recommend that you store this kit in a locked cabinet or other location that is not accessible to a child. **FOR INDOOR HOME USE ONLY.**

Digital Thermometer with Protective Case

WARNING: Before using digital thermometer, please read carefully. • This is **NOT** an ear thermometer. **NEVER** put probe into ear. • Take temperature measurements Underarm, Orally or Rectally. • **NEVER** allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision. • **DO NOT** expose the thermometer to extreme temperatures, humidity, direct sunlight or shock, and/or dust. • Keep battery out of child's reach. Swallowing a battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode. • Used batteries should be disposed of properly. • **POTENTIAL CHOKING HAZARD.** Do not leave child unattended when a thermometer probe cover is in use. • Probe covers are not recommended for oral use. • Keep probe covers out of reach of children. • Probe covers are intended for single use only. • **ALWAYS** supervise your child when using thermometer and thermometer probe covers. **NEVER** allow your child to use it alone or without close parental supervision.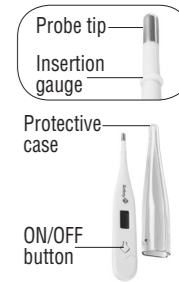


To Set Up Thermometer Before Use

Note: Ensure that thermometer is at room temperature, wait 15 minutes if needed.

- 1 **Set to Fahrenheit/ Celsius:** While Thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This should take approximately 5 seconds.
- 2 **Prepare for Use:** Press ON/OFF button. The last temperature measured will display with an M icon on the screen for a few seconds. When the L icon and measure of temperature appear, thermometer is ready to take temperature.

Battery Replacement

- 1 Indicator: When the "⏏" icon appears, the battery needs to be replaced.
 - 2 Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
 - 3 Use a toothpick to remove battery.
 - 4 Insert a new battery (LR41 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration.
 - 5 Place the battery cover on the thermometer and secure the cover.
- CAUTION:** Battery is to be inserted with the correct polarity. Check that all contact surfaces are clean and bright before installing battery.

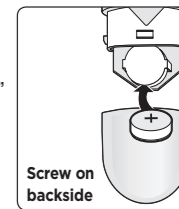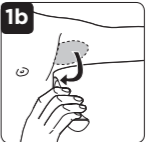
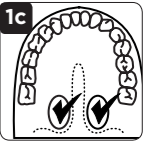
Important Information

• Read these instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading, and technique.

To Take Temperature

Please read instructions for each mode of taking temperature. **Note:** Ensure °F/°C is still flashing before taking temperature.

1. Choose Temperature Mode based on child's age:

Rectal-Birth & up	Underarm-under 4 years	Oral-4 years & up												
<p>Note: Highly accurate for measuring core body temp. Approximately 60 seconds to measure.</p>  <ol style="list-style-type: none"> 1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. 2. Position baby so that the back and legs form a 90° angle or L-shape (Figure 1a). 3. Insert tip gently into the rectum, until you reach insertion gauge (ring on probe tip). Do not force the tip into the rectum if there is resistance. 	<p>Note: Ensure underarm is dry, wipe if needed. Approximately 90 seconds to measure.</p>  <p>Place probe tip at center of underarm (Figure 1b). Bring arm down against the body and ensure probe is touching skin.</p>	<p>Note: Approximately 60 seconds to measure.</p>  <p>Place probe tip under the tongue as indicated by ✓ (Figure 1c). Instruct child not to bite down on probe.</p>												
<p>2. Record Measurement: When °F/°C stops flashing, measuring is complete. Remove probe tip, read and record temperature and time. Thermometer will shut off automatically after 9 minutes if not shut off manually. NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included below. DO NOT allow child to walk or run during temperature taking, probe could present a hazard.</p>	<p>3. Review Temperature: Temperature readings vary from person to person and depending upon the body location from which the temperature is measured. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading as well as technique. Figure 3 shows temperature ranges on average.</p>	<p>3</p> <table border="1"> <thead> <tr> <th>TEMPERATURE METHOD</th> <th>NORMAL TEMPERATURE</th> <th>FEVER RANGE</th> </tr> </thead> <tbody> <tr> <td>Oral</td> <td>97.0° - 99.6°F</td> <td>99.6°F +</td> </tr> <tr> <td>Rectal</td> <td>97.3° - 100.4°F</td> <td>100.4°F +</td> </tr> <tr> <td>Underarm</td> <td>95.4° - 98.6°F</td> <td>98.6°F +</td> </tr> </tbody> </table>	TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE	Oral	97.0° - 99.6°F	99.6°F +	Rectal	97.3° - 100.4°F	100.4°F +	Underarm	95.4° - 98.6°F	98.6°F +
TEMPERATURE METHOD	NORMAL TEMPERATURE	FEVER RANGE												
Oral	97.0° - 99.6°F	99.6°F +												
Rectal	97.3° - 100.4°F	100.4°F +												
Underarm	95.4° - 98.6°F	98.6°F +												
<p>4. What is "Normal": "Normal" temperature refers to an individual's average temperature while healthy. You should measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your child.</p>	<p>Care & Maintenance</p> <ul style="list-style-type: none"> • Clean probe tip before and after each use. • Use an alcohol swab to wipe clean. • Use a soft, dry cloth to clean thermometer body. • DO remove battery if stored for a long period of time. • DO store thermometer in a dry place at room temperature with the protective cap in place. • DO take special care not to scratch the display window. • DO NOT allow electronics to get wet. • DO NOT use abrasive cleaning agents, thinners or benzene for cleaning. • DO NOT drop thermometer. • Install new battery and retry function before calling Consumer Relations on electronics. 	<p>Technical Specifications</p> <p>Measurement Range: 90° F to 109.9° F</p> <p>Measurement Accuracy: +/- 0.2° F for temps below 96° F or above 107.6° F +/- 0.4° F for temps 60.8° F to 104° F</p> <p>Operation Environment: 60.8° F to 104° F</p> <p>Storage Environment: -13° F to 144° F</p> <p>Test Protocol:</p> <ol style="list-style-type: none"> 1 Put the tip of thermometer into water bath, approximately 2 cm below the surface. 2 Wait for 10 seconds, then turn on the thermometer by pressing the ON/OFF button. The thermometer is now in standard mode. 3 When the measurement is completed (approximately 60 seconds) record the temperature reading. 												

Care & Maintenance

- Clean probe tip before and after each use.
- Use an alcohol swab to wipe clean.

• Use a soft, dry cloth to clean thermometer body. • **DO** remove battery if stored for a long period of time. • **DO** store thermometer in a dry place at room temperature with the protective cap in place. • **DO** take special care not to scratch the display window. • **DO NOT** allow electronics to get wet. • **DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning. • **DO NOT** drop thermometer. • Install new battery and retry function before calling Consumer Relations on electronics.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Ecuador Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse. This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

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