Healthcare On-the-Go Kit IH535

Reversible Case

The reversible case keeps all of baby's healthcare items in one convenient place. In the cube shape, this case has its greatest capacity for bulky items. Reversed into the flat, pencil-case shape, it still holds all of the kit components but fits more compactly into a purse, diaper bag or suitcase.

To reverse from cube to pencil-case: Turn unzipped case inside-out and gently pull on ends, to straighten and flatten the case. Bring zipper pull to the outside of the case.

To reverse from pencil-case to cube: Turn unzipped case inside-out and gently re-shape ends, to "square off" and form the ends of the cube. Bring zipper pull to the outside of the case.

Use and Care: Spot clean. Wipe with a damp cloth.

Medicine Dropper

Safety 1st understands that giving a child medicine takes special attention. Our Medicine Dropper is specially designed to provide accurate dosage readings. The convenient spill guard plug helps reduce messy spills at home or away.

To Use:

- 1 Remove spill guard from dropper tip and squeeze air out of dropper.
- 2 Insert dropper into the medicine bottle and slowly release pressure on the bulb until the medicine reaches the desired dosage marking.
- 3 Remove dropper from the medicine bottle and tip upside down while securing the spill guard over the dropper tip.
- 4 Administer medicine while child is in an upright sitting position. Remove spill guard and point tip of dropper to the inside of baby's cheek and gently squeeze the bulb.
- 5 Administer medicine slowly enough so your child may swallow comfortably. Pause and continue if necessary.

To Clean:

Wash with warm, soapy water before first use and after each use. Rinse thoroughly.

CAUTION: KEEP ALL MEDICATIONS OUT OF CHILD'S REACH. WE RECOMMEND THAT YOU USE A CHILD RESISTANT TOP TO SEAL THE MEDICINE BOTTLE AND STORE IT IN A LOCKED CABINET OR OTHER LOCATION THAT IS NOT ACCESSIBLE TO A CHILD.

Newborn Nasal Aspirator

Our Newborn Nasal Aspirator quickly and conveniently removes excess mucus so your baby can breathe comfortably again.

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.

To Clean:

Clean before first use and after each use. Gently squeeze air from Aspirator. Rinse the bulb in warm, soapy water. Squeeze and release bulb several times to thoroughly wash interior of Aspirator. Rinse in the same fashion with warm, clean water. Make certain that all of the water is drained during drying. Avoid using abrasive pads or cleaners. DO NOT place in microwave.

CAUTION: NEVER PUSH THE ASPIRATOR TOO FAR INTO THE NOSTRIL AND ALWAYS DRY THOROUGHLY AFTER EACH USE. NEVER USE IN BABY'S MOUTH OR EARS. WHEN STORING, KEEP AWAY FROM HEAT AND BRIGHT LIGHT.

5 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.

FOR EXTERNAL USE ONLY.

Emergency Information Card

Record important phone numbers and medical information for quick reference.

CAUTION:

THESE PRODUCTS ARE NOT TOYS. DO NOT allow children to play with them. KEEP OUT OF REACH OF CHILDREN. FOR INDOOR HOME USE ONLY. Products should only be used with adult supervision. When not in use, keep out of reach of children.

©2023 Dorel Juvenile Group, Inc. Safety 1st™ and the Safety 1st logo and design are trademarks of Dorel Juvenile Group, Inc. (800) 544-1108 www.safety1st.com. Styles, colors, and labeling may vary. Los estilos, colores y etiquetado pueden variar. Made in CHINA. Hecho en CHINA. Distributed by (distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2233 Argentinia Road, Suite 110, Mississauga, ON L5N 2X7. Representante exclusivo en Chile: DOREL JUVENILE CHILE S.A. Av. del Parque 4161, oficina 602 A, Huechuraba, Santiago, Chile - Fono: 56 2 24971500 - www.infanti.cl - www.dorel.cl. Importado y distribuido en Perú por: COMEXA COMERCIALIZADORA EXTRANJERA S.A. SUCURSAL DEL PERÚ - Los Libertadores 455, San Isidro, Lima 27, Lima, Perú RUC 20431062870 - Fono: 4413029 - www.infanti.com.pe. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Via 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle, Benito Juárez C.P. 03100 México, Ciudad de México R.F.C. DJG140305GN9 Tel. (55) 67199202 11/23 4358-7989C

3-in-1 Digital Thermometer

The Safety 1st Digital Thermometer takes a clear and accurate temperature without the worries of a glass/mercury device. Effective for oral, rectal or underarm use.

TH093

Fahrenheit / Celsius

While thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This will take approximately 5 seconds.

How to Measure Temperature


Ensure that thermometer is at room temperature, waiting 15 minutes if necessary.

- Press and release ON/OFF button. All segments display briefly as the thermometer powers on.
- The last temperature measured displays with the **M** icon on screen for a few seconds. Then the thermometer will show an average temperature (37.0.9°C or 98.5.9°F) for 0.5 seconds before entering ready mode.
- **Lo** displays and °F flashes when the thermometer is ready to take measurement.

Specifics for each Temperature Mode

ORAL—For children 4 years and older.

1. Ensure that °F is still flashing, then place probe tip under the tongue in locations indicated by ✓. Instruct children not to bite down on the probe.

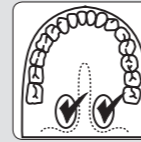
RECTAL—Highly accurate for measuring core body temperature. Pediatrician recommended mode from birth to age 1, and beyond as tolerated by child.

1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. For easier insertion, position baby so that the back and legs form a 90-degree angle (L-shape).
2. Ensure that °F is still flashing, then insert thermometer tip gently into the rectum, taking care not to insert past the insertion gauge (ring on the probe tip). Do not force the tip into rectum if resistance is encountered.

UNDERARM—For child under age 4 who will not cooperate with rectal temperature measurement.

1. Wipe underarm with a dry towel.
2. Ensure that °F is still flashing, then place probe tip at center of underarm. Bring arm down against the body. Probe must touch skin.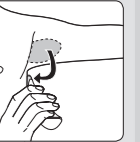

- When °F stops flashing, the measuring is complete. Approximate measurement times are: 60 seconds for oral and rectal temps, 90 seconds or longer for underarm temp.
- Remove probe tip from measurement site. Read and record the temperature and time for reference.
- Press button to shut off or thermometer will shut off automatically after 8-9 minutes.

IMPORTANT INFORMATION

Read these instructions carefully to ensure accurate temperature readings.

Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading as well as technique. NOTE: High, prolonged fever requires medical attention, especially for young children. Please contact your physician. Fever chart included.

DO NOT allow child to walk or run during temperature taking. Probe could present a hazard.

Normal Body Temperature

Temperature readings are different from person to person and also vary depending upon the body location from which the temperature is measured. For example: Oral temperatures tend to be comparable to ear temperatures and approximately 1° lower than rectal temperatures for the same individual. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading as well as technique.

"Normal" temperature refers to each individual's average temperature while healthy. You may want to measure and record your child's temperature a few times when he or she is well, so that you will know what is normal for your own child. On average, temperature ranges are as follows:

| TEMPERATURE METHOD | NORMAL TEMPERATURE | FEVER RANGE |
|--------------------|--------------------|-------------|
| Oral | 97.0° - 99.6°F | 99.6°F + |
| Rectal | 97.3° - 100.4°F | 100.4°F + |
| Underarm | 95.4° - 98.6°F | 98.6°F + |


How to Replace the Batteries

Dead Battery Indicator:

- When the "□" icon appears, the battery is exhausted and needs to be replaced.
- Battery is to be inserted with the correct polarity.
- Check that all contact surfaces are clean and bright before installing battery.

To Replace Battery:

- 1 Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
- 2 Use a toothpick to push the battery out of the metal housing.
- 3 Insert a new battery (LR41 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration.
- 4 Place the battery cover on the thermometer and secure the cover.

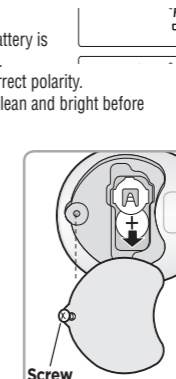Care and Maintenance

Cleaning Probe Tip:

- Clean before and after each use.
- Use an alcohol swab to wipe clean.

Cleaning Thermometer Body:

- Use a soft, dry cloth to clean thermometer body.

Notes:

- **DO** remove battery if stored for a long period of time.
- **DO** store thermometer in a dry place at room temperature with the protective cap in place.
- **DO** take special care not to scratch the display window.
- **DO NOT** allow electronics to get wet.
- **DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning.
- **DO NOT** drop thermometer.
- Install new battery and retry function before calling Consumer Relations on electronics.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or inconsequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

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