

ID Bands


Clear Band

with Color Insert Cards




To Use:


1. Type or write patient information on insert card starting at the perforation. Tear card from sheet.

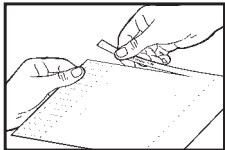

2. Estimate proper length by wrapping bracelet loosely around patient's wrist, leaving two fingers "comfort space" underneath band. Trim excess from band with scissors.

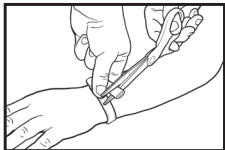

3. Trim insert to fit inside band. Slide card inside band and tear off insert tab.

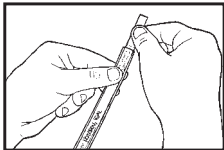

4. Wrap bracelet around wrist with patient information facing outward. Place connector into open end of band. Press firmly with a side to side motion until each end of band touches ridge in center of connector.



SYMBOL GLOSSARY

For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

