



+ Wall-Mounted Workstations

Lock Guide

Featuring Solutions From

Digilock®

Carstens® is proud to partner with Digilock®, a company that also believes in taking care of caregivers. For over 40 years, Digilock has led the way in technological innovation, design, and manufacture of personal security solutions. In hospital systems and healthcare facilities around the world, Digilock systems ensure that both everyday items and the most sensitive of supplies remain secure.



CarePod™ Narrow

Shown in River Rock with Driftwood laminate and a keypad lock

Lock Option Guide

3 Lock Styles

CarePod™ Wall-Mounted Workstations offer a choice of secure locking options to accommodate your facility's security requirements. Also available lockless upon request.



Key Lock



Keypad



RFID



Digilock® Versa Mini

Keypad

An efficient keyless security option, simply enter a 4-7 digit code on the alphanumeric keypad, then turn the knob to open. Each lock can be programmed with multiple unique access codes.

- Bold alphanumeric keypad buttons with visual and audible indicators

Operation	4-7 digit assigned code
Battery	2 lithium CR2450 batteries
Battery Life	3-5 years




Digilock® Versa Mini

Touch RFID

An ideal solution for facilities with existing electronic credentialing systems, just touch a security card or fob against the sensor, then turn the knob to open. Each lock can be programmed to provide or limit access to specific users as needed.

- Compatible with most major RFID systems

Operation	RFID iClass and Mifare (13.56 MHz)
Battery	4 premium AA batteries
Battery Life	3-5 years

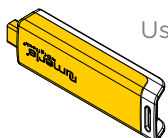

User codes managed/
programmed via
electronic keys

**20-500
users**

Up to 20 or 500 unique
user codes per lock
(depending on lock
management choice)



Automatically locks
when door is closed

Management Systems

Basic

This simple, cost-effective management solution allows for up to **20 unique users and up to 6 managers** across all CarePod Wall-Mounted Workstations in your facility.

Each facility lock system requires (1) yellow Programming Key to set up each Basic lock. A minimum of one (1) black Manager Key is required to set or change user codes and manage access for a Basic lock, with a maximum of (6) six black Manager Keys allowed per individual lock. All keys are sold separately.



Keys sold separately.

Actual key measures:
L5.125"
W1.3125"
D.75"

Manager Key¹

Programming Key²

Operates the lock	•	•
Overrides user access	•	•
Allows management inspection	•	•
Provides external power	•	•
Programs Manager Keys to the lock		•
Assigns user credentials	•	
Sets lock functionality		•

¹Up to six (6) black Manager Keys may be programmed to each lock

²One (1) yellow Programming Key is permitted per lock system or facility

Advanced

This management system offers audit trail capabilities and **accommodates up to 500 unique members** (250 users, 250 managers). It requires of one (1) white Data Key and one (1) Android tablet with Digilock's Numeris app preinstalled per facility.

With the Advanced system, you can set up CarePod lock groups, manage user access, conduct maintenance, and run diagnostics via the tablet app. It also can pull an **audit trail report of the last 500 lock operations** for any individual CarePod installed at your facility. Previous reports can be saved to the app or backed up to a computer.

This centralized system doesn't require network access—eliminating data security concerns. Advanced CarePod locks are designed to work with RFID badges and/or black Manager Keys.



Click to learn more



Digilock® Support

All CarePod™ Wall-Mounted Workstations featuring Digilock electronic locks come with the reassurance of phone and live-chat technical support. For help, visit digilock.com/contact to chat live with a Digilock team member, Monday – Friday, 7:00 am to 5:00 pm PST or call 1-800-989-0201 and select Option 3.



2-Year Limited Warranty

LIMITED WARRANTY. Security People Inc. dba Digilock (Company) warrants to the original purchaser of Product manufactured by Company to be free of defects in material and workmanship, provided: (1) Company has been notified of such defects within two years of purchase date and been given the opportunity of inspection by return of any alleged defective Product to Company, or its authorized distributor, free and clear of liens, encumbrances, transportation prepaid, accompanied by the statement of defects and proof of purchase; and (2) Product has not been subject to modification, abuse, misuse, mishandling, misapplication, neglect (including but not limited to improper maintenance), accident, improper installation, alteration and/or modification (including but not limited to use of unauthorized parts or attachments), disassembly, or adjustment or repair performed by anyone other than authorized service personnel during such period; and (3) such defect has not been caused by corrosion or ordinary wear and tear.

This warranty does not cover batteries, normal wear and tear of parts and/or damage resulting from any of the following: negligent use or misuse of product, use of improper voltage or current, and/or use contrary to operating instructions. The exclusive remedy shall not be deemed to have failed of its essential purpose so long as Company is willing and able to repair or replace non-conforming Product in the prescribed manner.

Unless otherwise expressed by Company on the sales order acknowledgment, Product is not designed or intended for exterior use or where exposed to moisture. Any use where Product is exposed to moisture is not covered by any warranties and voids any warranties and any resulting damage caused by direct exterior exposure or moisture is at the Buyer's own risk.

The liability of Company for any breach of the foregoing warranty shall not extend to dismantling, installing or reinstalling, but shall be limited to repairing or replacing said items, as Company's option, within a reasonable time after receipt of written notice from Buyer, within the period of time stipulated herein, of any such non-conforming Product, provided that (i) such notice is given immediately upon the detection of such non-conformance and (ii) such notice is received within the time period stipulated herein. BUYER AND COMPANY EXPRESSLY AGREE THAT THE BUYER'S SOLE AND EXCLUSIVE REMEDY AGAINST COMPANY SHALL BE FOR THE REPAIR OR REPLACEMENT (AT COMPANY'S OPTION) OF NON-CONFORMING PRODUCT AS PROVIDED HEREIN.

COMPANY SPECIFICALLY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS, IMPLIED, OR STATUTORY, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTY OF FITNESS FOR A PARTICULAR PURPOSE AND THE IMPLIED WARRANTY OF MERCHANTABILITY, AND COMPANY ALSO SPECIFICALLY DISCLAIMS ANY LIABILITY FOR CONSEQUENTIAL OR INCIDENTAL DAMAGES. IN NO EVENT SHALL COMPANY BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF OR IN CONNECTION WITH THIS AGREEMENT (INCLUDING BUT NOT LIMITED TO BREACH OF ANY OBLIGATION IMPOSED ON COMPANY HEREUNDER OR IN CONNECTION HERewith) OR THE USE, STORAGE, DISPOSAL OR HANDLING OF PRODUCT. Consequential damages for purposes hereof shall include, without limitation, loss of use, income or profit, shutdown, slowdown or stoppage of any production line or facility, or losses sustained as the result of injury (including death) to any person, or loss of or damage to property (including without limitation property handled or processed by the use of Product). Buyer shall indemnify Company against all liability, cost or expense which may be sustained by Company on account of any such loss, damage or injury.

Company will not evaluate warranted Product without Buyer obtaining a Return Merchandise Authorization (RMA) number from Company. Such returns must be prominently marked with the RMA number and shipped prepaid.

Company makes no other warranty, and the warranty is limited to the duration of the expressed warranty period as set forth above.

NOTE: Should Product be considered a consumer product as may be covered by the Magnusson Moss Federal Warranty Act, please be advised that: (1) some states do not allow limitations on incidental consequential damages or how long an implied warranty lasts so that the above limitations may not fully apply; (2) this warranty gives specific legal rights, and a buyer may also have other rights which may vary from state to state.

LIMITATION OF LIABILITY. Company's maximum liability for any damages resulting from or caused by Product, whether in contract, tort, or otherwise, is limited to the purchase price of Product. In no event shall Company be liable for any consequential damages of any nature arising from the sale or use of Product, whether in contract, tort or otherwise, by either use or purchase of Product, the user or purchaser agrees to this limit on Company's liability. Company is not responsible for any modification, addition or alteration to Product by others.

Company reserves the right to make changes in designs and specifications or to make additions or improvements on its products without notice and without incurring any obligation to incorporate them on products previously manufactured.



Download additional resources at carstens.com/carepod

For more information:
800.782.1524 / carepodsales@carstens.com
©Carstens, Inc. 03.2021
Literature made in the U.S.A.
Content subject to change.

