Important: Be sure to retain this manual for future reference and read it completely before using these products.

MILK COLLECTION BOTTLES

To Clean: Before first use, boil all parts for 5 minutes to sanitize and cool to room temperature (at least 5 minutes). May also be used with a microwave sanitation device. After each use wash all parts thoroughly in warm, soapy water. Rinse and dry thoroughly. Dishwasher-safe (top rack only).

To Thaw & Feed Breast Milk: Remove frozen breast milk from freezer and allow to thaw in refrigerator several hours before use. Before feeding baby, hold the sealed bottle under warm running water until milk is at room temperature. Gently invert the sealed bottle several times to mix the milk fat that has risen to the top during storage. Always test temperature of liquid before feeding baby.

To Freeze: Pump directly into storage bottle or pour expressed milk into storage bottles. Fill containers 3/4 full, as milk expands with freezing. Ensure sealed bottle by pushing round silicone disc through top of cap ring, until it is snugly seated to the cap ring. Screw cap ring onto bottle tightly. Write date and time milk was expressed on bottle using a grease pencil, or write on tape and apply to bottle. See the Breast Milk Storage Guidelines chart for storage time limitations.

If Feeding Directly From the Collection Bottle: You must use Evenflo Balance + Standard Nipples. Other brands or types of nipples will not work effectively with the included cap ring. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the TOP of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care to not over or under tighten. Evenflo Balance + Standard Nipples sold separately.

⚠ WARNING: Do not heat in microwave oven. Microwaving can destroy important nutrients in breast milk and can heat liquids quickly and unevenly. Burns or scalding may result. Breast milk should not be refrozen once it has been thawed. Do not pour freshly pumped breast milk onto frozen breast milk because thawing can occur.

INSULATED COOLER BAG

To Use: Place the sealed bottle(s) and frozen ice packs in the included cooler bag and zip close. When frozen ice packs are used and the cooler bag is kept at room temperature of 68 - 77°F (20 - 25°C), breast milk can be safely stored for up to 10 hours.

Care Instructions: Wipe clean with a damp cloth. Surface wash only.

Note: Avoid storing the cooler bag in a hot closet, cabinet, or car.

⚠ WARNING: For Adult Use Only. Keep out of reach of children. Discontinue use and replace if any tear or crack appears.

ICE PACKS

To Use: Place ice packs in freezer. Once frozen, place them in cooler bag with breast milk to keep milk cold.

This product, including the fluid inside ice packs, is non-toxic.

⚠ WARNING: For adult use only. Keep out of reach of children. Discontinue use and replace if any tear or puncture appears.

BREAST MILK STORAGE GUIDELINES*

	Freshly Expressed Breastmilk	Thawed Breastmilk (previously frozen)
Room Temperature	4 hours at 73 - 77°F (23 - 25°C)	DO NOT store
Refrigerator	5 - 7 days at 32 - 39°F (0 - 4°C)	24 hours
Self-contained Refrigerator Freezer	3 - 4 mos. at 0°F (-19°C)	NEVER re-freeze thawed milk
Deep Freezer	6 - 12 mos. at 0°F (-19°C)	NEVER re-freeze thawed milk

Note: Breast milk should be stored in the main refrigerator or freezer compartment, away from the door, to avoid changes in temperature.

*Human Milk Banking Association of North America. Best practices for expressing, storing and handling human milk in hospitals, homes and child care settings. Raleigh, NC: 3rd Edition, 2011

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©2019 EVENFLO FEEDING, INC.