ADVANCED SOLUTIONS™


Electronic Nasal Aspirator

User's Guide IH252, IH368


Thank you for purchasing this Safety1st® Advanced Solutions Electronic Nasal Aspirator. We're proud to be your choice to help you keep your child healthy and safe. Please be sure to read all warnings, instructions and notations before using this product.


Basics of Your Nasal Aspirator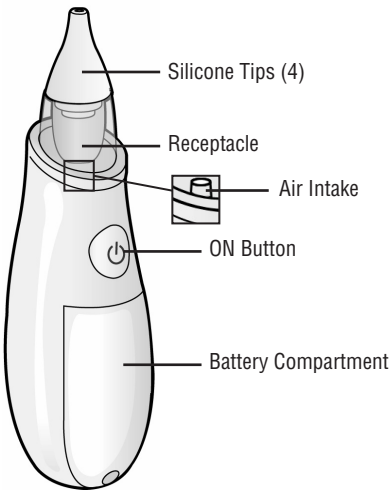

How to Replace the Batteries

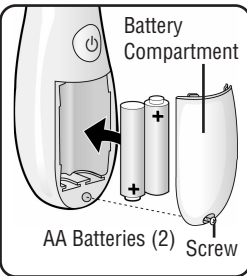- Use a small screwdriver to loosen screw. Remove battery cover.
- Use care when removing batteries to avoid damage to the battery contact surfaces.
- Insert 2 AA batteries (included). Position batteries correctly with the positive "+" side facing, as shown, in the battery compartment. Tilt negative side of the battery in first then the positive side as shown. This will help keep battery contacts in proper alignment.
- Place the battery cover on the unit and secure the screw.

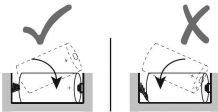WARNING

- **NEVER** push the aspirator too far into the nostril.
- **ALWAYS** wash and dry tips and receptacle thoroughly before each use and after each use.
- **NEVER** use in baby's mouth or ears.
- This product is not a toy - do not allow children to play with it. When not in use, keep out of reach of children. This product contains probes which are small parts.
- When storing, keep away from heat and bright light.
- **DO NOT** submerge base unit containing electronics in water.
- Dispose of batteries safely.
- Remove batteries when stored for long periods of non-use.


BEFORE USING THIS NASAL ASPIRATOR, PLEASE READ THE FOLLOWING CAREFULLY.

BATTERY WARNING


- Keep these instructions as they contain important information.
- **DO NOT** mix old and new batteries.
- **DO NOT** mix alkaline, standard (carbon zinc), or rechargeable (nickel-cadmium) batteries.
- Non rechargeable batteries are not to be recharged.
- Rechargeable batteries are to be removed from the unit before being charged.
- Rechargeable batteries are only to be charged under adult supervision.
- Only batteries of the same or equivalent type as recommended are to be used.
- Batteries are to be inserted with the correct polarity.
- Exhausted batteries are to be removed from the unit.
- The supply terminals are not to be short-circuited.

How to Use Your Nasal Aspirator

Please read this section carefully and all the way through before using this product.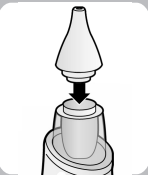


1. Press clean silicone tip onto Nasal Aspirator.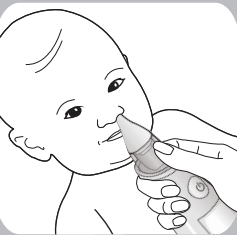


2. Have both child and aspirator in an upright position, and gently insert the aspirator tip until the tip seals the nostril. Ensure that the tip is properly positioned. If needed, allow the child to recline back and tilt the aspirator slightly. Ensure the ON button is facing upwards at all times to prevent mucus from blocking the air intake.

Helpful Hints:


- Block the opposite nostril for more effective suctioning.
- A bit of saline may help to loosen mucus.
- Opening baby's mouth can help reduce ear pressure.
- **DO NOT** use with ON button facing downward to prevent mucus leakage.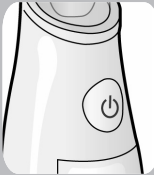

3. Press the ON button and hold for a few seconds to suction mucus into the reservoir. Just a few seconds of suctioning are necessary. Release ON button and remove tip from nostril.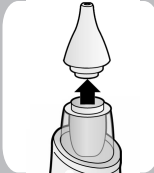

4. Remove tip and receptacle from Nasal Aspirator. Clean Nasal Aspirator as described in Care and Maintenance section.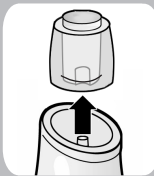


Care and Maintenance

Cleaning Silicone Tips:

- Wash in warm soapy water and rinse well.
- Dishwasher safe in a dishwasher basket on the top rack only.
- Optional: Microwave 90 seconds on high (tip must cool completely before use).

Cleaning Receptacle:

- Clean out receptacle using an ear swab dipped in rubbing alcohol.
- When receptacle is dry, reinstall it into base unit. You should hear a click when it is properly in place and the red line on the receptacle should line up with the yellow ring (Figure A).
- **DO NOT** place in microwave.

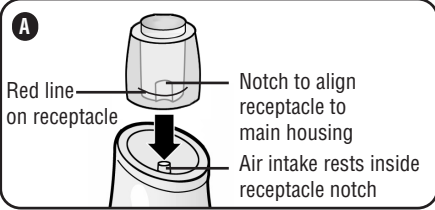Cleaning Base Unit:

- **DO NOT** submerge base unit with electronics in water. Wipe base unit clean only.
- **DO NOT** place in microwave.

Troubleshooting

Problem	Possible Cause	Solution
Low Suction	Tip is blocked	Clean tip
	Exhausted batteries	Replace batteries
Liquid Leaking	Air intake is blocked	Hold aspirator upright or tilted slightly with ON button facing upwards

Thank you again for purchasing a Safety 1st product. We are not only about safety, we're also about your satisfaction. If you have any questions or comments, please call our Consumer Relations department at 1-800-544-1108 or e-mail us at consumer@safety1st.com.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Advanced Solutions Electronic Nasal Aspirator) is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship, Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse.

This warranty excludes any liability other than that expressly stated previously, including but not limited to any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

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©2016 Dorel Juvenile. (800) 544-1108 www.safety1st.com