



## Oxivir® Tb Wipes

General Virucide,  
Bactericide,  
Tuberculocide,  
Fungicide,  
Sanitizer



A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP® technology to deliver fast, effective cleaning performance.

### Features

- One-step cleaner disinfectant
- Powered by AHP® technology
- No VOCs, no added fragrance and no NPEs
- Environmentally responsible – active ingredient breaks down to oxygen and water

### Benefits

- Effective against key pathogens – TB, MRSA, Norovirus and VRE
- Kills HBV, HCV and HIV in 1 minute. Tuberculocidal in 5 minutes and Fungicidal in 10 minutes
- Compatible with most hard, non-porous surfaces
- Convenient – pre-wetted wipes make even the toughest job easier

### Applications

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin



# Oxivir® Tb Wipes

General Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer



## Use Directions: As a One-Step Cleaner Disinfectant

1. Pre-clean heavily soiled areas
2. Apply product by coarse trigger spray or disposable wipe to hard, non-porous inanimate surfaces
3. All surfaces must remain visibly wet for 1 minute
4. Use 5-minute contact time for TB and a 10-minute contact time for fungi. Wipe surfaces dry, rinse, or allow to air dry.

**Note:** Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey Care representative.

## Product Specifications

Description	
EPA Reg. No.	70627-60 (Canisters, Buckets, Refills) 74559-3 (Soft Packs)
Color/Form	Colorless
pH	3
Scent	Characteristic
Shelf Life	3 years

## Available Items

SKU Code	Description/Package Size	Dilution
4599516	12 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-Use
5388471	12 x 60 count 7 x 8 in. Wipes / 17.8 x 20.3 cm Canisters	Ready-to-Use
5627427	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Buckets	Ready-to-Use
100823906	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Refills	Ready-to-Use
100949406	120 X 10 count 7.48 X 10 in. Wipes / 18.99 X 25.4 cm Soft Packs	Ready-to-Use
9P700827	6 x 1 Unit (Wall Bracket for use with 60 count Canisters)	
9P700798	1 Double Unit Non-Locking Stand	
9P700797	1 Double Unit Locking Stand	
9P700799	6 x 1 Unit Wire Bracket Locking on Plastic Wall Mount (for use with 160 count Canisters 6"X7")	
9P700800	6 x 1 Unit Wire Bracket Non-Locking on Plastic Wall Mount (for use with 160 count Canisters 6"X7")	
9P700829	4 x 1 Unit Large Bucket Wall Mount Bracket (for use with 160 count Buckets)	

## Safety Reminder

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use, and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDSs are available on-line at [www.sealedair.com](http://www.sealedair.com) or by calling 888 352 2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up to date shipping information.