



Oxivir[®] Tb Wipes

Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

A ready-to-use pre-wetted disinfectant cleaner wipe based on proprietary AHP[®] technology to deliver fast, effective cleaning performance.

Features & Benefits

- One-step cleaner disinfectant powered by AHP[®] technology
- Effective against key pathogens – TB, MRSA, Norovirus, VRE and *Candida auris*
- Kills HBV, HCV and HIV in 1 minute. Tuberculocidal in 5 minutes and Fungicidal in 3 minutes and 5 minutes for *Candida auris*.
- No VOCs, no added fragrance and no NPEs. Environmentally responsible – active ingredient breaks down to oxygen and water
- Compatible with most hard, non-porous surfaces
- Convenient – pre-wetted wipes make even the toughest job easier

Applications

- Clean and disinfect high-touch surfaces in just ONE Minute
- Non-irritating to eyes and skin





Oxivir® Tb Wipes

Virucide, Bactericide, Tuberculocide, Fungicide, Sanitizer

Use Instructions







As a One-Step Cleaner Disinfectant

- Pre-clean heavily soiled areas.
- Apply product by coarse trigger spray or disposable wipe to hard, non-porous inanimate surfaces.
- All surfaces must remain visibly wet for 1 minute.
- Use 5-minute contact time for TB and a 3 minute contact for fungi and 5 minute contact time for *Candida auris*. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Note: Rinsing is not necessary unless floors are to be coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

Technical data	Oxivir® Tb Wipes
EPA Reg. No.	70627-60 (Canisters, Buckets, Refills) 74559-3 (Soft Packs)
Color/Form	Colorless, wipe
pH	2.2 - 3.2
Scent	Characteristic of ingredients
Shelf Life	3 years

Product	Pack size	Dilution	Product code
Oxivir® Tb Wipes	12 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-use	4599516 
Oxivir® Tb Wipes	4 x 160 count 6 x 7 in. Wipes / 15.24 x 17.78 cm Canisters	Ready-to-use	101105152 
Oxivir® Tb Wipes	12 x 60 count 7 x 8 in. Wipes / 17.8 x 20.3 cm Canisters	Ready-to-use	5388471 
Oxivir® Tb Wipes	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Buckets	Ready-to-use	5627427 
Oxivir® Tb Wipes	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Refills	Ready-to-use	100823906 
Oxivir® Tb Wipes	120 X 8 count 7.48 x 10 in. Wipes / 18.99 x 25.4 cm Soft Packs	Ready-to-use	100949406 

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

AHP® and Design are trademarks of Diversey, Inc.