



# Avert<sup>®</sup> Disinfectant Cleaner Wipes

A ready-to-use, pre-wetted, one-step disinfectant cleaner wipe, that also kills *Clostridium difficile* (*C. diff*) spores on hard, non-porous surfaces such as those found in hospitals and other healthcare facilities.

## Features & Benefits

- Effective in 4 minutes against *Clostridium difficile* (*C. diff*) spores. Cleans, disinfects, and deodorizes in one labor-saving step
- Bleach-based formula means no quat-binding concerns. 1 minute contact time effective against a broad spectrum of pathogens including bacteria, antibiotic resistant bacteria, viruses, and fungi
- 12,500 ppm chlorine exceeds Health Canada, CDC, SHEA, IPAC and AORN recommendations

## Applications

- Hard surface cleaning and disinfection
- Designed for disinfecting medical hard, nonporous inanimate surfaces including: glass, glazed porcelain and stainless steel. Not recommended for use on: aluminum, brass, chipped enamel, clear plastic, copper, finished floors, natural marble, painted surfaces, and silver.

Note: This product is compatible with the listed materials. If product is intended to be used on other surfaces, it is recommended to test product by applying to a small discrete area to determine compatibility before proceeding with its use or contact a Diversey representative





# Avert® Disinfectant Cleaner Wipes

## Use instructions

### Special Instructions for Cleaning Prior to Disinfection against *Clostridium difficile* spores:

#### Personal Protection:

- Wear appropriate barrier protection such as gloves, gowns, masks or eye covering.

#### Cleaning Procedure:

- Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with this product. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths.

#### Infectious Materials Disposal:

- Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

#### Disinfection Procedure:

- Wipe hard, nonporous inanimate surfaces with towelette until thoroughly wet. Surfaces must remain wet for 4 minutes. Wipe surfaces and let air dry.

\*Rinse with potable water as necessary.

#### For Use as a One-Step Cleaner/Disinfectant, Deodorizer, and Fungicide:

1. Pre-clean heavily soiled areas.
2. Wipe hard, nonporous inanimate surfaces with towelette.
3. All surfaces must remain visibly wet for 1 minute.
4. Wipe surfaces and let air dry.

\*Rinse with potable water as necessary.

Technical data	Avert® Disinfectant Cleaner Wipes
DIN	02438526
Color/Form	Pale Yellow, Liquid premoistened towelette
pH	12.5
Scent	Chlorine
Shelf Life	12 months

Product	Pack size	Dilution	Product code
Avert® Disinfectant Cleaner Wipes	4 x 160 count 11 x 12 in. Wipes / 27.9 x 30.5 cm Buckets	Ready-to-use	100895933



## Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.