



INSTRUCTIONS AND SAFETY PRECAUTIONS

WARNING: To minimize the risk of injury from improper and unsafe use of this cane, read these instructions and safety precautions before using. Also, go to HurryCane.com and view the tips and tricks videos.


Intended Use and Limitations:

- This cane is a walking aid and is not intended to absorb the full weight of the user.
- Not to be used by persons heavier than 350 lbs (159 kg).
- Not intended for use on ice or other slippery surfaces as it has the potential to slip.


Safety Precautions:

- Consult a physician before using any assisted mobility device.
- Do not use unless push button is fully engaged.
- If the bottom base becomes loose, stop using cane immediately and contact HurryCane customer service.

SETUP INSTRUCTIONS

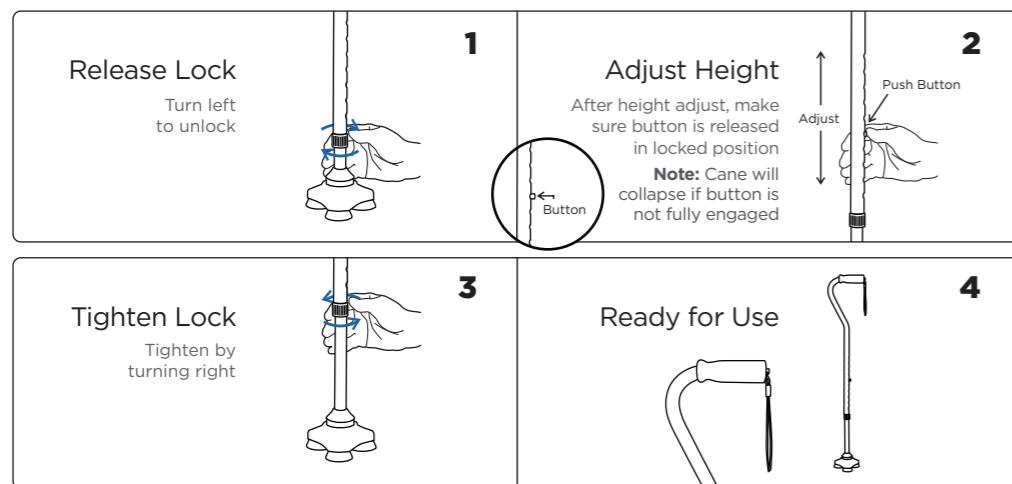

FEET CHECK INSTRUCTIONS

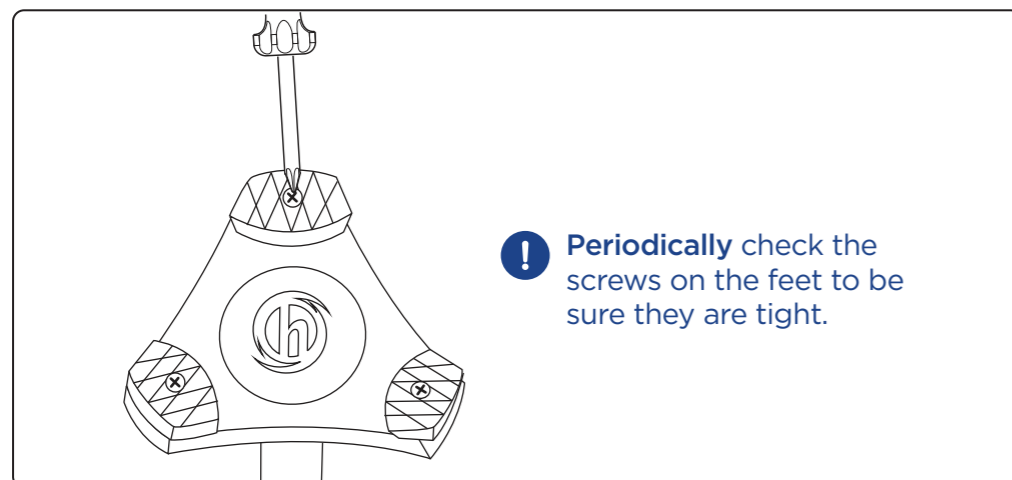

MAINTENANCE

- Periodically check the screws on the feet to be sure they are tight.
- Replace the feet every six months. See HurryCane.com to purchase new feet.
- Enhance your mobility at HurryCane.com for official HurryCane accessories.



Drive PreserveTech™ products are manufactured with an antimicrobial* agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated surfaces. Treated surfaces on this product include the handgrip.

*Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms.

SPECIFICATIONS

- FDA-listed
- Eight adjustable heights:
From 29 ½" to 38 ½"

Comments, Feedback, or Questions

Toll-Free Number: 1-800-933-1136
 E-mail: success@hurrycane.com
HurryCane.com
 Like us on Facebook: facebook.com/AllTerrainCane

RESET THE FLEX

When you want the cane to stand beside you without holding on to it, simply 'reset the flex' by following these 3 easy steps:

1 Pivot



Keeping the base firmly on the ground, move handle in a full circle.

2 Center



Move the handle directly above the base.



Carefully let go.

NOTE: If the cane falls over, you can raise it easily by putting your foot on the base and pressing down.

SURFACE TIPS

Although the HurryCane® is famous for providing superior stability on any terrain, it won't stand alone on uneven surfaces like these:



Plush Carpet



Throw Rugs



Uneven Wood Floors



©2022 HurryWorks LLC
 99 Seaview Boulevard | Port Washington, NY 11050
HurryCane.com | Made in China
 VER.C.6.22



Osmunda Medical Technology Service GmbH
 Treskowallee 108,
 10318 Berlin, Germany