

## INSTRUCTIONS *continued*

### Anti-tipper

Push the button clip down and adjust the anti-tipper to the desired length. Make sure the button is locked in place (you will hear a click as it locks).



### Adjusting Front Caster Position

Use two wrenches to unscrew the axle bolt holding the caster. Reposition the caster and re-secure it with the bolt.



### Extending the Seat Depth (from 16" to 18")

1. Remove both screws on both sides of the tubing underneath the front of the seat upholstery.
2. Remove the tube caps from either side.
3. Insert the seat extension. Secure the seat extension with the screws removed in Step 1.
4. Close the tube by adding the cap.



## MAINTENANCE

### GENERAL CARE

Wipe all chrome parts with a clean, soft cloth at least once a week. If exposed to moisture, dry immediately. Do not use an abrasive cleanser, as this will scratch the finish.

### TIRES

Replace tires if they become severely worn or cracking appears.

### FRONT CASTERS

Castors should be checked periodically and lubricated occasionally. Lubricate the casters by removing the fork and greasing the caster

## MAINTENANCE *continued*

stem bearings. Lubricate the wheel bearings by removing the wheel from the fork. Reassemble the fork and wheel by tightening the nuts until no play remains but wheels roll freely. If the nut is too loose, the caster will shimmy, if too tight, the chair will be hard to steer.

### WHEELS

Large rear wheels should be inspected at least once a month to ensure proper tightness. If a wheel wobbles or is loose, release the lock nut and tighten the axle bolt slightly. Test for proper tightness by leaning the wheelchair to one side and spinning the adjusted wheel.

## LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

\*This product meets or exceeds all federal safety and environmental regulations.



**SYMBOL GLOSSARY**  
For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

### WARNING:

Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

**Manufactured for: Dynarex Corporation**  
10 Glenshaw Street • Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com) • Made in China

## User's Manual

Reorder No. 10280



# DynaRide™ Series 5 Ultra Lite 18" Wheelchair



R220829

## WARNINGS

**Do not adjust or operate Wheelchair without carefully reading and understanding all instructions in this manual.**

- Be sure that attendants are fully knowledgeable regarding proper assisting procedures. Consult your physician, nurse, or physical therapist to determine the safest and most effective method of operation for your individual needs.
- This Wheelchair should not be used as a seat in any type of motor vehicle.
- Do not operate this wheelchair on any roadway or street or on any surface with vehicular traffic.
- Do not operate on inclines without proper assistance.
- When getting in and out of wheelchair or when in elevator or wheelchair lift, wheel locks must be fully engaged.
- Wheelchair should be on a smooth, stable and level area with wheel locks engaged before transferring user in or out of wheelchair.

### USING WHEEL LOCKS

The wheel locks should be engaged against both rear wheels before sitting down or getting up from the chair.

### STABILITY

DO NOT move forward on the seat while leaning forward out of the chair. If absolutely necessary, make sure the casters are in the forward position. This creates a longer wheelbase and gives the wheelchair more stability. Never reach further forward than the length of your arm, as this can cause tipping.

### SAFE TRANSFER

It is recommended that you work with a qualified healthcare professional before attempting to transfer on your own. Ensure that the wheelchair is on a smooth, stable, and level surface with wheel locks engaged before transferring. Position the wheelchair as close as possible to the seat you are transferring to. Rotate the front casters until they face fully forward. Ensure that armrests are out of the way and remove or swing footrests out of the way if possible. Do not stand on or apply weight to the footrest when getting up, as this can cause tipping.

### RAMPS AND SLOPES

Be certain of your physical capabilities and limitations before attempting to propel the wheelchair up a ramp. It is recommended that you consult with a qualified professional to learn the proper techniques for ramp negotiation before attempting to do so on your own.

Do not use the chair on an incline steeper than 10% without assistance. Never turn or change direction on a slope and always stay in the middle of the ramp. If it is necessary to stop while on a slope, avoid abrupt or sudden movements when you resume moving. Never use the rear wheel locks as breaks to slow or stop your chair, as this could cause the wheelchair to veer or tip to one side.

## INSTRUCTIONS

### Folding and Unfolding Wheelchair

To fold the wheelchair, tilt the wheelchair and lift the front of the seat rail up. To unfold/open the wheelchair, tilt the wheelchair and push the seat rail down. Be sure your hands are on top of the wheelchair seat, not in between the seat rail and chair frame.




### Installing the Back Rest Assembly

Unfold the wheelchair (as described above). Insert the back upholstery tubes into the sockets on either side of the wheelchair. Locate the height adjustment holes on the vertical tube of the lower frame. Align the upholstery tubes with the height adjustment holes at the desired height. On each side of the wheelchair, insert a bolt from the side through the mounting towards the wheelchair and secure it with a turning knob.



### Installing Leg Rests

With the footplate pointing to the side of the wheelchair, align the two mounting holes on the leg rest with the two pins on the wheelchair frame and insert. Swing the leg rest towards the wheelchair until it is locked in place.

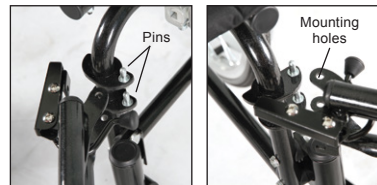

## INSTRUCTIONS *continued*

### Locking and Unlocking the Wheels

**To Lock:** Push the locking lever forward until it locks.

**To Unlock:** Pull the handle back to release the wheel locks.



### Swinging Back Armrests

To swing back the armrest, press the lever to release the lock and swing the armrest back.



### Extending Leg Rests

Release the lever that locks the inner leg rest tube. Push the button clip down and adjust the inner tube to the desired length.



### Quick-Release Wheel

Press the wheel axle inward and pull the wheel out at the same time.

