



CLEANPatch[®] Mattress repair system

A medical surface repair patch for hospital beds and stretchers that restores damaged mattresses to an intact and hygienic state.

Features & Benefits

- Restores damaged mattresses to an intact and hygienic state
- Made of latex-free, biocompatible materials. Extends the life of mattresses and stretchers and reduces the risk of cross contamination from damaged mattresses
- Peel and stick application
- Provides savings from equipment repair vs. premature replacement
- Proactively improves patient safety
- Fully cleanable with most hospital disinfectants

Applications

- Hospital bed mattresses
- Stretcher mattresses
- Operating room table pads





Use Instructions

- Assess the damage to determine if appropriate
- Measure the damage to determine which size of CleanPatch® to use
- Clean mattress as per hospital protocol. Let it dry.
- Remove the white backing from CleanPatch®
- Hold edges of the patch and position it over the damage
- Apply firm pressure to the patch, especially around the edges
- Slowly remove the clear top liner, pulling from the arrow toward the dot
- Smooth CleanPatch® from the center outwards. For best results wait 1 minute.
- Clean the patch and surrounding area per hospital protocol

Technical data	Clean Patch
Regulatory/Compliance	Health Canada (MDEL); FDA (Class I); CE
Color/Form	Dark blue material
Shelf Life	2 Years

Product	Shape Damage Size*	Shape	Quantity	Product code	
Clean Patch®	2 in. (5 cm) Up to 1/2 in. (1.3 cm)	Small Round	Box of 30	11001	
Clean Patch®	3.5 x 3.5 in. (9 x 9 cm) Up to 2 in. (5 cm)	Medium Square	Box of 20	10001	
Clean Patch®	3.5 x 6 in. (9 x 15 cm) Up to 4 in. (10 cm)	Large Rectangle	Box of 10	12001	

*There must be at least 1/2 inch margin around the damage to ensure proper adhesion

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.