


Reorder No. 10376




# *Drop-Arm Commode*




# COMMUNE ASSEMBLY

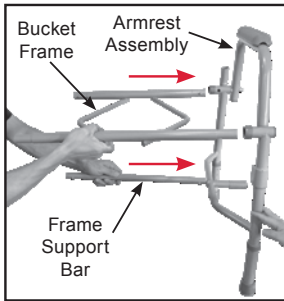

1. Attach the Bucket Frame to the Armrest Assembly by depressing the brass push button on the Bucket Frame and sliding it into the corresponding hole in the Armrest Assembly. Ensure the push button is fully engaged through the hole. Make sure the back of the Bucket Frame (the back of the Bucket Frame has the barcode sticker) corresponds to the back of the Armrest Assembly (the back of the Armrest Assembly has the back rest receptacle).
2. Attach the Frame Support Bar to the Armrest Assembly by depressing the brass push button on the Frame Support Bar and sliding it into the corresponding hole in the Armrest Assembly. Ensure the push button is fully engaged through the hole.

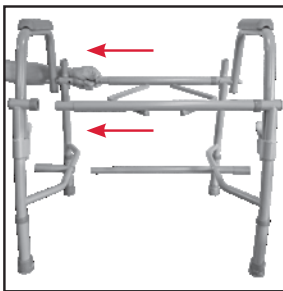

3. Repeat steps 1 and 2 to attach the Bucket Frame and Frame Support Bar to the other Armrest Assembly.

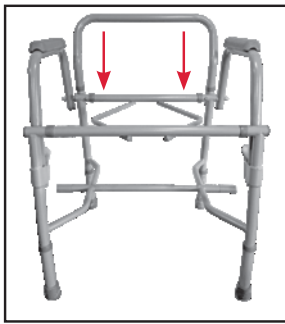

4. Attach the Backrest to the Backrest Receptacles in the Armrest Assembly by depressing the brass push button on the Backrest and sliding it into the corresponding hole in the Backrest receptacles. Ensure the push button is fully engaged through the hole.



5. Align the Toilet Seat to the opening in the Bucket Frame and the Toilet Seat clamps to the back rail of the Bucket Frame. Push down on Toilet Seat until clamps snap firmly into place. Repeat for the lid.



6. Insert Commode Bucket into the brackets of the Bucket Frame.

## ADJUSTING LEG HEIGHT

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. When desired height is reached, lock legs into place by aligning buttons into proper holes.
2. Make sure legs are locked into place by pushing down on or pulling legs.

## INSTRUCTIONS

1. To operate the drop arm, push the lever forward to release and slowly lower the arm so it rests against the bar at the bottom of the frame. To return the arm rest to the upright position, lift the arm up until the lever latch clicks into place.  
**CAUTION:** Do not lean or push down on the arm rest until the latch has been checked to insure the latch is in place and will hold weight.
2. Arms should be unlatched and dropped to assist in transfer onto/off the device.
3. To remove the toilet seat and lid, first remove the lid by grasping the lid near the clamps and pulling away from the back rail of the Bucket Frame. Repeat for the seat.

# WARNINGS

1. DO NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
2. Inspect all parts for shipping damage before assembly. If there is shipping damage—DO NOT USE. Contact dealer/carrier for further instructions.
3. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
4. Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Check the rubber tips on the leg extensions for rips, wear, or if they are missing. Immediately replace any or all if any of these imperfections exist.
7. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
8. Users with limited physical strength should be supervised or assisted while using commode.
9. Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.

# LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

 **WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)



**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

 **Manufactured for: Dynarex Corporation**  
**10 Glenshaw Street • Orangeburg, NY 10962**  
**USA • [www.dynarex.com](http://www.dynarex.com)**

Made in China

R230126