



Percept™/MC

60

Concentrated General Virucide Disinfectant Cleaner

An effective low-level disinfectant for non-critical medical devices and hard, non-porous, environmental surfaces.

Features & Benefits

- Formulated with patented AHP® technology containing no phosphates, quats or phenols. Fragrance-free, Volatile Organic Compound (VOC) free, Dye-free
- Kills Polio Virus, HIV-1, MRSA and VRE in 5 minutes. Non-corrosive and non-irritating to skin and eyes at use dilution as per OECD criteria
- Not manufactured using APE's/NPE's. Harnesses the cleaning power of hydrogen peroxide

Applications

- Exceptional cleaning on stainless steel, porcelain, chrome and other common surfaces
- Ideal for use in hospitals, long-term care facilities, schools and daycares





Use Instructions

Disinfection of Non-Critical Medical Devices & Nonporous, Hard Surfaces

- Pre-clean heavily soiled areas
- Dilute 1:16. Apply to surface with cloth or disposable wipe. Ensure surface remains wet for 5 minutes at 20°C. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Food contact surfaces require rinsing with potable water

Cleaning & Disinfecting Surfaces of Blood and Body Fluids

- Wear gloves and remove excess blood and fluid with absorbent materials
- To clean contaminated area, apply a 1:16 solution to the surface. Soak for 30 seconds and wipe dry.
- To disinfect contaminated area, apply a 1:16 solution to the surface. Ensure surface remains wet for five minutes. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.

Broad-Spectrum Sanitizing

- Dilute to 1:64 or 1:128 and apply to surface, allow surface to remain wet for 30 seconds. Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary.
- Food contact surfaces require rinsing with potable water.

Cleaning & Sanitizing Critical Care Floors

- Dilute 1:64 or 1:128. Use mop or autoscrubber and allow to air dry. No rinsing required.

Technical data	Percept™/MC
DIN	02241255
Color/Form	Colorless, liquid
pH	0.7 (Concentrate) 1.8 (Use Dilution 1:16)
Scent	Characteristic of ingredients
Shelf Life	1 Year

Product	Pack size	Dilution	Product code
Percept™/MC	2 x 1.5 gal. / 5.67 L Command Center™/MC	1:16	4337151 

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.

AHP and Design® are registered trademarks of Virox Technologies, Inc. used under license.