AquaChair Bathing System  
with Bidet

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
Item # RTL12A004-WH

**Scan for easy 3D instructions**

Download the BILT app before you begin

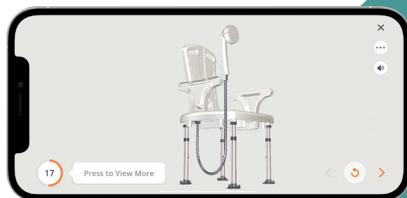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**drive**

## ⚠ warnings ⚠

- READ ALL WARNINGS AND INSTRUCTIONS BEFORE INSTALLATION OR USE - FAILURE TO DO SO COULD RESULT IN INJURY OR PROPERTY DAMAGE.
- Be careful to test water pressure before each use of the product, to ensure prevention of discomfort or injury from water pressure that is too high.
- Observe all local plumbing, electrical and building codes when installing or maintaining the product.
- Weight Capacity: 350 LBS / 158 KG

## included parts

- |                                 |   |
|---------------------------------|---|
| A. Seat                         | G. Control Panel Cover                      |
| B. Legs (4)                     | H. Adapter                                  |
| C. C Clips (4)                  | I. Flexible Steel Hose, 9ft                 |
| D. Backrest                     | J. Shower Spray w. Flexible Steel Hose, 80" |
| E. Armrests (2)                 | K. Commode Opening Cover                    |
| F. Bidet Control Panel Assembly | L. Commode Bucket & Cover                   |

# installation instructions

## STEP 1 - CHAIR ASSEMBLY

- Turn seat (A) upside down and insert legs (B) into leg receptacles. Secure legs using the c-clips (C) (Figure 1).
- Turn seat right side up and insert the backrest (D) into the receptacles located on the back of the seat, it will click into place when secure (Figure 2).
- Insert the armrests (E) into the receptacles located on each side of the chair, they will click into place when secure.

### Adjusting leg height

- Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height (denoted by the numbers on each leg).
- When desired height is reached, lock legs into place by depressing push buttons into proper holes (Figure 3).

## STEP 2 - BIDET ASSEMBLY




- Insert the bidet control panel assembly (F) on the desired side of the seat per illustration and attach the bidet assembly to the bidet house (Figure 4).
- Insert the control panel cover (G) on the other side (Figure 4).

### NOTE:

- When using as a commode, bidet assembly must be moved to the left or right-side house (Figure 5). Commode bucket and bidet assembly cannot be used simultaneously.
- Bidet control panel assembly is reversible for left or right-handed users

### NOTE:

- Check for secure fit routinely before using product.
- User's weight should be centered over seat.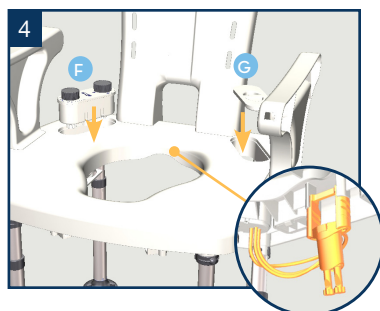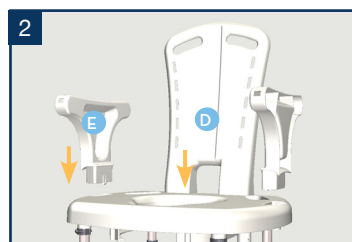
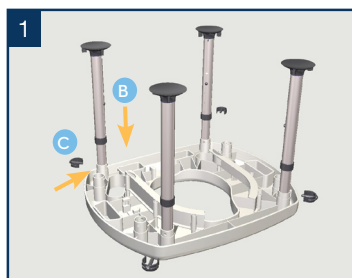


### STEP 3 - WATER CONNECTION

- Disconnect the existing shower head connection
- Ensure o-ring is placed in the adapter (H) and attach the adapter to the shower water connection in the wall.
- Screw the flexible steel hose (I) into the side connection of the adapter (H).
- Re-attach your existing shower head to the bottom connection of the adapter (H) (Figure 6).
- Attach flexible steel hose, 9ft (I) to the water port underneath the bidet control panel (Figure 7).
- Attach shower spray w. flexible steel hose (J) to the water port underneath the bidet control panel (Figure 7).
- Affix shower spray w. flexible steel hose (J) onto the shower spray holder on the armrest.
- Place commode opening cover (K) on the seat (Figure 8).

#### TIPS

- You may need a wrench to disconnect the existing shower head connection
- Wrap teflon tape tightly around threads to create a watertight seal
- Tighten by hand to avoid stripping the connection threads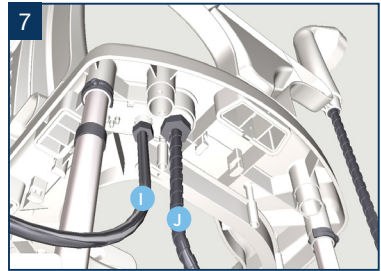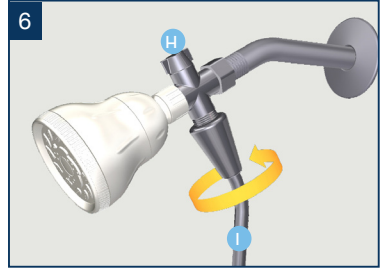


### STEP 4 - FINAL CHECK

- Check connections are properly secured
- Turn on the water and test each function (see control panel guide)

## control panel guide

ITEM	DESCRIPTION
A	Shower spray function control knob
B	Bidet function control knob
C	Female cleansing mode
D	Rear cleansing mode
E	Nozzle cleansing mode

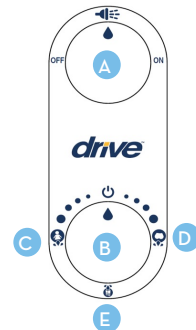## optional accessories

- An optional transfer assembly piece (RTL12A004-TR) is available for purchase.
- When used as a transfer bench this item is reversible to coordinate with your bathtub/shower environment.

## precautions

- Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- Initial temperature of water release is contingent upon ambient temperature of surroundings and will warm based upon your current water setting and supply.
- Always be sure that the seat is correctly and securely locked in place before using.
- This product is designed for use with most common types of showers and bathtubs. It will not fit all shower and bathtub types; be sure that the unit is correctly installed, fits properly and is stable before use.
- Make certain that the height adjusting snap buttons fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
- When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.
- DO NOT stand on chair.

## cleaning instructions

- Use a non-abrasive detergent or household cleaner with warm water.
- Rinse with warm water and wipe dry.
- DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may occur.

## maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit [www.drivemedical.com/help](http://www.drivemedical.com/help) for more info.

Drive PreserveTech™ products are manufactured with an antimicrobial\* agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated surfaces.

\*Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms

## one year limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship under normal use for (1) one year from date of purchase for the original consumer purchaser.

This device was built to exact standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

Drive's obligations under this warranty are limited to replacement, at Drive's option, of products or parts found to be defective, provided that such products were installed and used in accordance with instructions. Drive reserves the right to make such inspections as may be necessary to determine the cause of the defect.

This warranty does NOT apply to:

- Damage or loss which occurs during shipment.
- Damage or loss sustained through any natural or man-made causes beyond the control of Drive, including but not limited to fire, earthquake, floods, etc.
- Damage or loss resulting from sediments or foreign matter contained in the water system.
- Damage or loss resulting from negligent or improper installation including installation of a unit in a harsh or hazardous environment.
- Damage or loss resulting from removal, improper repair, modification of the product, or improper maintenance including damage caused by cleaning products.
- Damages due to normal "wear and tear" will not be considered defects.
- If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2022 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA

Made in China

 **WARNING:** This product can expose you to chemicals, including Styrene, which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

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