



Hospital's
Choice® 

Ready for Baby Deluxe Nursery Kit

IH417

Please be sure to read all warnings, instructions and notations before using this product. Please keep these instructions as they contain important information.

Fabric Storage Box

To assemble box, straighten all four sides of the box and put rigid base board into bottom, fabric side up. Surface wash only.



Bottle Medicine Dispenser

The Bottle Medicine Dispenser has easy-to-read dosage markings and measures up to 3 tsp. (15 ml).

To Use:

- 1 Remove top from measuring cup and fill cup to the prescribed dosage.
- 2 Snap top securely onto the measuring cup.
- 3 Bottle-feed child as normal, pausing as necessary so that child may swallow comfortably.



To Clean: Wash before first use and after every use. Disassemble top ring, nipple and measuring cup. Wash with warm, soapy water and rinse thoroughly. Alternatively, it is dishwasher safe. TOP RACK ONLY.

⚠ WARNING: Inspect bottle nipple regularly for tears, or other signs of wear or cracks and discard any worn or damaged nipple immediately. Made without BPA.

Medicine Dropper

The convenient spill guard plug helps reduce messy spills at home or away.

To Use:

- 1 Remove spill guard plug from dropper tip. Squeeze bulb to force air out of dropper.
- 2 Insert into medicine bottle. Slowly release pressure on bulb until medicine reaches the prescribed dosage marking.
- 3 Squeeze bulb slowly to help release liquid.



To Clean:

Wash with warm, soapy water before first use and after each use. Rinse thoroughly.

⚠ WARNING: Keep all medications out of child's reach. We recommend that you use a child resistant top to seal the medicine bottle and store it in a locked cabinet or other location that is not accessible to a child. Made without BPA.

Nasal Aspirator

To Use:

- 1 Place fingers as shown and thumb at bottom of aspirator (see photo), and squeeze air out of aspirator.
- 2 Hold baby upright and gently place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
- 3 Slowly release pressure on the bulb and the suction will draw excess mucus from baby's nasal passage. Repeat in other nostril.



To Clean: Aspirator is top-rack dishwasher safe or can be washed in warm, soapy water and rinsed. Clean thoroughly before first use and after each use. Make certain that all water is drained during drying. Avoid using abrasive pads or cleaners. **DO NOT** place in microwave.

⚠ WARNING: NEVER push the aspirator too far into the nostril and always wash and dry thoroughly after each use. NEVER use in baby's mouth or ears. When storing, keep away from heat and bright light.

Soothing Scrub Wet Brush

To Use:

- 1 Apply a small amount of baby oil, olive oil, or shampoo to baby's head.
- 2 Slide brush onto your hand and place bristles on baby's scalp. Apply gentle pressure, using a circular motion to massage the scalp and loosen the flakes.
- 3 Shampoo and continue to use brush to help remove flakes. Rinse and repeat as needed. NOTE: If condition persists contact your pediatrician. The soft bristles are also great for everyday bath and hair care.



To Clean: After each use, separate brush insert from handle and wash both parts in hot soapy water. Rinse and let drip dry. Both parts are also top rack dishwasher safe.

⚠ WARNING: These products are not toys. **DO NOT** allow children to play with them. When not in use, keep out of reach of children. **ALWAYS** supervise your child when using this kit. **NEVER** allow your child to use these products alone or without close adult supervision. We recommend that you store this kit in a locked cabinet or other location that is not accessible to a child. FOR INDOOR HOME USE ONLY.

Steady Grip Nail Clippers & 5 Emery Boards

These clippers have been specially designed with an extra large, easy-grip handle for convenient adult use.

To use:

- 1 Lift arm from hand grip and rotate it 180°.
- 2 Position nail between cutting edges and squeeze handle to cut nail.
- 3 Emery board use is recommended to smooth any remaining sharp nails after clipping and also to smooth minor rough or jagged fingernail edges at other times.



To clean: Wipe clippers with a damp cloth and dry after each use. Clippers may be sterilized by wiping with rubbing alcohol.

Note: It is best to trim nails after a bath when they are soft, or while baby is sleeping.

CAUTION: Be careful not to trim too close to baby's sensitive skin.

6 Alcohol Wipes

The Alcohol Wipes are provided to help keep thermometer clean and disinfected. Unwrap one alcohol swab from the packet and wipe thermometer clean after use, so that it is always ready to be used again. Discard swab after use.



FOR EXTERNAL USE ONLY.

5 Digital Thermometer Covers

NOTE: Use thermometer covers when taking temperature rectally; not for oral use (see Warning section).


1 Hold edge of cover and slip over end of thermometer. Pull the outside protective cover off. Make sure no air remains between the probe and the cover.

2 **Rectal Use:** Lubricate the thermometer cover with a water-soluble lubricant or petroleum jelly. Follow instructions under "Rectal Use." Discard thermometer cover after use. Do not reuse covers.

NOTE: To purchase thermometer covers call Consumer Relations at (800) 544-1108, and ask for part number 49520.

Feeding Tracker Bracelet

To Wear: Position bracelet on wrist so that letters and numbers appear right side up. Wrap around wrist and push plastic post on underside of bracelet into one of the holes on the other end of the bracelet. Bracelet should be snug enough to stay on wrist but loose enough to allow plastic sliders to move freely.

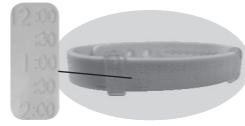

To Use:

Top slider- If you are breast feeding you may use the top slider to indicate whether your last feeding was from the left (L) or right (R) side. If you are not you can ignore the top slider, use it as an AM/PM indicator, or slide it down and use it as a second slider on the time-tracking bar instead.

Bottom slider- This slider is on a time-tracking bar containing times on a half hour interval. You can use this bracelet as a handy reminder of anything you need to track time for. Some examples might be: last feeding, next feeding, last time Mom drank water, next time Mom should drink water, duration/time baby's of last nap.

To Clean: Wash in warm, soapy water. Product is also top rack dishwasher safe in a basket.

Easy Grip Hair Brush & Comb Set

The Easy Grip Brush & Comb Set features large, soft-touch handles with a chunky design that is easy for parents or young children to grip. The hairbrush's soft bristles are extra gentle on baby's sensitive scalp, while the comb's dual-teeth design lets you adjust for the thickness or dampness of your child's hair.



To Clean: Wash in hot, soapy water. Rinse with warm clear water and let drip dry.

Easy Grip Toddler Toothbrush

The contoured, ergonomic handle of the Toddler Toothbrush makes it easy for small hands to hold and is roll-resistant. Place it on the table or sink and the convenient, roll-resistant handle keeps it from falling off.



To Use:

- 1 Begin use once all primary teeth are in or when recommended by your dentist.
- 2 Apply a pea-sized amount of toothpaste to the brush and have your child brush for 2-3 minutes using an up and down motion.
- 3 Continue to help and supervise your child during brushing – do not allow child to swallow toothpaste.

NOTE: Toothbrushes should be replaced every 2-3 months.


To Clean: Rinse each brush thoroughly before first use and after each use.

Important Note: Questions about your child's oral health should be answered by your family dentist.

CAUTION: Adult supervision required. Do not allow child to run or walk while carrying or using toothbrush. Do not let children play with toothbrushes.

Digital Thermometer with Protective Case

⚠ WARNING: BEFORE USING DIGITAL THERMOMETER, PLEASE READ CAREFULLY. • This is **NOT** an ear thermometer. **NEVER** put probe into ear. • Take temperature measurements Underarm, Orally or Rectally. • **NEVER** allow children to play with this product. It is not a toy. This is a sensitive electronic device and may be damaged if not used properly under parental supervision. • **DO NOT** expose the thermometer to extreme temperatures, humidity, direct sunlight or shock, and/or dust. • Avoid sudden impact and dust. • Keep battery out of child's reach. Swallowing a battery could be fatal. Battery should not be charged or placed into extreme heat as it may explode. • Used batteries should be disposed of properly. • **POTENTIAL CHOKING HAZARD.** Do not leave child unattended when a thermometer probe cover is in use. • Probe covers are not recommended for oral use. • Keep probe covers out of reach of children. • Probe covers are intended for single use only. • **ALWAYS** supervise your child when using thermometer and thermometer probe covers. **NEVER** allow your child to use it alone or without close parental supervision.

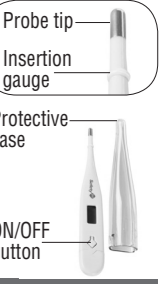

To Set Up Thermometer Before Use

Note: Ensure that thermometer is at room temperature, wait 15 minutes if needed.

1. **Set to Fahrenheit/ Celsius:** While Thermometer is off, press and hold the ON/OFF button until the unit of measure in the display screen switches. This should take approximately 5 seconds.
2. **Prepare for Use:** Press ON/OFF button. The last temperature measured will display with an **M** icon on the screen for a few seconds. When the **L** icon and measure of temperature appear, thermometer is ready to take temperature.

Battery Replacement

1. Indicator: When the "⏻" icon appears, the battery needs to be replaced.
 2. Use a Phillips head screwdriver to loosen battery cover screw. Remove cover.
 3. Use a toothpick to remove battery.
 4. Insert a new battery (LR41 or equivalent), correctly positioned with the positive "+" side facing up as shown in illustration.
 5. Place the battery cover on the thermometer and secure the cover.
- CAUTION:** Battery is to be inserted with the correct polarity. Check that all contact surfaces are clean and bright before installing battery.




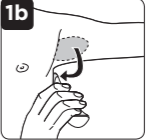
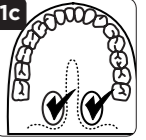
Important Information

• Read these instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors, including exercise, drinking hot or cold beverages prior to taking a reading, and technique. • **DO NOT** allow child to walk or run during temperature taking. Probe could present a hazard. • High, prolonged fever requires medical attention, especially for young children. Please contact your physician.

To Take Temperature

Please read instructions for each mode of taking temperature. **Note:** Ensure °F/°C is still flashing before taking temperature.

1. Choose Temperature Mode based on child's age:

Rectal-Birth & up	Underarm-under 4 years	Oral-4 years & up
<p>Note: Highly accurate for measuring core body temp. Approximately 60 seconds to measure.</p>  <ol style="list-style-type: none"> 1. Lubricate probe tip with a water-soluble lubricant or petroleum jelly. 2. Position baby so that the back and legs form a 90° angle or L-shape (Figure 1a). 3. Insert tip gently into the rectum, until you reach insertion gauge (ring on probe tip). Do not force the tip into the rectum if there is resistance. 	<p>Note: Ensure underarm is dry, wipe if needed. Approximately 90 seconds to measure.</p>  <p>Place probe tip at center of underarm (Figure 1b). Bring arm down against the body and ensure probe is touching skin.</p>	<p>Note: Approximately 60 seconds to measure.</p>  <p>Place probe tip under the tongue as indicated by ✓ (Figure 1c). Instruct child not to bite down on probe.</p>
<p>3 TEMPERATURE METHOD</p>	<p>NORMAL TEMPERATURE</p>	<p>FEVER RANGE</p>
Oral	97.0° - 99.6°F	99.6°F +
Rectal	97.3° - 100.4°F	100.4°F +
Underarm	95.4° - 98.6°F	98.6°F +

Care & Maintenance

- Clean probe tip before and after each use.
- Use an alcohol swab to wipe clean.

- Use a soft, dry cloth to clean thermometer body. • **DO** remove battery if stored for a long period of time. • **DO** store thermometer in a dry place at room temperature with the protective cap in place. • **DO** take special care not to scratch the display window. • **DO NOT** allow electronics to get wet. • **DO NOT** use abrasive cleaning agents, thinners or benzene for cleaning.
- **DO NOT** drop thermometer. • Install new battery and retry function before calling Consumer Relations on electronics.

One (1) Year Limited Warranty

Dorel Juvenile Group, Inc. warrants to the original purchaser that this product (Thermometer), is free from material and workmanship defects when used under normal conditions for a period of one (1) year from the date of purchase. Should the product contain defects in material or workmanship Dorel Juvenile Group, Inc., will repair or replace the product, at our option, free of charge. Purchaser will be responsible for all costs associated with packaging and shipping the product to Dorel Juvenile Group Consumer Relations Department at the address noted on the front of this document and all other freight or insurance costs associated with the return. Dorel Juvenile Group will bear the cost of shipping the repaired or replaced product to the purchaser. Product should be returned in its original package accompanied by a proof of purchase, either a sales receipt or other proof that the product is within the warranty period. This warranty is void if the owner repairs or modifies the product or the product has been damaged as a result of misuse. This warranty excludes any liability other than that expressly stated previously, including but not limited to, any incidental or consequential damages.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

THIS WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS THAT MAY VARY FROM STATE TO STATE.

©2017 Dorel Juvenile. (800) 544-1108 www.safety1st.com Styles and colors may vary. Los estilos y los colores pueden variar. Made in CHINA. Hecho en CHINA. Distributed by (distribuido por) Dorel Juvenile Group, Inc., 2525 State St., Columbus, IN 47201-7494. Dorel Juvenile Canada, 2855 Argentinia Road, Unit 4, Mississauga, ON, L5N 8G6 01/25/17 4358-7281 Importado y Distribuido en Chile por: Comercial e Industrial Silta S.A. - San Ignacio 0201 / Parque Industrial Portezuelo Quilicura, Santiago Chile. Fono (562) 2339 9000. En Perú por: Comexa S.A. - Los Libertadores 455 - San Isidro, Lima - Perú. Fono (511) 4469014. En Centro América, Caribe y Ecuador por: Best Brands Group S.A. - Av. Balboa, Bay Mall Plaza, Planta Baja, local 9, Panamá. - Fono: (507) 300 2884. En Colombia por: Baby Universe S.A.S. - Via 40 No. 77-29, Barranquilla, Colombia - Fono: (575) 353 1110 Importador: DJGM, S.A. DE C.V. Gabriel Mancera No. 1041 Col. Del Valle Delegación Benito Juárez C.P. 03100 México, D.F. R.F.C. DJG1403056N9 Tel. (55) 67199202



