## Nail Clippers IH2770500

### To Use:

1. Lift arm from hand grip and rotate 180°.
2. Position nail between cutting edges and squeeze handle to cut.

**To Clean:** Wipe with a damp cloth and dry after each use. Sterilize by wiping with rubbing alcohol.

## Nasal Aspirator

### To Use:

1. Squeeze air out of aspirator.
2. Hold baby upright and place tip into baby's nose so that it points partially up or down, but not directly toward the center of the nostril.
3. Slowly release pressure on the bulb for optimum suction.

### To Clean:

1. Clean thoroughly before first use and after each use.
2. Top-rack dishwasher safe or wash in warm soapy water and rinse thoroughly. Make certain that all of the water is drained during drying.
3. Avoid using abrasive pads or cleaners.
4. **DO NOT** place in microwave.

**CAUTION:** These products are not toys. Do not allow children to play with them. To avoid accidental injury, your child should never be left unattended. These products should only be used with adult supervision. When not in use, keep out of reach of children. **Nail clippers:** Be careful not to trim too close to baby's sensitive skin. **Nasal aspirator:** Never push the aspirator too far into the nostril and always wash and dry thoroughly after each use.

**NEVER** use in baby's mouth or ears. When storing, keep away from heat and bright light.

**FOR INDOOR HOME USE ONLY. MADE WITHOUT BPA.**

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