



Rescue™ Gel Sporicidal

Ready-to-Use sporicide for hard, non-porous surfaces that is effective against *Clostridium difficile*, *Clostridium sporogenes* and *Bacillus subtilis*, utilizing AHP® Hydrogen Peroxide technology.

Features & Benefits

- Ready-to-Use sporicide powered by AHP® technology
- Effective against *Clostridium difficile*, *Clostridium sporogenes*, *Bacillus subtilis* in 10 minutes, providing cleaning and disinfection on high touch surfaces
- The AHP® active breaks down to oxygen and water after use
- Designed to ensure bathroom surfaces (toilet bowls, sinks, commodes and basins) remain wet for the full contact time

Applications

- For use on toilet bowls, commodes, sinks and basins





Rescue™ Gel Sporicidal

Use Directions*

Heavily Soiled Surfaces Require Cleaning Prior to Disinfection

Personal Protection: Disposable gloves, gowns, face masks, or eye coverings as appropriate, must be worn during all cleaning of body fluids, and decontamination procedures.

Cleaning Surfaces Contaminated with Body Fluids: Appropriate Personal Protection must be worn during all cleaning and disinfection procedures. Remove excess body fluids with absorbent materials. Apply the gel to surface, let stand for 1 minute, thoroughly rinse. Disinfect contaminated area.

Sporicidal Disinfection of Hard Non-porous Surfaces: This product is NOT to be used as a terminal sterilant/High Level Disinfectant for critical instruments and medical devices. This product is intended for use as an Environmental Surface Sporicide on specific surfaces such as toilets, sinks, basins and commodes, etc.

For Toilets and Commodes:

- Apply the gel to the interior surfaces of the toilets or commode bowl (using the flip-top applicator).
- To achieve Sporicidal disinfection ensure surface remains wet for 10 minutes.
- Using a wet cloth, brush or sponge scrub the surfaces of the sink or commode, then flush or rinse.
- Rinse after disinfection to remove all residue.

Note: This product is not intended for use as a general environmental surface cleaner or disinfectant except on targeted surfaces specified on the label.

Disposal of Infectious Material: Products contaminated with blood or body fluids should be disposed of according to Federal, Provincial, and local regulations for infectious waste disposal.

* Refer to product label for complete use instructions.

Technical data	Rescue™ Gel Sporicidal
DIN	02447592
Color/Form	Colorless Gel
pH	2.7
Scent	Characteristic of ingredients
Shelf Life	2 Years

Product	Pack size	Dilution	Product code
Rescue™ Gel Sporicidal	12 x 946 mL / 1 U.S. Qt. Bottles	Ready-to-use	101103737 

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide® and Design are trademarks of Diversey, Inc.

www.diversey.com

© 2020 Diversey, Inc. All Rights Reserved. PSS90447 (20/056)