Airgo[®]

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2

EN CHECK ALL PARTS

Carefully examine all parts for shipping damage. DO NOT use the Rollator if there is apparent damage or if there any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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SAFETY PRECAUTIONS

EN

READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR ROLLATOR:

- Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
- Before each use, make sure that all parts are secure and that the frame is fully opened.
- Risk of fall and serious injury may result if brakes are not locked while the user attempts to sit down.
- Do not navigate independently while seated. Do not attempt to sit on the seat while on an incline. Do not lean when you are seated. Doing so may cause the walker to tip and a serious risk of injury may result.
- Falls are possible if you use the seat without installing the backrest.
- Do not use your rollator while folded. The stability of any rollator is compromised when its frame is folded, increasing the likelihood of a fall.
- Do not exceed the following weight limits:
Models 700-917, 700-920, 700-921, 700-922, 700-923, 700-924, 700-926, 700-927: **300 lb.**
Models 700-928, 700-929: **400 lb.**

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WALKING TIPS

EN

- Imagine a line drawn between the rear wheels. Part of one foot should be ahead that line as you walk.
- Perform simple stretches before walking. This will help you walk further and avoid injury.
- Bring a bottle of water to stay hydrated.
- Walk on soft ground that is more joint-friendly such as grass or dirt paths.
- Use the curb climber to help safely lift your rollator over obstacles, such as sidewalks.

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SPECIFICATIONS

Project: 911-70092X eXcursion Quickstart Rev8 0115
Item No.: 700-917, 920, 921, 922, 923, 924, 926, 927, 928, 929 P/N: 911-70092X
Flat Size: 70.6 x 59.8 cm
Assembled size: 64.8 x 55.9 cm (open) 21.6 x 27.9 cm (folded)
Colours: _B/W _XCMYK _Spots:
Special Instructions: This is a double-sided, folding quickstart.
Generated by:

911-70092X REV 8 0115

21.6 cm

21.6 cm

21.6 cm

27.9 cm

27.9 cm

EN UNFOLDING THE FRAME

Push down on the seat supports to expand and open the frame. The safety lock will click into place when the rollator is fully unfolded (A).

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EN INSERT THE HANDLEBARS

Ensure the brake cables are on the outside of the frame (A), and that they pass through the cable guide. For height adjustment, stand upright and set the handgrips even with your wrists (B). Hand tighten the triangular hand screws.

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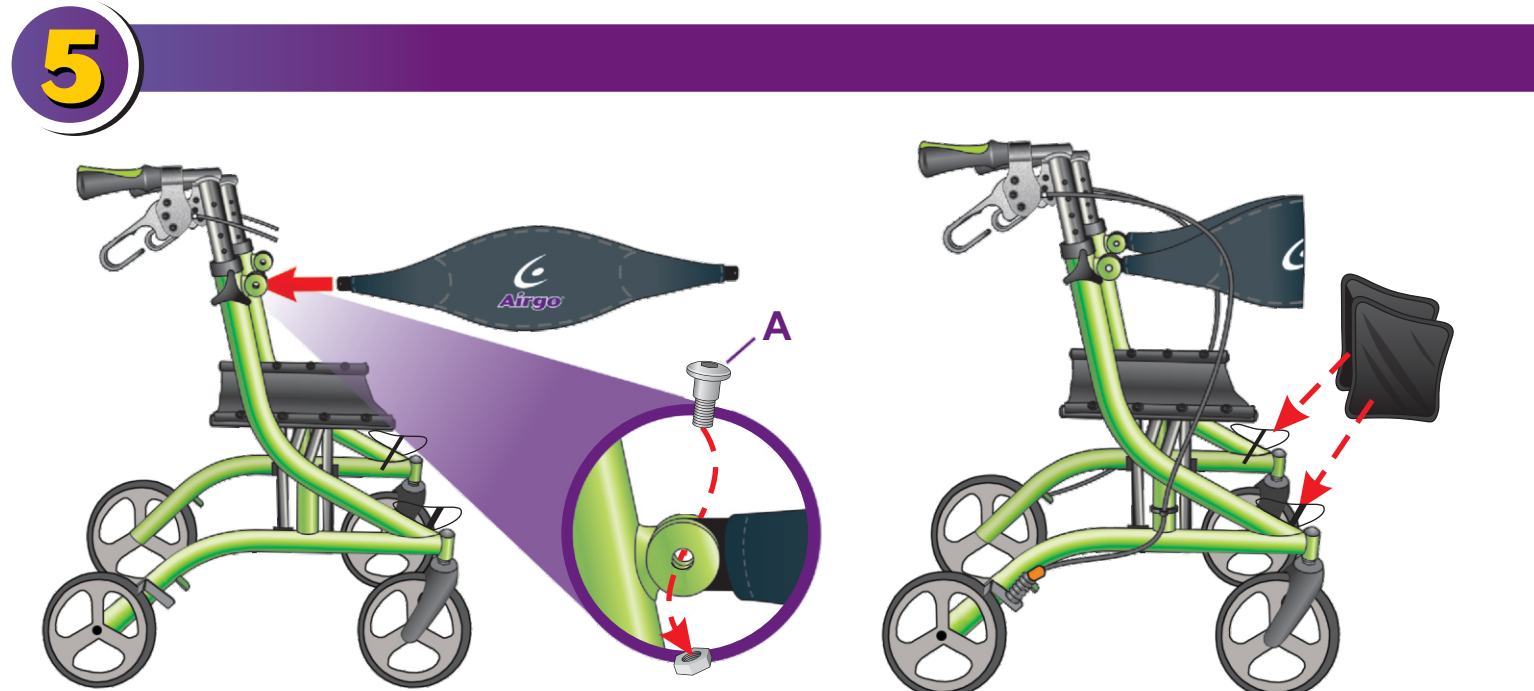
EN INSTALL THE BACKREST & THE BASKET

Insert the backrest so that the Airgo logo faces the front wheels. Slide one end of the backrest into the fan bracket. Insert the screw (A) from the inside of the unit out, and repeat. Install the basket by sliding it onto the mounts starting from the back, and then pulling to the front.

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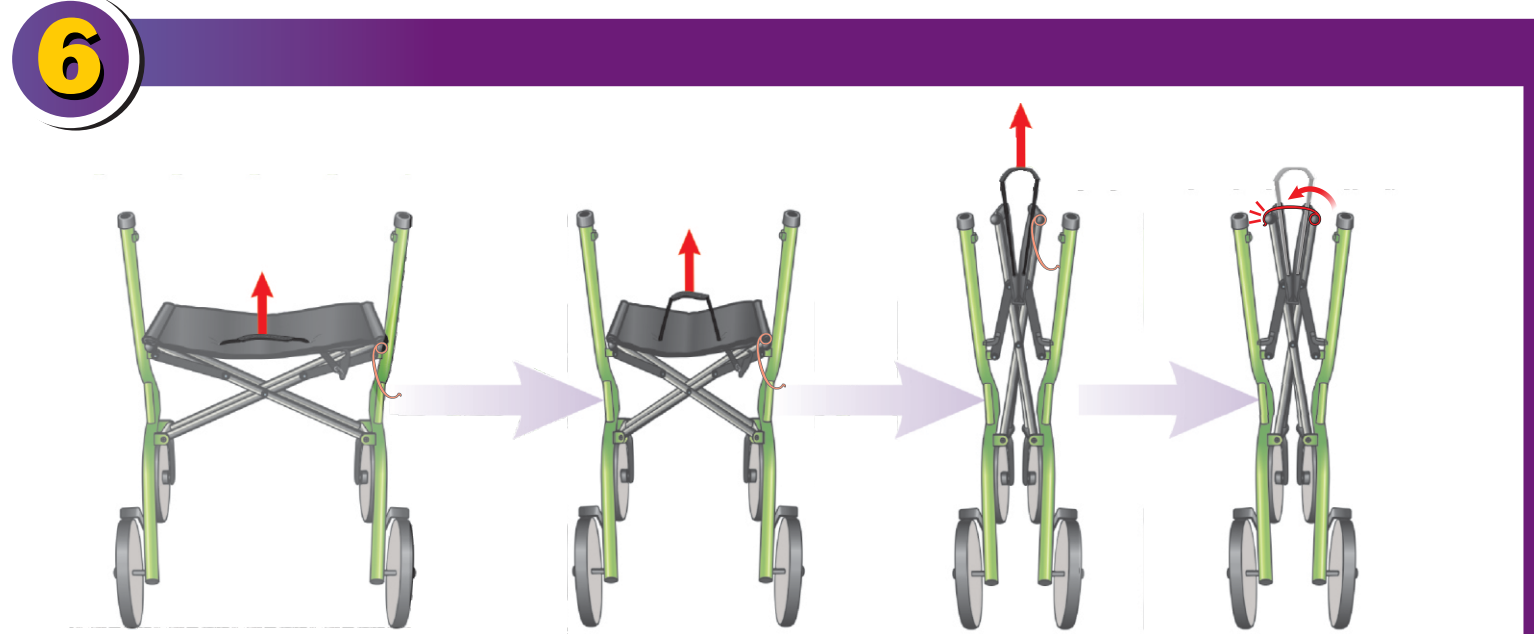
EN FOLDING

Pull the folding strap on the seat to release the safety lock and initiate folding. Use the folding clip to lock the rollator in the folded position.

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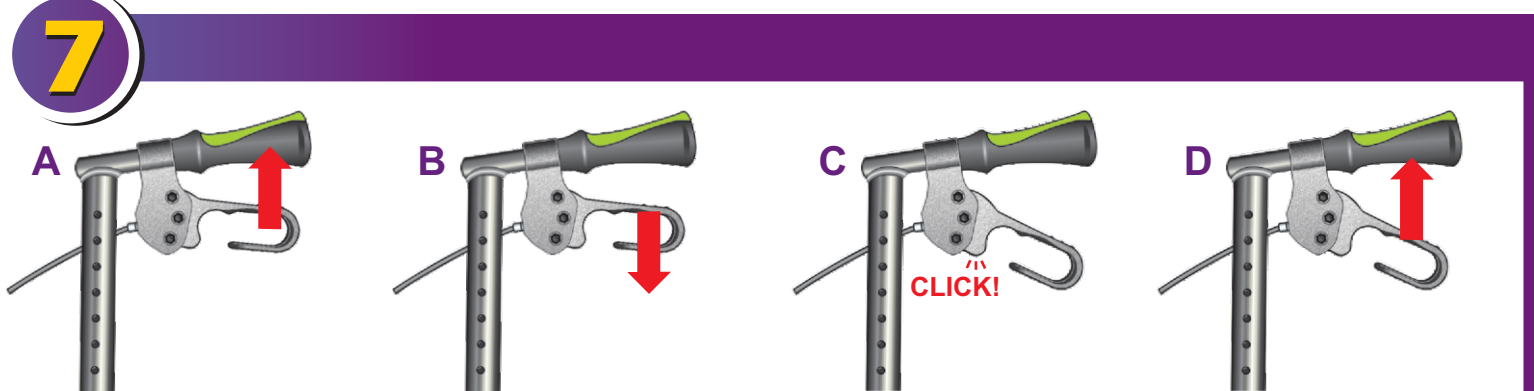
EN USING THE BRAKE HANDLES

Squeeze up to slow down (A). To lock, push down until they click (B-C). To release, pull back up (D). ALWAYS lock the brakes before using the seat.

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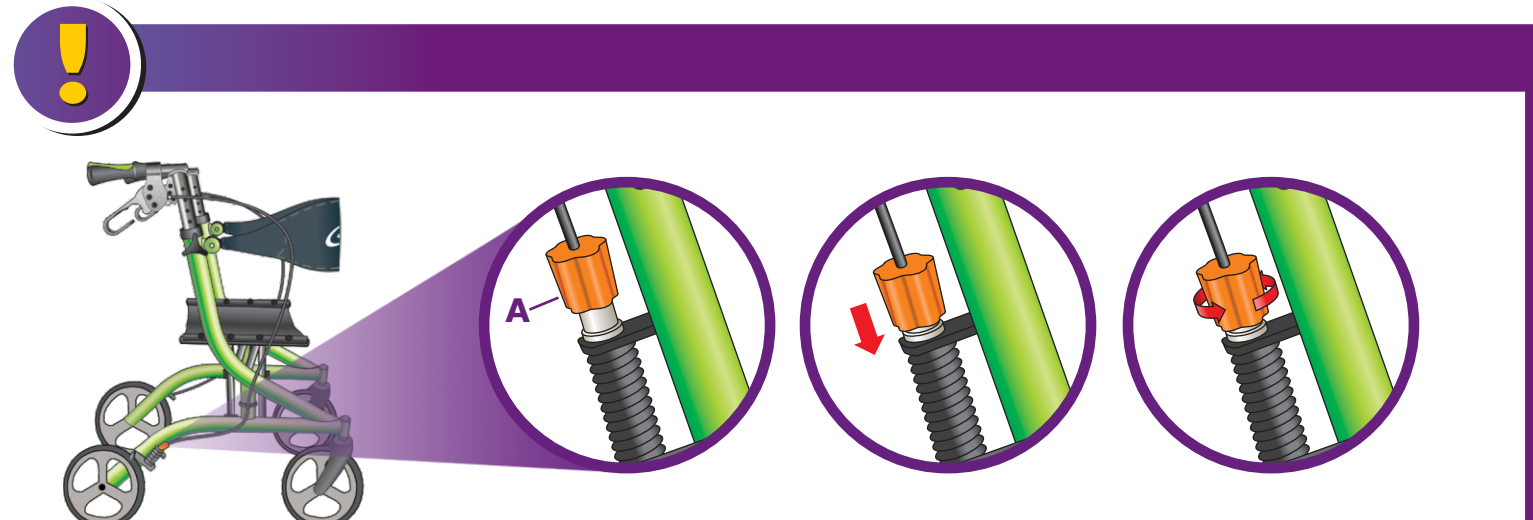
EN PERIODIC BRAKE ADJUSTMENT

Push down the adjustment knob (A) and rotate. Clockwise will loosen the brakes, counter-clockwise will tighten them. Your brakes must be checked periodically for safety. IMPORTANT: Before using your rollator again, ensure the brakes are working properly. DO NOT use if the wheels rotate while the brakes are locked. For help please call us at CAN: 866-752-1363, USA: 866-694-5085.

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EN CUP HOLDER

• A refreshment holder for your Airgo, to stay hydrated wherever you go!
• Swivels from left to right for maximum convenience.

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EN CANE HOLDER

• Allows you to conveniently store your cane on your rollator.
• Easy to use, simply strap your cane along with you.

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Made in China / Fabriqué en Chine

EN LIMITED LIFETIME WARRANTY

Drive Medical covers this Airgo® rollator by a Lifetime Warranty on the Frame and a Five (5) Year Warranty on Brake Handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. Drive Medical reserves the right to replace or repair any part that has become defective. For warranty service and general inquiries in Canada and the U.S., call our customer service team at CAN: 866-752-1363, USA: 866-694-5085. Provide the following information when you call: date of purchase, receipt and serial number (on crossbar underneath the seat). Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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Drive Medical LTD